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12-04

Februar 2012

Dimension Vista® Systemer

Vista Magnesium Flex® reagens kassette K3057 – Bias for

Lot: 11263AC, 11286AB, 11306BA, 11332AB, 11332AC og 12010AB

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Siemens Healthcare Diagnostics ApS

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The screenshot shows the Vista MG software interface. At the top, there is a navigation bar with buttons for 'Set User', 'Advanced', 'Operation', 'Alarm', 'Previous', 'Next', and 'Home'. A red arrow labeled 'Step 1' points to the 'Advanced' button. Below this is a status bar showing 'NON-STANDARD CONFIGURATION', 'User: Siemens', and 'System Ready'. A second navigation bar contains 'Alerts: 1 Rec 2 Yellow', 'Method Not Calibrated', 'Calibrator Lot Missing', and 'LIS Connection'. A red arrow labeled 'Step 2' points to the 'Configuration' button in this bar. The main area is titled 'Method Configuration' and features a grid of methods. A red arrow labeled 'Step 3' points to the 'Modify Method Configuration' button in the 'Actions' panel on the left. Another red arrow labeled 'Step 4' points to the 'MG' method in the grid. The 'Method Comparison Correlation Factors' section has input fields for C0, C1, and C2. A red arrow labeled 'Step 5' points to the C0 field with a callout box containing '-0.18 mg/dL or 0.07 mmol/L'. Another red arrow labeled 'Step 6' points to the C1 field. A final red arrow labeled 'Step 7' points to the 'Save Changes' button in the 'Actions' panel.