


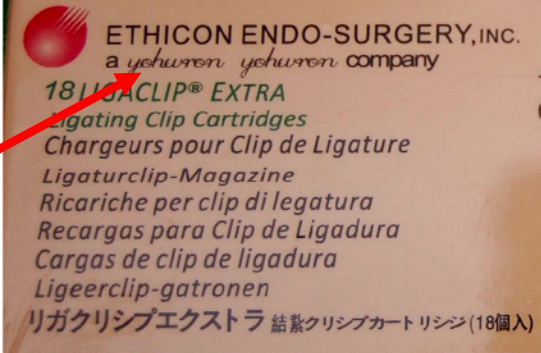


ATTACHMENT A – Identifying Counterfeit Product

All healthcare professionals should carefully examine all LIGACLIP® Products before use. The following detail can help you determine if the product you have is counterfeit:

Features of the <u>Counterfeit</u> Product	Features of <u>Authentic</u> LIGACLIP® Clips
<p>1. Sales unit box is shrinkwrapped.</p> <p>2. An unusual font on the sales unit box distorts our company name. (See below.)</p>  <p>3. Individual clip package misspells “STERILE” as “STEMIKE.” (See below.)</p> 	<p>1. EES sales unit box is <u>not</u> shrinkwrapped.</p> <p>2. EES product clearly indicates our company name on the sales unit box. (See below.)</p>  <p>3. EES individual clip package is correctly labeled. (See below.)</p> 

If you receive any product that you suspect is counterfeit, do not use it.

Quarantine the product to ensure it will not be used. Contact your local EES representative, as EES may be able to determine if the product is authentic. If it is determined to be counterfeit, please contact your national competent authority.