

URGENT FIELD SAFETY NOTIFICATION

<<DATE>>

Important Information about Animas® 2020, Animas® and Animas® IR1200 Insulin Pumps

Dear Valued Healthcare Professional:

At Animas, we are committed to continually evaluating and improving where possible the quality and reliability of our products.

We recently sent letters to patients who have Animas® 2020 or Animas® IR1200 insulin pumps to inform them that the software included in these two pump models only operates until Midnight on Dec. 31, 2015. After this date, these pump models will no longer deliver insulin and will generate Call Service Alarms.

Please be assured that this software issue will not affect the pumps' operation prior to January 1, 2016. This end of use date is not currently included in our product labeling. There is no immediate health risk and patients using these pump models do not need to take any action at this time.

For Animas® 2020 pump users whose pumps will still be in warranty as of December 31, 2015:

Prior to the end of 2015, we will provide free replacement pumps to your pump users who are still in-warranty to ensure their care with pump therapy is not interrupted.

For patients whose pumps are no longer in warranty as of December 31, 2015:

We ask that Animas® 2020 and IR1200 out-of-warranty pump users contact an Animas representative to review their options for purchasing or upgrading to our latest insulin pump prior to the end of 2015.

Should your patients have any questions or concerns, please direct them to call the Animas Customer Support Department by dialing <<XXX>>.

Thank you for the quality care and support you provide to your patients.

Sincerely,

Animas Corporation