


**Spring Arm Modification Instructions**

**Ondal AC Swing on ML 301 Examination Lights**

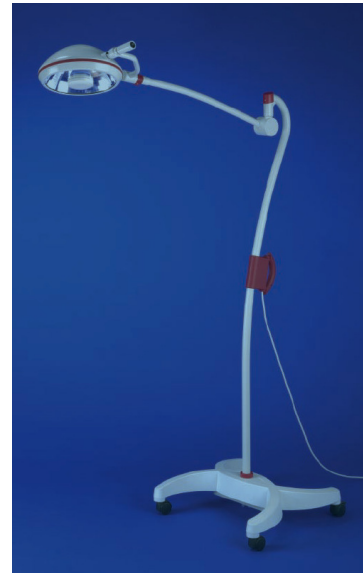


Our supplier Ondal Medical Systems GmbH recently informed us that there may be a risk of fatigue fracture in the plastic joint of Ondal AC Swing arms.


For this reason, the spring arm must be replaced as a precautionary measure.

**Identification of old-type spring arms with plastic joint in comparison with new spring arms with metal joint.**

The Ondal AC Swing spring arm with plastic joint has a curved tube.

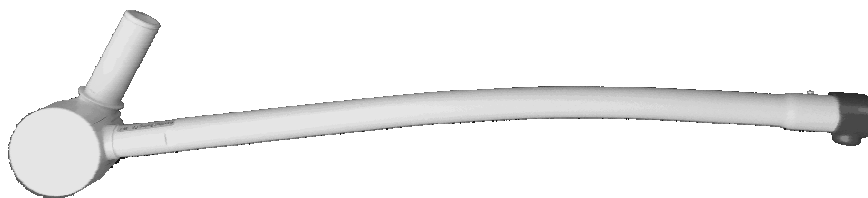

The Ondal AC Swing spring arm with metal joint has a straight tube.

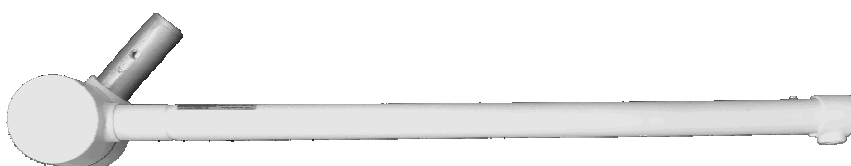

For all ML 301 examination lights with a curved spring arm, the spring arm must be replaced with a straight-tube version.

**Item numbers of the new spring arms:**

For the ML 301 mobile light shown:

Martin item no.: **08-693-23-04** Campaign "ML 301 Swing Spring Arm, Stand"

For the ML 301 ceiling-mounting and wall-mounting lights shown:

Martin item no.: **08-693-22-04** Campaign "ML 301 Swing Spring Arm, Ceiling/Wall"

Both spring arms are available to you at a campaign price of EUR 140.00 net per unit, with the statutory value-added tax to be added as appropriate.

Outside of the European Union, this price may vary due to customs duties.

**Replacement notices:**

To facilitate replacing the spring arm, a supporting document is available in the form of the 5-language ML 301 Service Instructions with item no. 90-735-58-22. This document can be ordered free of charge with the corresponding spring arm.

In particular, the safety instructions provided in the ML 301 Service Instructions must be observed when replacing the arm.

Once the repair has been completed, it is mandatory to check/test all functions before returning the system to service.

Besides, a technical safety check must be performed and the results documented.

**Gebrüder Martin GmbH & Co. KG**

Ludwigstaler Straße 132 · D-78532 Tuttlingen  
Postfach 60 · D-78501 Tuttlingen/Germany  
Tel. +49 7461 706-0 · Fax +49 7461 706-193  
info@klsmartin.com · www.klsmartin.com