


ORTHO BioVue[®] System Cassette Inspection Procedure

| Step | Action |
|------|---|
| 1. | Perform a visual inspection of the affected lots of ORTHO BioVue [®] System Cassettes prior to use. (Refer to CL13-198_Affected Products) |
| 2. | Position the cassette so that the Orientation Mark is located on your right-hand side. |
| 3. | <ul style="list-style-type: none"> • If the Name of the Reagent/Cassette is facing you as shown in Figure 1a, it is acceptable to use the cassette for testing. • If the Barcode is facing you as shown in Figure 2b, do not use the cassette. |
| 4. | Do not use ORTHO BioVue [®] System Cassettes with an incorrectly positioned label (Figure 2b). Contact your Customer Technical Support representatives for to report the issue and request assistance at insert appropriate number . |

Properly Positioned Label

(Front View - Figure 1a)

(Back View -Figure 1b)

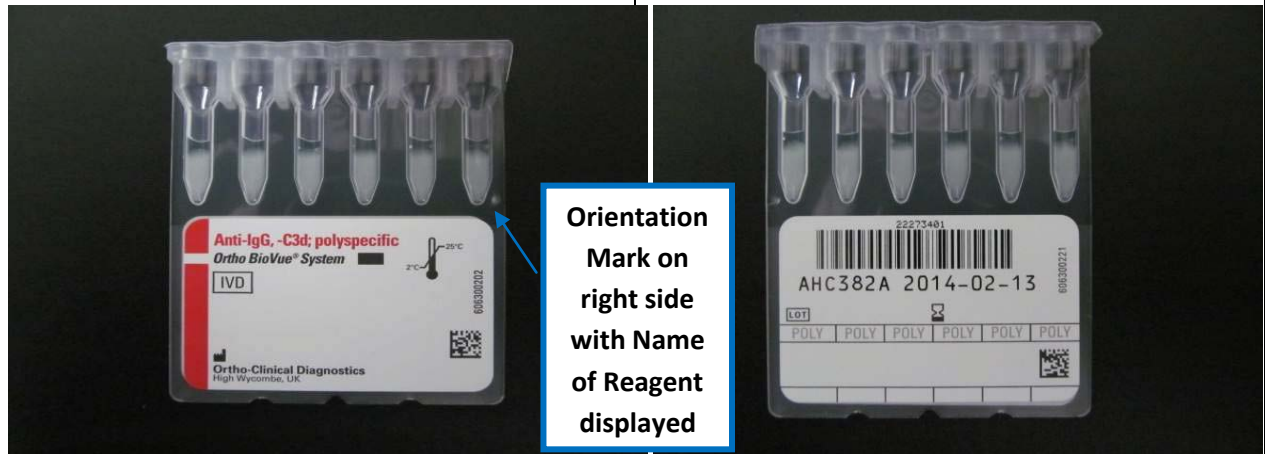

Improperly Positioned Labels

(Front View - Figure 2a)

(Back View - Figure 2b)

