

**To operators and users of  
Dräger infant warmers and open infant care units  
Babytherm 8004/8010 and Babytherm 8000WB for newborns**


August 2014

**Important safety information!!!  
Babytherm 8000WB, Babytherm 8004 and Babytherm 8010  
Warmers for newborns**


Dear Sir / Madam,

As part of our market and product monitoring we have become aware of a case in which a side panel on a Babytherm 8010 did not fully slide to the bottom in the corner guides and was not completely locked. The side panel did not open during this incident and no patient was affected.

When reviewing the locking mechanism in a number of hospitals it was possible to fully close the side panels by pushing down on them slightly. However, it cannot be guaranteed that the panel will lock into position under its own weight. In this respect, we would like to remind you to pay attention to the information provided in the Instructions for Use: "Check that the side panels lock securely into position and that they remain securely locked"

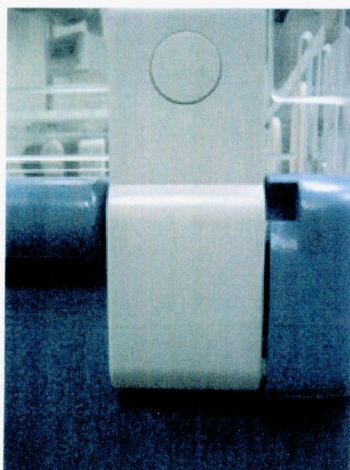

Securely locked

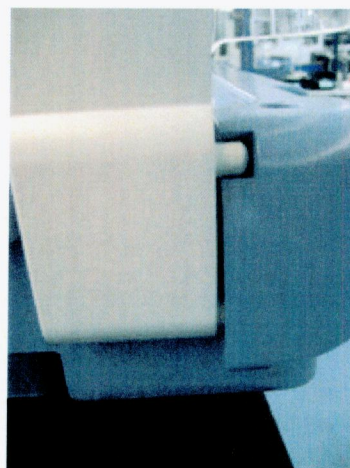

Not securely locked

Dräger Medical GmbH  
Moislinger Allee 53-55  
23558 Lübeck, Germany  
Postal address:  
23542 Lübeck, Germany  
Tel. +49 451 882-0  
Fax +49 451 882-2080  
info@draeger.com  
www.draeger.com  
VAT ID DE813745277

Bank information:  
Commerzbank AG, Lübeck  
Acct. no. 014 679 500  
Bank number 230 400 22  
IBAN DE95 2304 0022 0014 6795 00  
SWIFT code COBADEFF230

Sparkasse zu Lübeck  
Account no. 1 071 117  
Bank number 230 501 01  
IBAN DE15 2305 0101 0001 0711 17  
SWIFT code NOLADE21SPL

Company headquarters: Lübeck  
Commercial register:  
Municipal Court Lübeck HRB 4358 HL

Chairman of the Executive Board:  
Professor Dr. Nikolaus Schweickart

General managers:  
Stefan Dräger (Chairman)  
Dr. Herbert Fehrecke  
Andreas Frahm  
Gert-Hartwig Lescow  
Anton Schrofner

Page 2 / 2

If the locking mechanism is not functioning, please contact your DrägerService representative.

Furthermore, as part of the above review, we identified that the inner 70mm panels are to be inserted pursuant to the Instructions for Use are not always used.

However, the inner panels are part of the safety concept of the Babytherm 80xx. If, contrary to the information provided in the Instructions for Use, the inner panels are not inserted and if a side panel has not been fully locked into position, this may allow the baby's movements to open the side panel. It may be possible for a child to fall out!

We hope that this preventive measure will help to increase patient safety.

Please feel free to contact your local Dräger representative if you have any questions.

With best regards,



for Frank Clanzett  
Head of Postmarket Surveillance  
Quality and Regulatory Affairs



Tobias Winkler  
Vice President  
Respiratory Care & Thermo Regulation  
Product & Application Management