

Anspach Pneumatic Systems.

High speed performance and power in a variety of applications.

Instructions for Use



High Speed
Power Tools

Table of Contents

Introduction	Synthes Anspach	2
	Pneumatic Instrument Systems	2
	Indications	3
	Warnings and Cautions	4
	Technical Specifications	5
	Glossary of Symbols	6
Operating Instructions	AutoLube III	7
	Black Max	10
	XMax and microMax	11
	iMRI Instrument System	12
Attachments and Dissection Tool Assembly	Straight Attachments and Dissection Tool Assembly	15
	For Black Max	15
	For XMax and microMax	17
	Minimal Access Attachments and Dissection Tool Assembly	23
	Micro Curved Attachment	25
	MicroSaws and Small Attachments Assembly	26
	Handheld Attachments Assembly	29
Information	Anspach Powered Irrigation System	32
	Cleaning and Maintenance	34
	Sterilization	37
	Troubleshooting	38
	Ordering Information	41
	Warranty and Return Policies	43

Anspach Pneumatic Systems. High speed performance and power in a variety of applications.

Synthes Anspach

Synthes Anspach manufactures pneumatic and electric high performance instruments and attachments to meet the specific needs of both surgeon and staff. While the advanced design of these instruments provides unparalleled performance and reliability, they also allow for effortless assembly, meeting the needs of both surgeon and staff for the most demanding applications. No tools are required to assemble dissection tools or attachments.

All Synthes Anspach instruments are manufactured to conform to rigorous quality standards to provide dependable performance.

⚠ Important: Before using any of the Synthes Anspach High Performance Systems, it is imperative that all individuals working with the system read this operating manual. The surgeon is responsible for learning the proper techniques in the use of this system, as improper use may cause injury.

To schedule a hands-on training course, please contact your local Synthes sales representative.

If you have any questions after reading this manual, please call your local Synthes sales representative.



Pneumatic Instrument Systems

The XMax sets a new standard for the ultimate combination of high power, smooth operation, low noise level, and small size. This new level of performance provides the perfect pneumatic system to handle a complete range of surgical procedures ranging from power demanding applications to the most delicate dissection. With a minimum speed of 80,000 rpm at 120 psi (8.3 bar) input pressure, the XMax system is one of the quietest pneumatic handpieces available, with 35% more power than Black Max and 50% more than microMax Plus.

The microMax Plus operates at 80,000 rpm. It is small and lightweight to minimize fatigue for long procedures. It has 60% more power than microMax.

The Black Max, operating at 80,000 rpm at 120 psi (8.3 bar), offers smooth, controlled bone dissection.

The iMRI Instrument System provides an Intraoperative MRI compatible configuration. The iMRI Instrument System utilizes non-magnetic materials in the handpiece, attachment, and dissection tool components that have been approved for use with field strengths up to 1.5 Tesla.

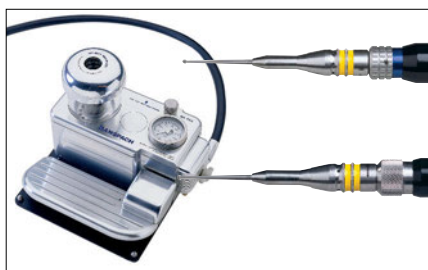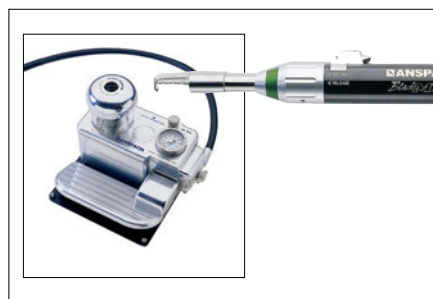

Indications

XMax

Cutting and shaping bone

Black Max


Cutting and shaping bone

microMax

Cutting and shaping bone

iMRI Surgical Drill System

The Anspach iMRI Surgical Drill System (including pneumatic drill motor, foot control, nosepiece attachments and cutters), is indicated for intraoperative (non-imaging) use within a Magnetic Resonance Imaging (MRI) environment rated at 1.5 Tesla or less, for surgical cutting, shaping and removal of bone, including bones of the skull and spine.

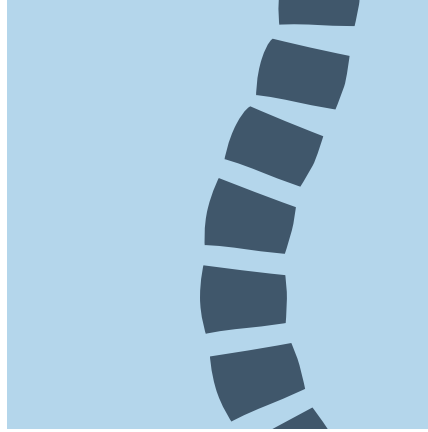

Spine surgery



Neurosurgery



ENT surgery

Warnings and Cautions

Surgeon is responsible for learning proper techniques in use of equipment; improper use may cause serious injury to user or patient or damage to system.

Instrument operator and all operating room personnel must wear eye protection.

Visually inspect for damage before using; do not use if damage is evident.

Do not use accessories other than those provided by Anspach and specified for use with Anspach systems.

Use care to protect hose when handling, cleaning, and during system use.

Damage to hose can cause leaking, rupture, or other related failures.

Handpiece's outer hose can rupture without warning if occluded.

Do not step on, set equipment on, pinch, kink, clamp, or otherwise occlude handpiece hose during use.

Do not use if pressure relief valve has cracks or hose has cuts or abrasions.

Do not use if the product sterilization barrier or its packaging is compromised.

Do not use, or discontinue use of powered equipment exhibiting excessive temperatures that can cause patient injury (necrosis) and/or user discomfort.

Use of damaged or improperly maintained power equipment and/or misused powered equipment can result in excessive temperatures.

To insure equipment operates as designed, read and follow manufacturer's instructions, including those for proper service and maintenance.

Do not operate handpiece without an attachment and the corresponding dissection tool. (i.e. Only use Black Max cutting burrs with Black Max attachments.)

Use caution to avoid cutting or tearing gloves while handling dissection tools.

Dissection tools must be adequately retained within attachment to prevent distal migration, which may cause injury.

Confirm attachment is proper size for dissection tool and that it is secure.

Gently pull on dissection tool shaft to ensure it is fully seated and properly installed.

Do not engage safety mechanism while handpiece is running; doing so makes safety mechanism inoperable.

Only cut visible areas unless an image intensifier is utilized.

Delicate structures in proximity to dissection must be thoroughly protected to prevent injury.

Irrigation is necessary for proper operation.

Maintain firm control of instrument at all times.

Do not bend or use as a lever.

Use a gentle tapping motion or side-to-side motion and let instrument do cutting.

Do not use excessive force.

Forceful side loading of dissection tool may cause fracture of dissection tool, which may cause injury.

Dissection tools are disposable and intended for single patient use only. Do not resterilize and/or re-use dissection tools.

Use standard protocol for disposal of sharp instruments.

Technical Specifications

BM

Black Max Specifications*

Speed	up to 80,000 rpm
Handpiece, muffler, hose	3.81 m (12.5 ft)
Outside housing (diameter)	19 mm (.75 in)
Length of housing	91 mm (3.6 in)
Handpiece weight	70 g (2.6 oz)
Handpiece, muffler, hose weight	0.65 kg (1.4 lbs)

XM

XMax Specifications

Speed	up to 80,000 rpm
Handpiece, muffler, hose	3.81 m (12.5 ft)
Outside housing (diameter)	17.4 mm (.685 in)
Length of housing	102.02 mm (4.025 in)
Handpiece weight	75.3 g (2.7 oz)
Handpiece, muffler, hose weight	0.65 kg (1.45 lbs)

The device complies with the following standards:

IEC 60601-1/IEC 60601-1-2

IEC 61000-3-2/IEC 61000-3-3/IEC 61000-4-2/IEC 61000-4-4/

IEC 61000-4-5/IEC 61000-4-11/IEC 61000-4-8/

IEC 61000-4-6/IEC 61000-4-3



With regard to electrical shock, fire, mechanical hazards, this device is certified to UL 60601-1 and CAN/CSA C22.2 No. 601.1

This device complies with applicable EEC directives.



*The Synthes Anspach Black Max Instrument System is not sponsored by, endorsed by or otherwise associated with United States Manufacturing Corp. or its Blackmax prosthetic limbs.

MM

microMax Specifications

Speed	up to 80,000 rpm
Handpiece, muffler, hose	3.81 m (12.5 ft)
Outside housing (diameter)	15.8 mm (.625 in)
Length of housing	89 mm (3.5 in)
Handpiece weight	56.70 g (2 oz)
Handpiece, muffler, hose weight	0.5 kg (1.1 lbs)

MM

microMax Plus Specifications

Speed	up to 80,000 rpm
Handpiece, muffler, hose	3.81 m (12.5 ft)
Outside housing (diameter)	15.8 mm (.625 in)
Length of housing	95.25 mm (3.75 in)
Handpiece weight	75.3 g (2.7 oz)
Handpiece, muffler, hose weight	0.65 kg (1.4 lbs)

iMRI

iMRI Handpiece Specifications

Speed	10,000–75,000 rpm
Handpiece and Hose	3.81 m (12.5 ft)
Outside Housing (diameter)	17.5 mm (.69 in)
Length of Housing	101 mm (4 in)
Handpiece and hose weight	635 g (1.41 lb)

Patent Information













Synthes Anspach products are manufactured under one or more of the following U.S. Patent Nos. RE 37,358; 5,405,348; 5,601,560; 5,630,818; 5,741,084; 5,904,687; 6,607,533; 6,733,218; 6,746,153; 6,749,341; 6,969,368; 7,128,544; 7,144,415; 7,217,090; 7,255,546; 7,261,526 and 7,458,979.


Additional U.S. and International patents pending.


Additional Information


All specifications are subject to change.

Glossary of Symbols


	Reference Number (A.K.A. Item Number, Catalog Number, Part Number)
	Lot (A.K.A. Lot Number, Batch Number, Batch Code)
	Serial Number
 00/0000	Use By Date (A.K.A. Expiration Date, Expiry)
	Manufacturer
	Authorized European Representative
	Sterile R (A.K.A. Sterilized Using Irradiation)
	Single Use Only (A.K.A. Do Not Reuse)
	Attention (A.K.A. Caution, Warning, Danger, Important, Note, Refer to Accompanying Documentation)
	Keep Dry (A.K.A. Protect from Moisture)
	Sterile unless damaged or open
	United States Federal law restricts this device to sale by or on the order of a physician or other licensed health-care provider.


 CE Mark (A.K.A. CE Mark [notified body number], Conformité Européenne)
 Meaning: Device complies with applicable EEC Directives

 MR Conditional. No known hazards at 1.5T.

 Lock (Run)

 Unlock (Load)

 Transmitter
Interference may occur in the vicinity of equipment marked with this symbol.

 Type B Electrical Equipment

Instrument System Key

 Black Max  microMax/microMax Plus
 XMax  iMRI

Latex-free Certification

There is no latex material in Synthes Anspach products or packaging.

AutoLube III Operating Instructions For Black Max, XMax and microMax

Assembly Procedures of AutoLube III and Handpiece

AUTOLUBE-III Foot Control

For additional part numbers, refer to the "Ordering Information" section





1. Remove the red plug (installed for shipping only) from the recirculating bowl located on the top of the AutoLube III.
2. Turn the silver handle located on the back of the Autolube III to the "OPEN" position.
3. While maintaining the handle in the "OPEN" position, insert the muffler end of the pneumatic handpiece into the cavity located on top of the recirculating bowl.
4. Insert the muffler until the red ring on the muffler cannot be seen.
5. Rotate silver handle to the "LOCK" position. Check for secure engagement.

Operating Pressure

Anspach pneumatic handpieces run on air or medical grade nitrogen. Standard operating pressure is 90-120 psi (6.2–8.3 bar). If more or less power is desired, the psi may be regulated between 40 and 140 psi.

1. Connect the air hose from the AutoLube III to the hospital air source, nitrogen tank or hospital nitrogen system.
2. Adjust the air source until the desired air pressure registers on the gauge of the AutoLube III. Pressure will not register on the AutoLube III until the muffler is inserted and the foot pedal is activated.
3. The wall/nitrogen tank pressure may read as much as 40 psi higher than the AutoLube III gauge.


⚠ Handpiece is fully functional at this time.

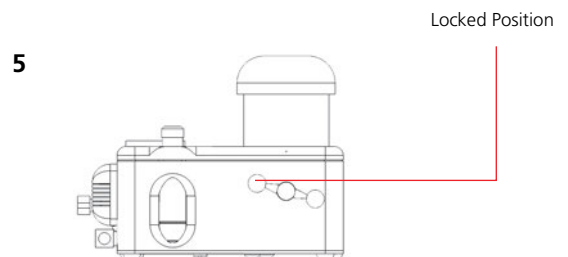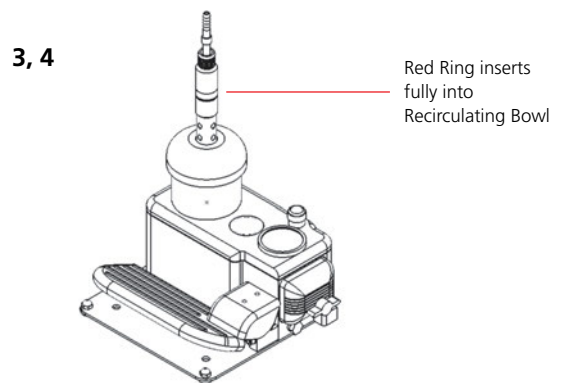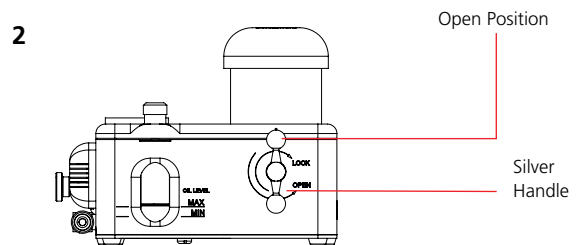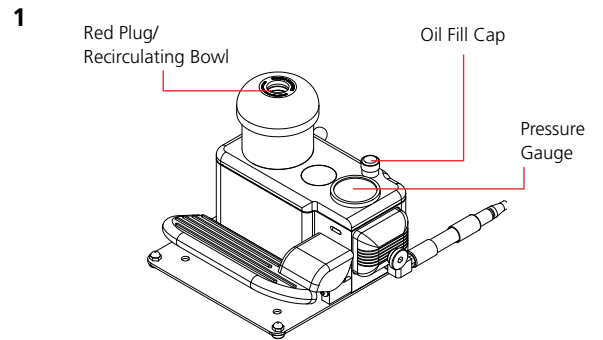

Pressure Relief Valve

⚠ Handpiece hose may include a pressure relief valve to remove excess pressure in the case of hose occlusion between the AutoLube and the valve. If the pressure relief valve is actuated during use, remove the source of the occlusion. Do not attempt to disassemble the pressure relief valve.

Pressure Relief Valve

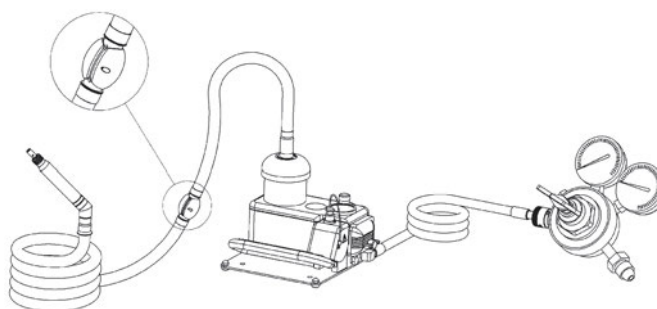


AutoLube Lubrication

OIL-II Instrument Oil with 1 Replacement Filter

⚠ The AutoLube III has been delivered without oil or a filter installed. A bottle of Anspach lubricating oil, filter, and muffler have been shipped with the unit and must be added to the AutoLube III prior to use.


1. Before adding oil, disconnect the AutoLube III from the air source.
2. Remove Oil Fill Cap from the top of the AutoLube III and fill with Anspach lubricating instrument oil (OIL-II).

Do not fill above the "MAX Oil Level" line next to the oil reservoir. Always replace the oil fill cap securely. One full bottle of Anspach lubricating oil should be added when the oil level reaches the "MIN Oil Level" line etched next to the oil reservoir.

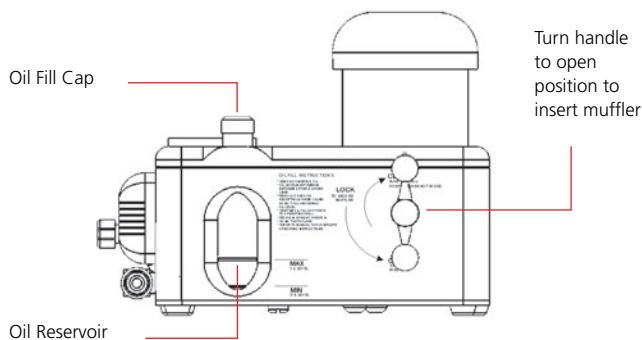


AutoLube Lubrication


1.

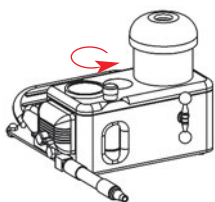


2.

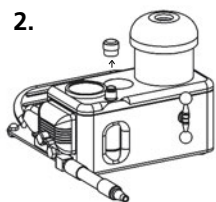



3.

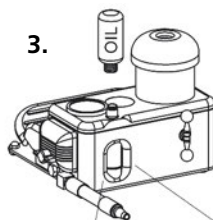

4.

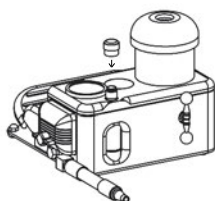

5.

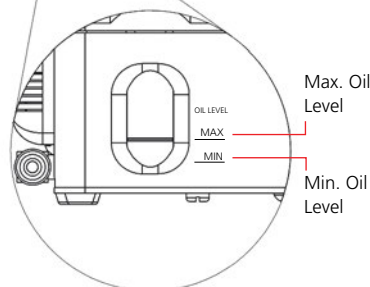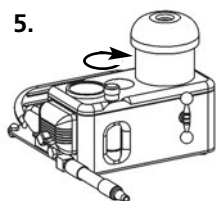

3. Before each use, a visual check of the oil reservoir should be made to ensure there will be an adequate supply of oil to circulate throughout the procedure.
4. Install filter and muffler per following "Autolube-III Filter" section.

Note: Always store and transport AutoLube III in an upright position.




AutoLube III Filter

FILTER-II AutoLube II & III Filters (5/pkg.)

The filter, muffler filter, and vented cover have been packed separately in the AutoLube III box. The filters should be changed each time a bottle of Anspach lubricating oil is added.

1. Insert the filter by taking the tab and sliding the "horse-shoe" shaped portion of the filter into the cavity in a downward motion. The tab will extend out of the cavity.
2. Push muffler filter into vented cover with the hole in muffler filter centered over the attachment screw. The muffler filter is oversized and must be compressed to fit.
3. Place the vented cover over the filter cavity and secure by turning the grey wingnut in a clockwise direction.

AutoLube Filter

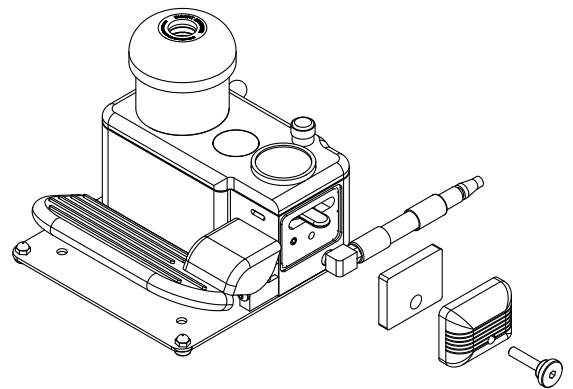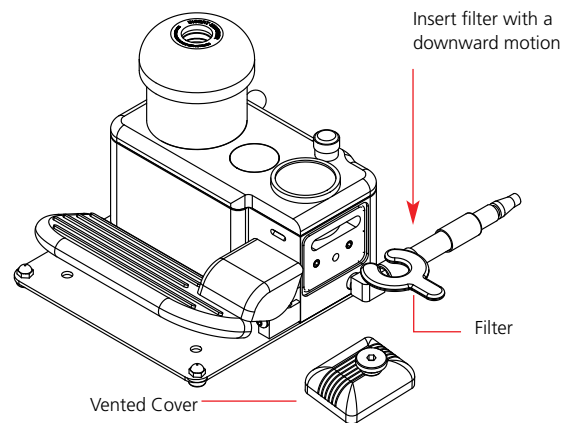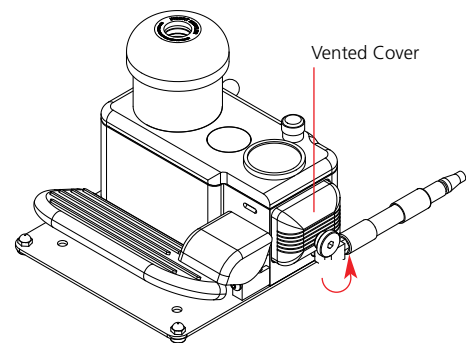

Black Max Operating Instructions

The Black Max Handpiece **BM**

BLACKMAX-N	Swivel/Angle Motor Assembly
BLACKMAX-O	Fixed/Straight Motor Assembly
BLACKMAX-N-LP	Low Pressure Swivel/Angle Motor Assembly
BLACKMAX-NS	Neuro Safety
BLACKMAX-NS-LP	Low Pressure Neuro Safety

Operation

⚠ Important: Firm control of the Black Max must be maintained at all times. Never force the dissection tool. Let the instrument do the cutting. Never use the attachments as a retractor to bend or pry.

Only Anspach Black Max dissection tools should be used with the Black Max.

Always use continuous irrigation to prevent heat build-up.




Always wear safety glasses.

Black Max Directional Features

The words "SECURE" and "RELEASE" engraved on the side of the Black Max handpiece refer to the locking and unlocking the attachment onto the handpiece.

The Lever, located on the top of the handpiece, is used to lock and unlock dissection tools within the handpiece.

⚠ Caution: If the dissection tool is not inserted properly, the Lever will not secure the dissection tool. Do not force the Lever. For instructions on attachment assembly, refer to the specific instructions for each attachment in this manual.

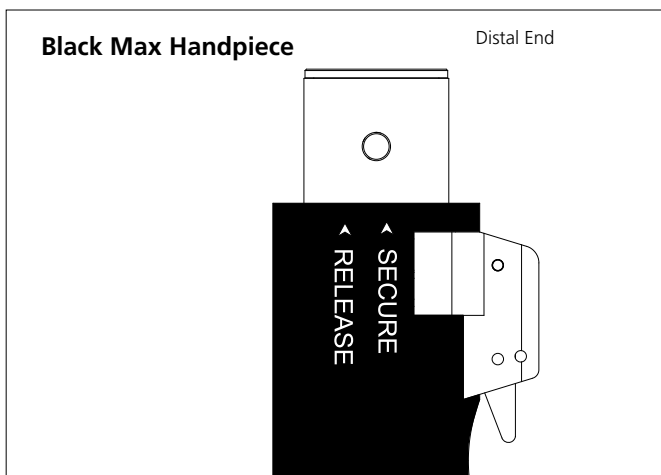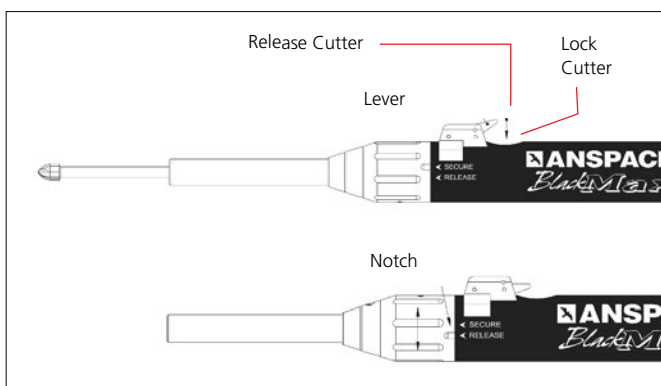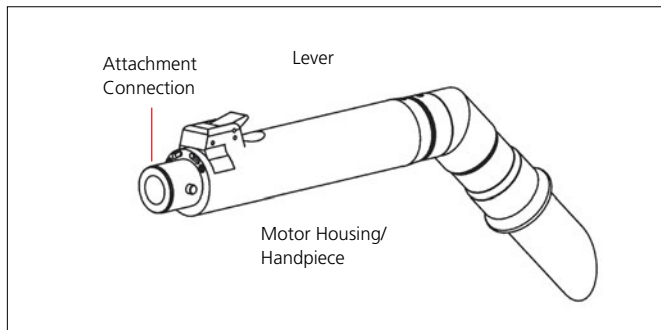

XMax and microMax Operating Instructions

The XMax and microMax Handpieces **XM** **MM**

XMAX	XMax Motor
MICROMAX	microMax Motor
MICROMAX-LP	microMax Motor, Low Pressure
MICROMAX-PLUS	microMax Plus Motor
MICROMAX-PLUS-LP	microMax Plus Motor, Low Pressure

Operation

⚠ Important: Firm control of the handpiece must be maintained at all times. Never force the dissection tool. Let the instrument do the cutting. Never use the attachments as a retractor to bend or pry.

Only Anspach XMax/microMax dissection tools should be used with the XMax and microMax.

Always use continuous irrigation to prevent heat build-up.

Always wear safety glasses.

XM

XMax Directional Arrows

Notice the directional arrows and printing on the distal end of the handpiece housing: "RUN" (🔒), "LOAD" (🔓) and "SAFE." These terms refer to locking and unlocking the attachment and dissection tool onto the handpiece (Figure 1).

For instructions on attachment assembly, refer to the specific instructions for each attachment in this manual.

MM

microMax Directional Arrows

Notice the directional arrows and printing on the side of the handpiece at the distal end: "SECURE" and "RELEASE." These terms refer to locking and unlocking the attachment and dissection tool onto the handpiece (Figure 2).

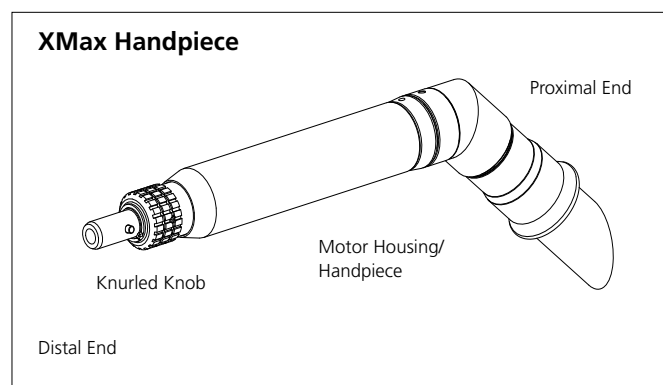


Figure 1

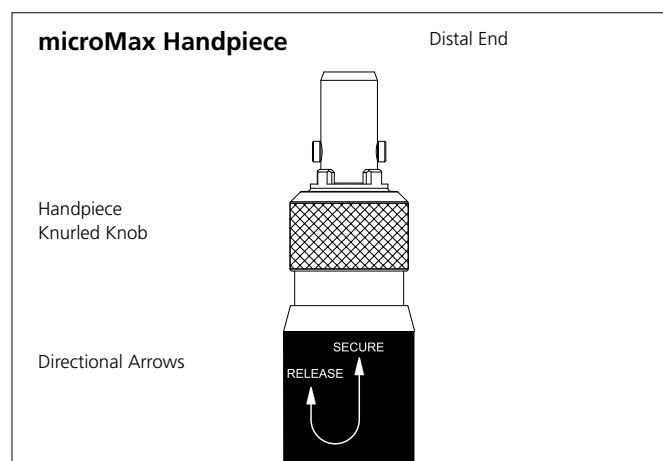


Figure 2

iMRI Instrument System **iMRI**

IMRI-MOTOR	iMRI Motor
IMRI-FOOT-CTRL	iMRI Air Control
IMRI-SHORT	iMRI Short Attachment
IMRI-SHORT-HD	iMRI Short HD Attachment
IMRI-CRANI-A	iMRI Crani-A Attachment
IMRI-OIL	iMRI Synthetic Oil

For additional components, refer to page 41.

Use, Care and Operating Instructions

⚠ Important: Do not use any attachments or cutters with iMRI handpiece, not manufactured by Anspach or not identified as "iMRI". Do not operate handpiece without attachment and cutter installed.

Read and understand thoroughly before operating your new iMRI Instrument System. Failure to do so could lead to injury to you or your patient or cause severe damage to this equipment.

The iMRI Instrument System is a design modification of the Anspach microMax System. Refer to the "XMax and microMax Operating Instructions" section for additional assembly instructions.

iMRI System and cutters are constructed of materials that permit intraoperative use in an MRI unit rated at 1.5 Tesla or less and only when MRI is not being used for actual imaging purposes.

⚠ Caution: Do not use in MRI units rated at greater than 1.5 Tesla. Remove iMRI System from MRI imaging volume during imaging. Failure to do so could result in unacceptable image noise and/or distortion.

Testing of iMRI System as established by ASTM F2213-02 "Standard Test Method for Measurement of Magnetically Induced Torque on Passive Implants in Magnetic Resonance Environment" and F2052-02 "Standard Test Method for Measurement of Magnetically Induced Displacement Force on Medical Devices in Magnetic Resonance Environment" confirm there is no detectable magnetically induced torque on handpiece at 1.5 Tesla (when tested in a 1.5 T magnet) and only slight deflection force (10° w/18% of force due to gravity) when subjected to a high field strength and high field gradient location (1.5 Tesla) when tested in a Siemens 1.5T Magnetom Symphony, Maestro Class Unit.

iMRI System is designed for surgical procedures typically involving no more than 15 minutes of continuous (actively drilling) operation. If a procedure is expected to last longer, you should be aware handpiece speed and temperature may be affected.

Normal speed of iMRI System will be less than microMax due to magnetic affects of MRI and because iMRI System does not have continuous oiling. The lower speed will not adversely affect your ability to perform procedure and you will experience little to no magnetic pull on handpiece during use. However, the following effects may be noted:

1. Handpiece speed will slow when handpiece rotating mechanism is held 90° to magnetic field.
2. Handpiece temperature will increase when handpiece is held in 90° position for greater than 5 minutes. Handpiece should not become uncomfortably warm during normal use; procedures lasting greater than 15 minutes could cause handpiece temperature to rise.
3. Lock mechanisms of iMRI System and cutters are essentially the same as microMax. However, to help prevent unintended use of non-iMRI devices, geometry has been slightly modified to prohibit attachment of non-iMRI handpiece compatible devices.

⚠ Important: Attempts to attach microMax components to iMRI handpiece could result in significant damage to iMRI Handpiece.

Straight attachments should be used for no more than 10 procedures. Craniotome attachments should be used for no more than five procedures. Attachments are not repairable and should be replaced after the recommended number of uses.

Metal component parts of iMRI handpiece and attachments, and all iMRI cutters are constructed of materials mostly un-affected by magnetic properties of MRI. However, these materials have significantly less hardness, which directly affects durability. Proper use, cleaning, maintenance and pre-operative inspection will help ensure safe use and prolong useful life of iMRI handpiece and attachments. Periodic routine servicing by Anspach is strongly recommended.

iMRI cutters will dull more quickly than standard cutters because they are manufactured of non-magnetic reactive material. Use with iMRI only. During surgical use, you should remain cognizant of cutter performance and have additional cutters available at all times. Continued use of dull cutters, in addition to possible necroses, may put additional stress on attachment and handpiece, leading to premature failure of those devices. Before each use, assemble system and test to insure proper performance.

Proper system performance is usually indicated when there is no unusual difficulty in attaching attachments and cutters, and when system operates at full speed (unloaded with cutter attached) with no unusual noise, vibration, wobble or whipping.

⚠ Important: Carefully inspect handpiece and attachments post-cleaning and again pre-operatively to insure there is no corrosion or remaining debris which could adversely affect operation or lead to attachment bearing failure during use.

Continued use of damaged or worn handpiece or attachments can lead to bearing (or other) failures, which can result in foreign debris entering surgical wound site. Never reprocess or reuse iMRI cutters. iMRI cutters must be properly disposed of after single use.

iMRI Operating Instructions

⚠ Warning: Do not run handpiece without cutter and attachment installed.

1. Foot control must be placed on floor within reach of surgeon. Foot control is not subject to sterilization so should never be placed such that sterile field is compromised.
 2. Place three (3) drops of iMRI OIL into muffler end of iMRI handpiece hose assembly. Do not exceed three (3) drops.
-

⚠ Important: iMRI OIL is for use with iMRI Handpiece only. Do not use with other Anspach "Autolube" systems. iMRI OIL used in AutoLube can damage the sight window, resulting in oil leakage.

3. Connect muffler end of hose to foot control and inspect handpiece to insure it is "iMRI" and that internal aspect of handpiece is free of damage or debris.
-

⚠ Caution: Do not run handpiece without cutter and attachment installed. Damage to handpiece may result if handpiece is operated without cutter and attachment installed.

4. Select appropriate attachment. Inspect for any signs of corrosion or damage and insure it is "iMRI." Attach attachment to handpiece and turn clockwise to lock onto handpiece. Verify locking by pulling on attachment.
5. Select appropriate cutter and following all established sterile techniques, remove it from its packaging, insert it into and through attachment, into handpiece and lock into place. Verify locking by pulling on cutter. If using craniotome, cutter should be put into handpiece first then put craniotome over cutter.
6. Attach iMRI foot control to air (or nitrogen) supply. Fully depress foot pedal and adjust supply air pressure such that gauge on foot control indicates 120 psi.

7. With handpiece in MRI field but away from patient, fully depress foot control and assess handpiece for noise, vibration, wobble, and whipping. If assembly was without difficulty and handpiece sounds and feels right, initiate or continue with use.

8. During procedure, continuously assess cutting effectiveness. Though iMRI cutters are designed to cut effectively, they are constructed of a much softer metal than standard (non-iMRI) Anspach cutters which you may be accustomed to. Because of material softness, iMRI cutters will dull more quickly. If you notice a reduction in cutting effectiveness, stop and replace cutter.
-

⚠ Warnings: Do not reprocess or use reprocessed cutters. iMRI cutters are constructed of stainless steel that is subject to dulling and possible distortion during initial use. Reprocessors may not be able to reprocess cutters to their original design condition and reuse could result in excessive whipping or vibration that could cause severe patient injury including death. Reprocessed cutters may also be duller than new cutters, resulting in less effective use and possible overheating causing necrosis of tissues and/or surgeon discomfort.

Continued use of dull cutters can cause user to apply greater pressure while cutting, which can seriously damage attachment, and possible bearing failure. Bearing failure can result in foreign debris entering surgical wound site.

iMRI craniotome is constructed of a softer metal than a non-iMRI craniotome and excessive applied force will cause craniotome leg/foot to bend or fracture. Do not bend attachment or use craniotome as a lever. Do not force cutter. Forceful side load may cause cutter fracture, which could cause injury. It can also result in handpiece speed reduction and increased temperature.

9. During surgical procedure, when MRI is used for actual imaging, remove iMRI System and Cutters from imaging volume.

Black Max Standard Attachments and Dissection Tool Assembly

Black Max Dissection Tool Identification

All dissection tools are packaged sterile, unless otherwise indicated, and are manufactured for single use. Each dissection tool has a series of identification numbers and letters imprinted on the label and on the shaft. The first group of letters and numbers designate the part number, the second group of letters and numbers designate the lot number. Each attachment is laser-marked with its corresponding part number.

The first group of upper case letters of the dissection tool part number correspond to the attachment's color-coded ring and/or name and the last digit represents the dissection tool style.

For example:

Black Max Dissection Tool Part Number: SIL-8


Dissection Tool Description: #8 style dissection tool used with the B-SILVER attachment.

Black Max Straight Attachments

Refer to "Ordering Information" section for part numbers.

1. Lift Lever on handpiece. Align notch on attachment with "RELEASE" on side of Black Max handpiece. Turn attachment so notch is aligned with word "SECURE."
2. Insert dissection tool into attachment. Return Lever on Black Max handpiece to down position.
3. Pull dissection tool to ensure proper engagement.
4. Adjust air source until desired air pressure registers on AutoLube gauge.

Note: Air pressure will not register until foot pedal is depressed.

 Handpiece is now fully functional.

The Operating pressure is 120 psi (8.3 bar), as indicated on the AutoLube gauge, when using these attachments, except for those specified in "Assembly Procedures of AutoLube III" section.

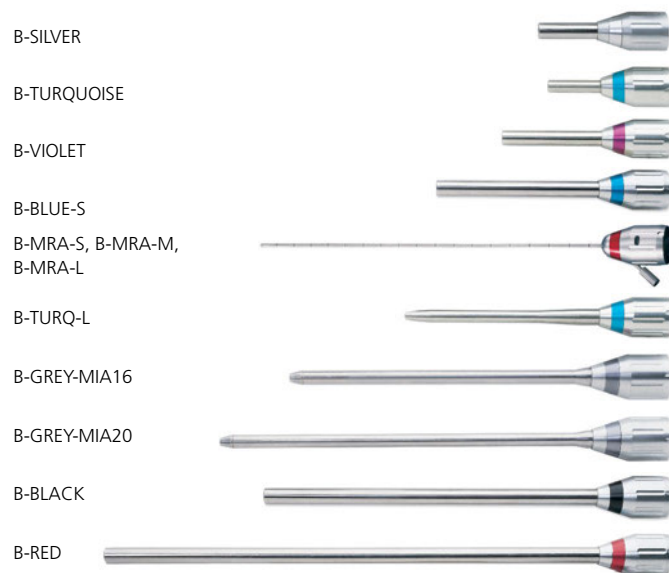


Figure 1: Distal end of handpiece or attachment is pointed away


Black Max Contra and Right Angle Attachments

B-ORANGE-45 Contra Angle Attachment (Multiple Angle)
 B-ORANGE-90 Right Angle Attachment (90°)

1. Lift Lever on handpiece. Align notch on attachment with "RELEASE" on side of Black Max handpiece. Turn attachment so notch is aligned with word "SECURE."
2. Push latch on the distal end of the attachment up with your thumb.
3. Insert dissection tool into the side of the distal end (opposite side of latch) while rotating slowly until dissection tool seats.
4. Release latch and check security of attachment by pulling it away from handpiece and check dissection tool by pulling on dissection tool head. Latch will be flush with attachment when dissection tool is secure.



Figure 2: Distal end of handpiece or attachment is pointed away

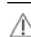 Handpiece is now fully functional.

Black Max Craniotome and Specialty Attachments

Refer to "Ordering Information" section for part numbers.

1. Lift Lever on Black Max. Insert dissection tool into handpiece. Return Lever to down position. Pull dissection tool to ensure proper engagement.
2. Slide attachment over dissection tool. Align notch on attachment with word "RELEASE" on the Black Max. Next turn attachment so notch is aligned with word "SECURE."
3. The handpiece is now operational. Operating pressure is 120 psi (8.3 bar), as indicated on the AutoLube gauge, when using these attachments.

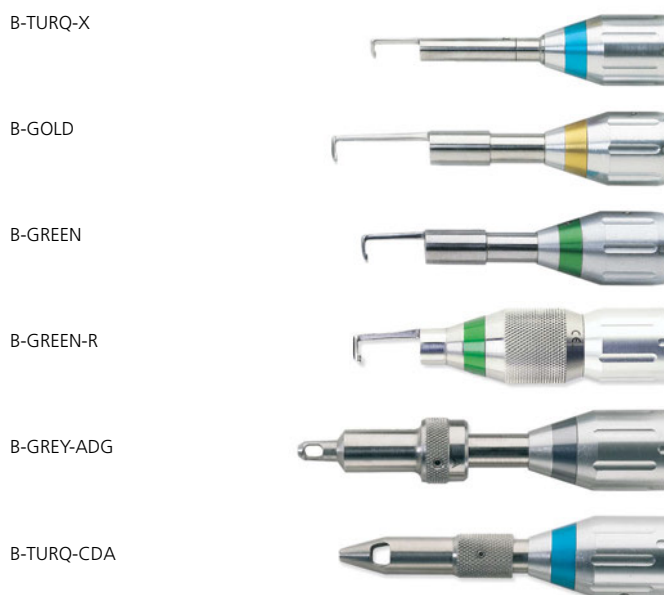


Figure 3

Black Max QD Angle Attachments

Refer to "Ordering Information" section for part numbers.

1. Lift Lever on Black Max. Align notch on angle attachment with word "RELEASE" on the side of the Black Max. Next, turn angle attachment so notch is aligned with word "SECURE" which is laser etched on the side of the Black Max.
2. Pull down retaining sleeve on angle attachment and rotate to left when attachment is pointed away from you.
3. Insert dissection tool.
4. Release retaining sleeve by rotating to left.
5. Pull dissection tool to ensure proper engagement.
6. Return Lever on Black Max handpiece to down position.




Figure 4

⚠ Handpiece is now fully functional.


Operating pressure is 120 psi (8.3 bar), as indicated on the AutoLube gauge, when using these attachments.

XMax and microMax Standard Attachments and Dissection Tool Assembly

Straight Attachment Assembly

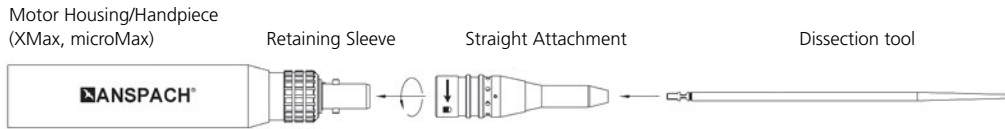

Craniotome Attachment Assembly

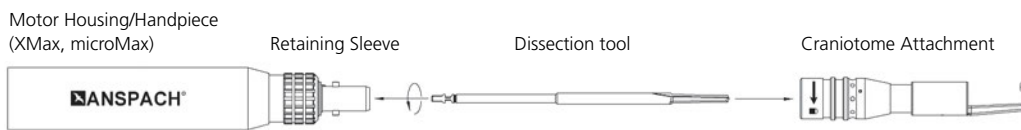

Dissection Tool Identification

All dissection tools are packaged sterile, unless otherwise indicated, and are manufactured for single use. Each dissection tool has a series of identification numbers and letters imprinted on the label and on the shaft. The first letter describes the type of attachment, and the following characters indicate the size and style of the dissection tool.

For example:

Dissection Tool Part Number: S-2B

Dissection Tool Description: 2 mm Fluted Ball for Short attachment

Each attachment is laser-marked with its corresponding part number.

Straight Attachments

SHORT	5 cm Short Attachment
MEDIUM	8 cm Medium Attachment
LONG	11 cm Long Attachment
MIA16	16 cm Minimally Invasive Attachment
SHORT-HD	5 cm Heavy Duty Short Attachment
MEDIUM-HD	8 cm Heavy Duty Medium Attachment
LONG-HD	12.4 cm Heavy Duty Long Attachment
XL-HD	20.2 cm Heavy Duty Extra Long Attachment

MM

Assembly for handpieces with "SECURE" and "RELEASE"

1. Move handpiece knurled knob to "SECURE" position.
2. Slide attachment over distal end of handpiece housing, push down and rotate attachment to right approximately one quarter turn until fully seated.
3. Pull down on handpiece knurled knob and insert dissection tool into distal end of attachment. Rotate dissection tool slowly until dissection tool seats.
4. Release handpiece knurled knob to "SECURE" position.

 Handpiece is now fully functional.

5. Check security of dissection tool by pulling on dissection tool shaft.
6. Attachment and dissection tool are secure.

Disassembly

1. Move handpiece knurled knob to "RELEASE" position.
2. Remove dissection tool from distal end of attachment.
3. Rotate attachment to left approximately one quarter turn and remove from distal end of handpiece.

SHORT



MEDIUM



LONG



MIA16



SHORT-HD



MEDIUM-HD



LONG-HD






XL-HD




XM

Assembly for handpieces with and .

(Distal end of handpiece or attachment is pointed away from user.)

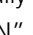1. Move handpiece knurled knob to "RUN" () position.
2. Slide attachment over distal end of handpiece. Pull attachment towards handpiece, and rotate attachment to right approximately one quarter turn until fully seated. Handpiece knurled knob moves up to seat next to attachment.
3. Pull handpiece knurled knob towards "LOAD" () position and insert dissection tool into distal end of attachment. Rotate dissection tool slowly until fully seated.
4. Release handpiece knurled knob into "RUN" () position.

 Handpiece is now fully functional.

5. Pull dissection tool to ensure proper engagement.
6. Attachment and dissection tool are secure.

Disassembly

1. Move handpiece knurled knob to "SAFE" position.
2. Remove dissection tool from distal end of attachment.
3. Rotate attachment to left approximately one quarter turn, and remove from distal end of handpiece.


Orange 45° or 90° Contra Angle Attachments

ORANGE-45 45° Contra Angle Attachment
ORANGE-90 90° Contra Angle Attachment

⚠ Set AutoLube operating pressure to 40 psi (2.8 bar).

MM

Assembly for handpieces with and .

1. Ensure knurled knob on handpiece is in "RUN" () position.
2. Slide attachment over distal end of handpiece. Push attachment towards handpiece, and rotate attachment clockwise until fully seated. Handpiece knurled knob moves up to seat next to attachment.
3. Push latch on the distal end of the attachment up with your thumb.
4. Insert dissection tool into the side of the distal end (opposite side of latch) while rotating slowly until dissection tool seats.
5. Release latch and check security of attachment by pulling it away from handpiece and check dissection tool by pulling on dissection tool head. Latch will be flush with attachment when dissection tool is secure.

⚠ Handpiece is now fully functional.

Disassembly

1. Lift latch on the distal end of the attachment with your thumb. Remove dissection tool and release latch.
2. Move handpiece knurled knob to "SAFE" position.
3. Rotate attachment counterclockwise and remove from distal end of handpiece.

ORANGE-45




ORANGE-90



XM

Assembly for handpieces with and .

1. Ensure knurled knob on handpiece is in "RUN" () position.
2. Slide attachment over distal end of handpiece. Push attachment towards handpiece, and rotate attachment clockwise until fully seated. Handpiece knurled knob moves up to seat next to attachment.
3. Push latch on the distal end of the attachment up with your thumb.
4. Insert dissection tool into the side of the distal end (opposite side of latch) while rotating slowly until dissection tool seats.
5. Release latch and check security of attachment by pulling it away from handpiece and check dissection tool by pulling on dissection tool head. Latch will be flush with attachment when dissection tool is secure.

⚠ Handpiece is now fully functional.

Disassembly

1. Lift latch on the distal end of the attachment with your thumb. Remove dissection tool and release latch.
2. Move handpiece knurled knob to "SAFE" position.
3. Rotate attachment counterclockwise and remove from distal end of handpiece.

Craniotome and Specialty Attachments

CRANI-P	6.5 cm Pediatric Craniotome
CRANI-A	6.5 cm Adult Craniotome
CRANI-L	7.5 cm Large Craniotome
CRANI-A-R	6.5 cm Rotating Adult Craniotome
ADG	7.2 cm Adjustable Drill Guide
CDA	6.8 cm Controlled Depth Attachment

MM

Assembly for handpieces with "SECURE" and "RELEASE"

1. Pull down on handpiece knurled knob and insert dissection tool into distal end of handpiece. Rotate dissection tool slowly until dissection tool seats.
2. Release handpiece knurled knob into "SECURE" position.

 Handpiece is now fully functional.

3. Check security of dissection tool by pulling on dissection tool shaft.
4. Slide attachment over dissection tool and distal end of handpiece, push down and rotate attachment to right approximately one quarter turn until fully seated.
5. Attachment and dissection tool are secure.

Disassembly

1. Move handpiece knurled knob to "RELEASE" position.
2. Rotate attachment to left approximately one quarter turn and remove from distal end of handpiece.
3. Remove dissection tool from distal end of handpiece.

CRANI-P



CRANI-A



CRANI-L



CRANI-A-R



ADG





CDA




XM

Assembly for handpieces with and .

(Distal end of handpiece or attachment is pointed away from user.)

1. Pull handpiece knurled knob towards "LOAD" () position and insert dissection tool into distal end of handpiece. Rotate dissection tool slowly until dissection tool is fully seated.
2. Release handpiece knurled knob into "RUN" () position.

 Handpiece is now fully functional.

3. Pull dissection tool to ensure proper engagement.
4. Slide attachment over distal end of handpiece. Pull attachment towards handpiece, and rotate attachment to right approximately one quarter turn until fully seated. Handpiece knurled knob moves up to seat next to attachment.
5. Attachment and dissection tool are secure.

Disassembly

1. Move handpiece knurled knob to "SAFE" position.
2. Rotate attachment to left approximately one quarter turn, and remove from distal end of handpiece.
3. Remove dissection tool from distal end of attachment.


QD Angle Attachments

- QD8 8 cm Quick Disconnect Angle Attachment
- QD11 11 cm Quick Disconnect Angle Attachment
- QD14 14 cm Quick Disconnect Angle Attachment

MM

Assembly for handpieces with "SECURE" and "RELEASE"

1. Pull down handpiece knurled knob and slide angle attachment over distal end of handpiece until fully seated. Rotate attachment to right approximately one quarter turn.
2. Release handpiece knurled knob to "SECURE" position.
3. Pull down retaining sleeve on angle attachment and rotate to left.
4. Insert dissection tool into angle attachment. Rotate dissection tool slowly until dissection tool seats.
5. Release retaining sleeve on angle attachment by rotating retaining sleeve to right.

 Handpiece is now fully functional.

6. Check security of dissection tool by pulling on dissection tool shaft.

Disassembly

1. Pull down on retaining sleeve on angle attachment and rotate to left.
2. Remove dissection tool from distal end of attachment.
3. Move handpiece knurled knob to "RELEASE" position.
4. Rotate attachment to left approximately one quarter turn and remove from distal end of handpiece.

QD8



QD11




QD14




XM

Assembly for handpieces with and .

(Distal end of handpiece or attachment is pointed away from user.)

1. Move handpiece knurled knob to "RUN" () position.
2. Slide attachment over distal end of handpiece. Pull attachment towards handpiece, and rotate attachment to right approximately one quarter turn until fully seated. Handpiece knurled knob moves up to seat next to attachment.
3. Pull retaining sleeve on angle attachment towards handpiece and rotate to left.
4. Insert dissection tool into angle attachment. Rotate dissection tool slowly until it is fully seated. Release retaining sleeve on angle attachment by rotating retaining sleeve to right.

 Handpiece is now fully functional.

5. Pull dissection tool to ensure proper engagement.
6. Attachment and dissection tool are secure.

Disassembly

1. Pull retaining sleeve on angle attachment towards handpiece and rotate to left.
2. Remove dissection tool from distal end of attachment.
3. Move handpiece knurled knob to "SAFE" position.
4. Rotate attachment to left approximately one quarter turn, and remove from distal end of handpiece.

Micro Dissection Attachment

MDA Micro Dissection Attachment

⚠ Important: The Micro Dissection Attachment is to be used only for delicate bone dissection applications. Excessive force or side loads during use can cause rapid temperature increase at the distal end of the attachment. Always use copious irrigation with this attachment.

MM

Assembly for handpieces with “SECURE” and “RELEASE”

1. Hold MDA by drive shaft. Insert dissection tool into MDA attachment until cutter seats in attachment (drops slightly proximally and stops rotating). This is best accomplished by pushing slightly while rotating.
2. Check security of dissection tool by pulling on dissection tool shaft.
3. Pull down on handpiece knurled knob and slide MDA Attachment over distal end of handpiece until fully seated. Rotate attachment to right approximately one quarter turn.
4. Release handpiece knurled knob to “SECURE” position.

⚠ Handpiece is now fully functional.


Disassembly

1. Move handpiece knurled knob to “RELEASE” position.
2. Slowly rotate dissection tool in alternating directions while pulling distally.
3. Rotate attachment to left approximately one quarter turn and remove from distal end of handpiece.



XM

Assembly for handpieces with and .

1. Hold MDA by drive shaft. Insert dissection tool into MDA attachment until cutter seats in attachment. (Drops slightly proximally and stops rotating.) This is best accomplished by pushing slightly while rotating.
2. Pull dissection tool to ensure proper engagement.
3. Move handpiece knurled knob to “RUN” () position.
4. Slide attachment over distal end of handpiece. Pull attachment towards handpiece, and rotate attachment to right approximately one quarter turn until fully seated. Handpiece knurled knob moves up to seat next to attachment.
5. Attachment and dissection tool are secure.

⚠ Handpiece is now fully functional.

Disassembly

1. Move handpiece knurled knob to “SAFE” position.
2. Slowly rotate dissection tool in alternating directions while removing distally.
3. Rotate attachment to left approximately one quarter turn and remove from distal end of handpiece.

Minimal Access Attachments and Dissection Tool Assembly

Note: Instructions are the same for straight or angled drivers and for straight or curved bearing sleeves in both 10 cm and 15 cm lengths. The four words labeled on the drivers mean the following:

CUTTER – Insert or remove dissection tool.

RUN – Attachment and dissection tool are now functional.

ADJUST – Bearing Sleeve can be moved 3 mm forward or back.

TUBE – Insert or remove bearing sleeve.

⚠ Important: The Minimal Access Attachments are to be used only for delicate bone dissection applications. Excessive force or side loads during use can cause rapid temperature increase at the distal end of the attachment. Always use copious irrigation with this attachment.



MA-D20BM	20° Angle Driver (for Black Max)
MA-D20	20° Angle Driver (for XMax and microMax)
MA-DRIVER	Straight Driver (for XMax and microMax)

BM

Black Max 20° Angle Driver Installation

1. Ensure locking lever is up.
2. Align notch on BM 20° angle driver with “RELEASE” arrow on handpiece while inserting driveshaft of BM 20° angle driver into handpiece. Rotate BM 20° angle driver to right until notch is aligned with “SECURE” arrow on handpiece. Push locking lever down to locked position.
3. Gently twist back and forth to ensure driver is locked into position.

XM

XMax Driver Installation

1. Ensure handpiece is in “RUN” (🔒) or “SECURE” position.
2. Insert driveshaft of driver into distal end of handpiece, push and rotate to right until it stops. Listen for a click as handpiece knurled knob moves up to seat next to driver.
3. Gently twist back and forth to ensure driver is locked into position.

MM

microMax Driver Installation

1. Ensure knurled knob on handpiece is in “RELEASE” position.
2. Insert driveshaft of driver into distal end of handpiece and rotate to right until it stops. Rotate knurled knob on handpiece to “SECURE” position.
3. Gently twist back and forth to ensure driver is locked into position.

Bearing Sleeve and Dissection Tool Assembly (BM 20° Angle Driver, 20° Angle Driver, or Straight Driver)

MA-10S	Bearing Sleeve, 10 cm Straight
MA-10C	Bearing Sleeve, 10 cm Curved
MA-15S	Bearing Sleeve, 15 cm Straight
MA-15C	Bearing Sleeve, 15 cm Curved
MA-15ST	Bearing Sleeve, 15 cm Straight Taper
MA-19ST	Bearing Sleeve, 19 cm Straight Taper

1. Rotate knurled knob on driver to line up arrow with "TUBE."
2. Fully insert desired bearing sleeve into distal end of driver by lining up black line on bearing sleeve with black line on driver.
3. Rotate knurled knob to line up arrow with "CUTTER". Check security of bearing sleeve by pulling distally.
4. Insert dissection tool into distal tip of bearing sleeve. Rotate dissection tool slowly until it is fully seated. Rotate knurled knob on driver to line arrow with "RUN."

Note: Additional force is required to seat dissection tool into curved bearing sleeve.

5. Check security of dissection tool by pulling distally on dissection tool head.
6. Bearing sleeve and dissection tool are secure.

Dissection Tool Disassembly

1. Rotate knurled knob on driver to line up arrow with "CUTTER." Remove dissection tool from distal end of attachment.

Bearing Sleeve Adjustment

1. Rotate knurled knob on driver to line up arrow with "ADJUST."
2. Pull bearing sleeve either 3 mm distally or proximally to desired exposure.
3. Rotate knurled knob on driver to line up arrow with "RUN."

Bearing Sleeve Disassembly

1. Rotate knurled knob on driver to line up arrow with "TUBE." Remove bearing sleeve distal end of driver.

BM MM

Driver Removal for Black Max and microMax

1. Driver removal is reversal of installation instructions.

XM


Driver Removal for XMax

1. Move handpiece knurled knob to "SAFE" position.
2. Rotate attachment to left approximately one quarter turn, and remove from distal end of handpiece.

Micro Curved Attachment (MCA) and Curved Burr Support Sleeves (Curved Burr)

Micro Curved Attachment Assembly

Side View

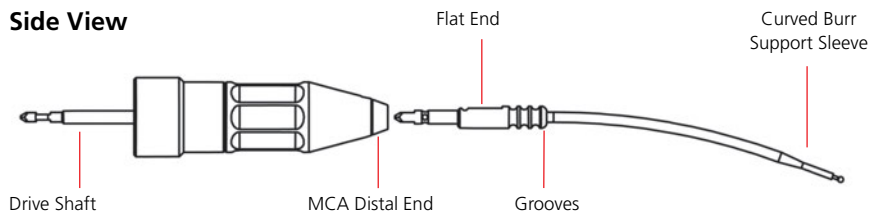

Note: Distal end of handpiece and attachment are pointed away from the user.

MCA Installation

MCA Micro Curved Attachment

XM

MCA Installation for XMax Handpieces

1. Ensure handpiece is in "RUN" (🔒) or "SECURE" position.
2. Insert MCA's drive shaft into distal end of handpiece, push and rotate to right until it stops. Listen for a click as handpiece knurled knob moves up to seat next to MCA.
3. Gently twist back and forth to ensure MCA is locked into position.


MM

MCA Installation for microMax Handpieces

1. Ensure knurled knob on handpiece is on "RELEASE" position.
2. Insert MCA's Drive Shaft into distal end of handpiece, push and rotate to right until it stops. Rotate knurled knob on handpiece to "SECURE" position.
3. Gently twist back and forth to ensure MCA is locked into position.

Curved Burr Support Sleeve Assembly

Top View

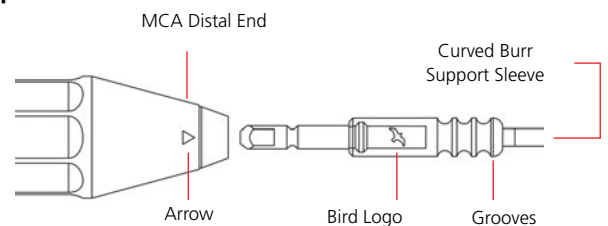

1. Line up arrow located on the distal end of MCA with flat end (bird logo) of Curved Burr, push until fully seated. Listen for a click (see illustration).
2. Gently pull Curved Burr to ensure it is locked into position.

Curved Burr Support Sleeve Disassembly

1. Remove Curved Burr from distal end of MCA by grasping Curved Burr at grooves (see illustration).

XM

MCA Disassembly for XMax Handpieces

1. Move handpiece knurled knob to "SAFE" position.
2. Rotate MCA to left approximately one quarter turn, and remove from distal end of handpiece housing.

MM

MCA Disassembly for microMax Handpieces

1. MCA disassembly is the reverse of assembly.

MicroSaws and Small Attachments Assembly

The microSaws were designed for small bone dissection. The microSaw Drivers provide direct connection to the Black Max, XMax, and microMax Systems. The Drivers also enable rapid exchange of saw head styles through a unique quick-disconnect system.

BM-DRIVER



Keyless Driver Installation

BM

Black Max Keyless Driver Installation

BM-DRIVER Keyless Driver

1. Ensure locking lever is up.
2. Align notch on keyless driver with "Release" arrow on handpiece while inserting drive shaft of Keyless Driver into handpiece. Rotate keyless driver clockwise until notch is aligned with "Secure" arrow on handpiece. Push locking lever down to locked position.

DRIVER



Disassembly

1. Raise locking lever.
2. Rotate BM-DRIVER counterclockwise and remove from handpiece.

XM MM

XMax and microMax, Keyless Driver Installation

DRIVER Keyless Driver

1. Ensure knurled knob on handpiece is in "Run" or "Secure" position.
2. Insert drive shaft of keyless driver into handpiece and rotate clockwise until locked. Gently twist back and forth to ensure it is locked into position.

Disassembly

1. Pull knurled knob down and twist counterclockwise until it seats into position.
2. Rotate DRIVER counterclockwise and remove.

Oscillating, Sagittal, and Reciprocating Saw Attachment Assembly

S-SAW	Sagittal microSaw Attachment
R-SAW	Reciprocating microSaw Attachment
O-SAW	Oscillating microSaw Attachment

1. Insert attachment drive shaft into distal end of keyless driver. Rotate until three tabs on attachment engage three slots in keyless driver.
2. Continue to rotate counterclockwise while pushing attachment into keyless driver until attachment locks into position and release ring snaps into position. When fully engaged, contours on keyless driver body, release ring, and saw attachment will be in alignment.

Oscillating Saw Blade Assembly

1. Press down on release button to open saw blade mounting plates.
2. Insert saw blade into opening and align saw blade hub with locking pins in mounting plates. Oscillating saw blades can be aligned in 45° increments.
3. Release button and ensure mounting plates are firmly seated on saw blade without gaps.

Sagittal Saw Blade Assembly

1. Press down on release button to open saw blade mounting plate.
2. Insert saw blade into opening thru slot on mounting plate and align saw blade hub with locking pins in mounting plate.
3. Release button and ensure mounting plate is firmly seated on saw blade without gaps.

Reciprocating Saw Blade Installation

1. Rotate chuck counterclockwise until chuck has opened sufficiently to accept a saw blade.
2. If blade has a shank with a shaft, insert shaft of blade in chuck jaws until it is fully inserted and tighten chuck knob clockwise until snug.
3. If blade has a flat shank, insert blade into slots in chuck until it is fully inserted and tighten chuck knob clockwise until snug.


S-SAW



R-SAW



O-SAW

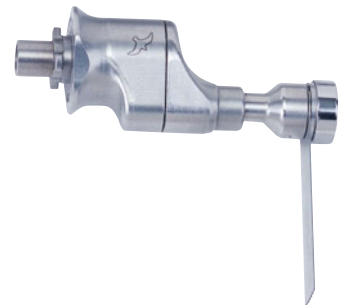

Saw Blade Removal

1. Saw blade removal is reverse of installation.

Disassembly

Saw and Small Attachment Removal

1. With one hand supporting attachment, rotate release ring on keyless driver with other hand counterclockwise until attachment is released.

J-Latch and Keyless Jacobs Chuck Attachment Assembly

BM-DRIVER	Keyless Driver
DRIVER	Keyless Driver
SA-JACOBS	Jacobs Chuck
SA-JLATCH	J-latch Chuck

Note: Anspach does not supply dissection tools for SA-JLATCH or SA-JACOBS small attachments. Keyless driver refers to either BM-DRIVER or DRIVER.

1. Insert attachment drive shaft into distal end of keyless driver. Rotate until three tabs on attachment engage three slots in keyless driver.
2. Continue to rotate counterclockwise while pushing attachment into keyless driver until attachment locks into position and release ring snaps into position.
3. When fully engaged, contours on keyless driver body, release ring, and small attachment will be in alignment.

BM-DRIVER



DRIVER



SA-JLATCH



J-Latch Dissection Tool Assembly

1. Pull back distal sleeve in direction of arrow ▼ on SA-JLATCH attachment and hold it down to insert dissection tool into distal end of attachment.
2. Rotate dissection tool slowly until fully seated.
3. Release distal sleeve and check security of dissection tool by pulling distally on dissection tool.
4. Dissection tool and attachment are secure.

SA-JACOBS



J-Latch Dissection Tool Removal

1. Pull back distal sleeve in the direction of arrow ▼ on SA-JLATCH and hold it down while pulling dissection tool out distally for removal.
2. Release the distal sleeve.

Keyless Jacobs Chuck Dissection Tool Assembly

1. Press down on center ▲ of release button located on SA-JACOBS attachment while rotating knurled knob on attachment clockwise until fully opened.
2. Insert dissection tool into distal end of attachment until it touches bottom.
3. Rotate knurled knob counterclockwise while centering cutter on attachment until tight; release button.
4. Check security of dissection tool by pulling distally on dissection tool.
5. Dissection tool and attachment are secure.

⚠ Caution: Do not press down on release button while attachment is in use.

Keyless Jacobs Chuck dissection tool removal

1. Press down on center ▲ of release button located on SA-JACOBS attachment while rotating knurled knob on attachment clockwise until fully opened.
2. Pull dissection tool out distally for removal.
3. Release button.

Handheld Attachments Assembly

Chuck Driver

BC-SYSTEM Chuck Driver System (for Black Max)

C-SYSTEM Chuck Driver System (for microMax)

This attachment provides the ability to use your Anspach instrument system to drive twist drills up to 4 mm in diameter. (Approx. 3,500 rpm)

⚠ Important: Do not use the Chuck Driver with the XMax.

BM

Black Max Chuck Driver System Assembly for handpieces with "SECURE" and "RELEASE"

1. The words "SECURE" and "RELEASE" engraved on side of handpiece refer to locking and unlocking attachment on handpiece respectively. Lift lever on handpiece.
2. Align notch on Chuck Driver with "RELEASE" on handpiece.
3. Turn Chuck Driver so notch is aligned with "SECURE" on handpiece.
4. Press down lever on handpiece.
5. The Chuck Driver should be used with a drill guide for additional stability.

MM

microMax Chuck Driver System for handpieces with "SECURE" and "RELEASE" Assembly

1. Move handpiece knurled knob into "SECURE" position.
2. Slide Chuck Driver System over distal end of handpiece until fully seated. Rotate attachment to right approximately one-quarter turn.
3. Check security of attachment by pulling distally.
4. The Chuck Driver should be used with a drill guide for additional stability.

Disassembly

1. For removal, move Chuck Driver System attachment slightly proximally while rotating to left.



Sagittal Saws

B-SAGITTAL	Black Max Sagittal Saw
MM-SAGITTAL	Sagittal Saw (for XMax and microMax)

The B-SAGITTAL and MM-SAGITTAL operate at 120 psi, providing 20,000 cycles per minute while it oscillates through a blade arc of 4 degrees.



BM

Black Max Sagittal Saw Assembly for handpieces with "SECURE" and "RELEASE"

1. The words "SECURE" and "RELEASE" engraved on side of handpiece refer to locking and unlocking attachment on handpiece respectively. Lift lever on handpiece.
2. Align notch on Sagittal Saw with "RELEASE" on handpiece.
3. Turn Sagittal Saw so notch is aligned with "SECURE" on handpiece.
4. Press down lever on handpiece.
5. For B-SAGITTAL, see Blade Assembly.

MM


microMax Sagittal Saw for handpieces with "SECURE" and "RELEASE"

1. Remove red storage cover. Pull down retaining sleeve of handpiece and rotate to left.
2. Slide Sagittal Saw over distal end of handpiece until fully seated.
3. Return handpiece retaining sleeve to original position by rotating to right and pushing distally.
4. Sagittal Saw is now secure with handpiece.
5. See Blade Assembly.

XM

Sagittal Saw Assembly and Disassembly for handpieces with and .

(Distal end of handpiece or attachment is pointed away from user.)

1. Remove red storage cover. Move handpiece knurled knob to "RUN"  position.
2. Slide Sagittal Saw over distal end of handpiece. Pull Sagittal Saw towards handpiece, and rotate to right approximately one-quarter turn until fully seated. Handpiece knurled knob will move up to seat next to Sagittal Saw.
3. Sagittal Saw is secure.
4. See Blade Assembly.

Disassembly

1. Move handpiece knurled knob to "SAFE" position.
2. Rotate Sagittal Saw to left approximately one-quarter turn, and remove from distal end of handpiece.

Blade Assembly (B-SAGITTAL/MM-SAGITTAL)

1. Raise and fully extend extension handle on saw blade attachment mechanism.
2. Rotate extension handle counter-clockwise to open saw blade mounting plates.
3. Insert saw blade into opening and align saw hub with locking pins in mounting plates. Blades can be aligned in 45 degree increments.
4. Rotate extension handle clockwise to secure saw blade and lower extension handle into closed position.

Compact SpeedReducer

- CSR60BM Perforator Driver with Hudson End (for Black Max)
- CSR60 Perforator Driver with Hudson End (for XMax and microMax)

This Perforator Driver attachment allows Hudson style cranial perforators to be used with Black Max, XMax, and microMax Systems (60:1 Ratio, approximately 1,300 rpm.)

Refer to the Cranial Perforator manufacturer's operating instructions for rotational speed requirements.

BM

Black Max Compact SpeedReducer Assembly for handpieces with "SECURE" and "RELEASE"

1. Remove red storage cover.
2. Lift lever on handpiece.
3. Slide Compact SpeedReducer over distal end of handpiece until fully seated. Rotate Compact SpeedReducer to right approximately one-quarter turn.
4. Press down lever on handpiece.
5. Pull back retaining sleeve; insert Hudson-end of cranial perforator and release retaining sleeve.

Disassembly

1. Lift lever on handpiece.
2. Rotate Compact SpeedReducer to left approximately one-quarter turn and remove device.
3. Replace red storage cover.

MM

Compact SpeedReducer Assembly for handpieces with "SECURE" and "RELEASE"

1. Remove red storage cover.
2. Move handpiece knurled knob to "SECURE" position.
3. Slide Compact SpeedReducer over distal end of handpiece until fully seated. Rotate Compact SpeedReducer to right approximately one-quarter turn until knurled knob locks in place.
4. Pull back retaining sleeve, insert Hudson-end of cranial perforator and release retaining sleeve.

Disassembly

1. Move handpiece knurled knob to "RELEASE" position.
2. Rotate Compact SpeedReducer to left approximately one-quarter turn and remove device.
3. Replace red storage cover.



CSR60

XM

Compact SpeedReducer Assembly for handpieces with "SAFE" and "RELEASE"

(Distal end of handpiece or attachment is pointed away from user.)

1. Remove red storage cover.
2. Move handpiece knurled knob to "RUN" position.
3. Slide SpeedReducer over distal end of handpiece. Pull SpeedReducer towards handpiece and rotate to right approximately one-quarter turn until fully seated. Handpiece knurled knob will move up to seat next to SpeedReducer.
4. Pull SpeedReducer retaining sleeve towards handpiece. Insert Hudson-end of cranial perforator and release retaining sleeve.

Disassembly

1. Pull SpeedReducer retaining sleeve towards handpiece and remove cranial perforator from distal end of SpeedReducer.
2. Move handpiece knurled knob to "SAFE" position.
3. Rotate SpeedReducer to left approximately one-quarter turn and remove from distal end of handpiece.
4. Replace red storage cover.

Anspach Powered Irrigation System

Irrigation Pump Assembly

IRR-PUMP-II Powered Irrigation Pump

1. Insert AC cord into Irrigation System and AC cord plug into ground power outlet (100-240 VAC, 50-60 Hz, 0.30–0.15 A).
2. Connect AUTOLUBE-III-IRR to Irrigation Pump with footswitch cord.
3. Mount irrigation pump on IV Pole using clamp on rear of pump.
4. Hang irrigation bag onto IV Pole.

Irrigation Tubing Set-up

IRRIGATE-TUBE Sterile Tubing and Hoseclips for Irrigation System

IRRIGATE-TUBE-HF High Flow Sterile Tubing and Hoseclips for Irrigation System


1. Lift top of pump head and insert large diameter tubing. Refer to diagram on irrigation pump housing to ensure tubing is routed properly. (Smaller diameter tubing flows towards Synthes Anspach handpiece.)
2. Close lid on pump head.
3. Insert bayonet end of tubing into irrigation bag.

⚠ Important: Tubing can disconnect from connectors without warning if occluded. DO NOT step on, set equipment on, pinch, kink, clamp, or otherwise occlude tubing during use.

Irrigation Attachment Clip Assembly

Refer to "Ordering Information" section (page 41) for part numbers.

1. Slide irrigation tubing onto proximal end of Irrigation Attachment Clip.
2. Attach Irrigation Attachment Clip to Anspach Handpiece.
3. Secure tubing to handpiece hose using Hose Clips, which are included with tubing.

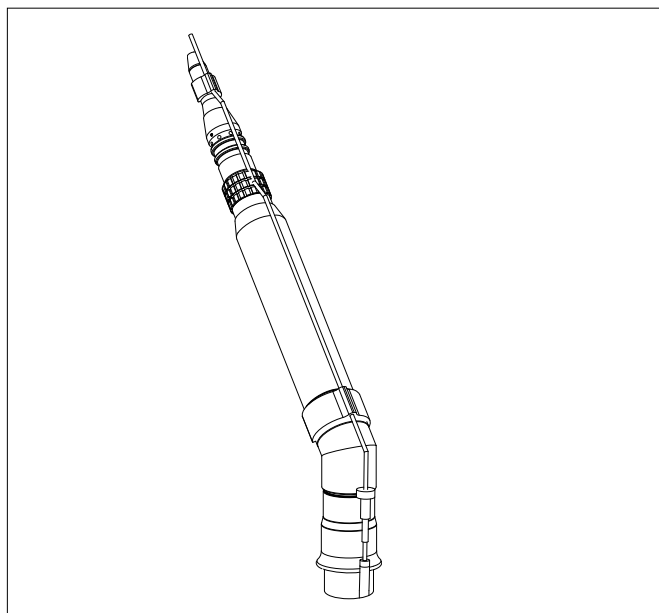

Irrigation Pump Operation

1. Turn Main power switch to ON position.
2. Adjust flow dial on front panel to control flow.
3. Depress foot pedal on AUTOLUBE-III-IRR.


⚠ Important: United States Federal law restricts this device to sale by or on order of a physician or other licensed health-care provider. Do not sterilize, and do not allow liquid into pump. Surgeon is responsible for learning proper techniques in use of equipment; improper use may cause serious injury to user or patient or damage to system. Instrument operator and all operating room personnel must wear eye protection. Inspect system for damage before using; do not use if damage is evident. Do not modify ground or power cord. Do not operate in the presence of flammable anesthetic mixtures. Do not operate with pumphead open. Disposable tubing, attachment clip, and hose clip are for SINGLE USE ONLY.

⚠ Caution: Power source should comply with applicable IEC, CEC, and NEC requirements. Grounding reliability can only be achieved when this equipment is connected to a receptacle marked "HOSPITAL GRADE."

⚡ With respect to protection against electric shock, equipment is Class I type B.

If fuse replacement is necessary, use only Time Delay fuse rated 2.5 A, 250 V.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee interference will not occur in a particular installation.

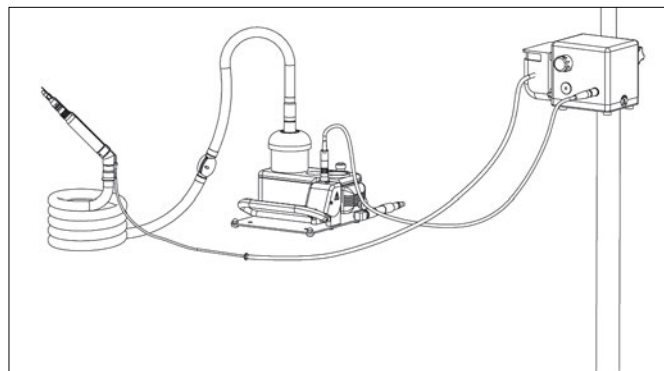

If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Re-orient or relocate the device receiving interference
 - Increase separation between the equipment
 - Connect the equipment into an outlet different from that which the other device(s) are connected.
 - Consult the manufacturer or field service technician for assistance.
1. With respect to electric shock, fire and mechanical hazards only, in accordance with UL 2601-1. [3AF3]
 2. With respect to electric shock, fire, mechanical and other specified hazards only in accordance with CAN/CSA C22.2 NO. 601.1, (CAN/CSA 601.2XX, if applicable) Medical equipment certified for Canada. [3AF3].

Cleaning and Maintenance

⚠ Important: Prior to use, and while cleaning, visually inspect the instrument(s). Do not use the instrument(s) if damaged.

AutoLube

⚠ Caution: DO NOT IMMERSE OR STERILIZE! DO NOT allow liquid to enter AutoLube, except as specified in the operating manual for adding Anspach oil.

1. Clean AutoLube Foot Control by wiping with nonabrasive cloth and disinfectant or mild detergent and water after each case.
2. Store in upright position.
3. Check oil level and add Anspach oil and filter when level is below "MIN Oil Level" line.


Attachments

⚠ Caution: Wear eye protection.

Note: "DO NOT IMMERSE" as marked on attachments is for OR personnel only.

DO NOT use ultrasonic equipment or corrosive or harsh chemical soaps. Do not rinse attachment with saline solution.

1. Remove the attachment and dissection tool from the handpiece.
2. Fully immerse the attachment in an alkaline cleaner (non-chlorinated detergent), prepared as described on the product label, at room temperature, in a suitable container and agitate for 15 seconds.

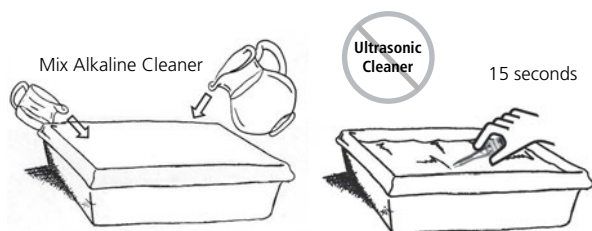


Item Numbers 3 and 4 refer only to the Straight, Craniotome, Specialty, and QD Angle Attachments. For any other attachments, proceed to Item Number 5.

3. Gently insert and remove Anspach Attachment Cleaning Brush (ACB) wetted with alkaline cleaner through the distal (tip) opening of the attachment as many times as necessary to remove any foreign debris. DO NOT FORCE the brush into or through the attachment.


⚠ Caution: DO NOT insert anything into the attachment except the ACB as specified.




4. Remove the brush (ACB).
5. Rinse the attachment in a suitable container filled with water and agitate for 15 seconds.

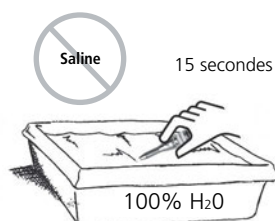

6. Repeat steps 1 and 2 if there is evidence of soils or residuals on the attachment surface.
7. After cleaning and rinsing, fully immerse the attachment in instrument milk (Non-silicone Based Medical Lubricant) prepared as described on the product label, at room temperature, in a suitable container and agitate for 15 seconds.

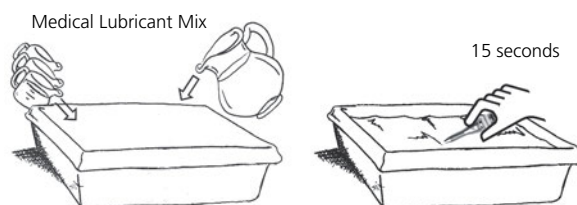

-
8. Remove the attachment and allow it to drain.
 - a. DO NOT rinse out instrument milk (lubricant).
 - b. DO NOT apply mineral oil or other lubricants, which may cause the attachment to overheat.

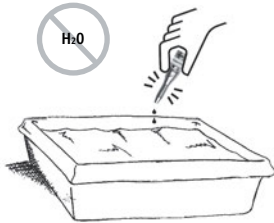

Note: For best use results, the recommended alkaline cleaner and lubricant should be used. Isopropyl alcohol, Viraguard® or 70% ethanol may be substituted for the alkaline cleaner if necessary.

Dissection Tools

⚠ **Caution:** Disposable. Single use only.

iMRI Foot Control

⚠ **Caution:** DO NOT IMMERSE OR STERILIZE!

1. Clean iMRI Foot Control by wiping with nonabrasive cloth and disinfectant or mild detergent and water after each case.
2. Dry thoroughly with nonabrasive cloth.

Irrigation Pump

⚠ **Caution:** DO NOT IMMERSE OR STERILIZE! Do not allow liquid to enter Irrigation Pump.

1. Disconnect all power from pump and remove and discard tubing, Attachment Clip, and Hose Clip before cleaning.
2. Clean pump by wiping with nonabrasive cloth and disinfectant or mild detergent and water.
3. Dry thoroughly with nonabrasive cloth.

Minimal Access Attachment Bearing Sleeves

Follow the Attachment cleaning protocol for the bearing sleeves. The Minimal Access Attachment Bearing Sleeves are non-repairable. Reuse until bearing wear is detected and then replace. No bearing sleeve service or repair will be provided.

Handpieces

⚠ Caution: Wear eye protection. DO NOT IMMERSE! Do not insert brush into handpiece. Do not use saline solution for cleaning. Do not use cleaners containing chlorinated phenols of any concentration. Using cleaners/disinfecting agents containing chlorinated phenols will result in premature hose failure. Do not use oil to lubricate knurled knob.

Prior to sterilization, wipe with a clean, water-dampened cloth and mild detergent (neutral pH). Silicone Spray or Instrument Milk may be added to lubricate the handpiece knurled knob. Silicone spray may be used on the hose to prolong its flexibility and to facilitate foreign particle removal.

The handpiece should be tested by running it with an attachment and dissection tool prior to each use for two minutes to ensure proper lubrication. If the distal tip and main body of the handpiece are uncomfortably hot to the touch, return the instrument to Anspach for servicing. If occlusion of the handpiece hose is suspected, or if the pressure relief valve is activated, return instrument to Anspach for servicing.

⚠ Caution: Handpiece must not be exposed to ingress of water or to severe physical trauma; degradation of unit function and/or performance may occur.

Sterilization Trays

Sterilization trays can be cleaned with water and a mild detergent.

Sterilization

AutoLube

⚠ The AutoLube should not be sterilized.

iMRI Foot Control

⚠ The iMRI Foot Control should not be sterilized.

Irrigation Pump

⚠ The Irrigation Pump should not be sterilized.

Handpieces and Attachments

Metal devices, tools and equipment are constructed of materials unaffected by normal environmental conditions of current standard sterilization means, when proper operational techniques are employed.

Effectiveness of sterilization equipment or sterilization processes are directly dependent upon numerous factors beyond Synthes Anspach's control including among other things; sterilization means, processes and wrapping techniques employed, brand, model and condition of sterilization equipment, care and maintenance techniques employed and operator knowledge and experience.

Before sterilizing: Ensure sterilization equipment is in proper working order as specified by the manufacturer. Assure equipment manufacturer's instructions are properly employed by trained and qualified personnel. Assure actual cycle employed has been properly validated for the device(s)/ load configuration being processed and appropriate sterilization indicator devices are included for each process and cycle.

Synthes Anspach cannot anticipate all possible equipment, processes and/or conditions that may be encountered. The suggested operation conditions are to be considered as a starting point for determination of the overall process capability, without regard for type or condition of equipment used or methods, techniques or practices employed by the user. Use of proper sterilization indicator devices is strongly recommended. It is recommended that a drying cycle be included to avoid possible adverse effects caused by exposure to condensation.

Recommended Sterilization Exposure Times

1. Steam Autoclave/Gravity Air Displacement
132°C ± 1°C (270°F ± 2°F)
15 minutes wrapped or unwrapped
30 minute dry time
2. Steam Autoclave/Prevacuum
132°C ± 1°C (270°F ± 2°F)
3 minutes wrapped or unwrapped

(Black Max lever guard must be in the open position.)

Process parameters which include higher temperatures up to 138°C do not affect performance of the product and do not violate the product warranty.

Warranty. It is recommended a drying cycle be included to avoid possible adverse effects caused by exposure to condensation.

Pneumatic Handpieces (Black Max, XMax, microMax)

Problem	Possible Cause	Solution
Lack of power or excessive handpiece noise	Inadequate lubrication	Add Anspach oil to AutoLube
	Inadequate operating pressure	Check pressure on AutoLube. Extra long extension hose from air source could reduce operating pressure Check Schraeder Connection. Check for handpiece hose occlusion
	Muffler not connected properly	Check connection between muffler and AutoLube. Muffler must be inserted past red line
	Handpiece hose damaged	Return to Synthes Anspach for service
	Faulty internal component	Return to Synthes Anspach for service.
	Pressure Relief Valve activated	Check for handpiece occlusion, remove the source of the occlusion. Return to Synthes Anspach for service
Handpiece does not rotate when foot pedal is activated	Motor stalling	Rotate dissection tool, then activate foot pedal
	Faulty attachment	Clean or replace attachment
	Motor Lock engaged	Unlock handpiece
	Faulty internal component	Return to Synthes Anspach for service
Handpiece vibration or extremely hot	Inadequate lubrication	Add Anspach oil to AutoLube
	Motor out of balance	Return to Synthes Anspach for service
Excessive vibration of dissection tool	Dissection tool may be bent	Replace with new dissection tool
	Dissection tool may not be fully seated	Reassemble dissection tool and attachment
	Improper attachment and dissection tool combination	Only use correct dissection tool with appropriate attachment
	Possible attachment bearing damage	Return attachment to Synthes Anspach for service.

Problem	Possible Cause	Solution
Black Max Lever will not lower	Dissection tool not seated properly	Reassemble dissection tool and attachment
	Debris built up inside handpiece	Remove attachment, lift lever, and tap distal tip of handpiece against hard surface until debris is dislodged
	Broken dissection tool in handpiece	Lift lever, tap distal tip of handpiece against hard surface until dissection tool is dislodged
XMax/microMax knurled knob will not secure	Dissection tool not seated properly	Reassemble dissection tool and attachment
	Debris built up inside handpiece	Remove attachment and tap distal tip of handpiece against hard surface until debris is dislodged
	Broken dissection tool in handpiece	Remove attachment and tap handpiece against hard surface until dissection tool is dislodged
Attachment is hot	Debris lodged inside attachment	Clean attachment using Anspach attachment cleaning instructions located in this manual
	Possible bearing damage	Return attachment to Synthes Anspach for service
Craniotome attachment is bent	Excessive force used in operation	Do not use, replace attachment
Inoperative Foot Control	Oil leaking out of AutoLube	Replace AutoLube filter, check oil level. Do not overfill AutoLube
	Faulty air connection	Check connection at air source
	AutoLube was sterilized or submerged in fluids	Return to Synthes Anspach for service.
	Water in the AutoLube	Return to Synthes Anspach for service

Ordering Information

XM MM

XMax and microMax Attachments

SHORT	5 cm Short Attachment
MEDIUM	8 cm Medium Attachment
LONG	11 cm Long Attachment
LONG-S	10.5 cm Long Attachment
LONG-01	10.5 cm Long Attachment, Non-tapered End
MIA16	16 cm Minimally Invasive Attachment
SHORT-HD	5 cm Heavy Duty Short Attachment
MEDIUM-HD	8 cm Heavy Duty Medium Attachment
LONG-HD	12.4 cm Heavy Duty Long Attachment
XL-HD	20.2 cm Heavy Duty Extra Long Attachment
ADG	7.2 cm Adjustable Drill Guide
CDA	6.8 cm Controlled Depth Attachment
CRANI-A	6.5 cm Adult Craniotome
CRANI-A-01	6.5 cm Adult Craniotome, Thin
CRANI-P	6.5 cm Pediatric Craniotome
CRANI-L	7.5 cm Large Craniotome
CRANI-A-R	6.5 cm Rotating Adult Craniotome
CRANI-L-R	7.5 cm Large Rotating Craniotome
MA-DRIVER	Straight Driver
MA-D20	20° Angle Driver
MA-15S	Bearing Sleeve, 15 cm Straight
MA-15C	Bearing Sleeve, 15 cm Curved
MA-15ST	Bearing Sleeve, 15 cm Straight Taper
MA-19ST	Bearing Sleeve, 19 cm Straight Taper
MA-10S	Bearing Sleeve, 10 cm Straight
MA-10C	Bearing Sleeve, 10 cm Curved
MCA	Micro Curved Attachment
ORANGE-45	45° Contra Angle Attachment
ORANGE-90	90° Contra Angle Attachment
QD8	8 cm Quick Disconnect Angle Attachment
QD8-S	7.5 cm Quick Disconnect Angle Attachment
QD11	11 cm Quick Disconnect Angle Attachment
QD11-S	10.5 cm Quick Disconnect Angle Attachment
QD14	14 cm Quick Disconnect Angle Attachment
QD14-S	13.5 cm Quick Disconnect Angle Attachment
CSR60	Perforator Driver with Hudson End (60:1 Ratio)
DRIVER	Keyless Driver
MM-SAGITTAL	Sagittal Saw
O-SAW	Oscillating microSaw Attachment
R-SAW	Reciprocating microSaw Attachment
S-SAW	Sagittal microSaw Attachment
SA-JACOBS	Jacobs Chuck
SA-JLATCH	J-latch Chuck

BM

Black Max Attachments

B-SILVER	6.1 cm Attachment
B-TURQUOISE	5.9 cm Attachment
B-VIOLET	8.2 cm Attachment
B-BLUE-S	11.25 cm Attachment
B-TURQ-L	12.6 cm Attachment
B-GREY-MIA16	16 cm MIA
B-GREY-MIA20	20 cm MIA
B-BLACK	19 cm Attachment
B-RED	26.65 cm Attachment
B-MRA-S	9.4 cm Revision Attachment
B-MRA-M	16.5 cm Revision Attachment
B-MRA-L	22 cm Revision Attachment
B-ORANGE-45	Contra Angle Attachment (Multiple Angle)
B-ORANGE-90	Right Angle Attachment (90°)
B-GREY-ADG	8.7 cm Adjustable Drill Guide Attachment
B-TURQ-CDA	Controlled Depth Attachment
B-GOLD	Large Craniotome
B-GOLD-R	Rotating Large Craniotome
B-GREEN	Standard Craniotome
B-GREEN-R	Rotating Green Craniotome
B-TURQ-X	Pediatric Craniotome
B-QD8	8 cm Angle Attachment
B-QD8-S	7.5 cm Angle Attachment
B-QD11	11 cm Angle Attachment
B-QD11-S	10.5 cm Angle Attachment
B-QD14	14 cm Angle Attachment
MA-D20-BM	20° Angle Driver
MA-15S	Bearing Sleeve, 15 cm Straight
MA-15C	Bearing Sleeve, 15 cm Curved
MA-10S	Bearing Sleeve, 10 cm Straight
MA-10C	Bearing Sleeve, 10 cm Curved
MA-15ST	Bearing Sleeve, 15 cm Straight Taper
MA-19ST	Bearing Sleeve, 19 cm Straight Taper
B-SAGITTAL	Black Max Sagittal Saw
BM-DRIVER	Keyless Driver
O-SAW	Oscillating microSaw Attachment
R-SAW	Reciprocating microSaw Attachment
S-SAW	Sagittal microSaw Attachment
CSR60BM	Perforator Driver with Hudson End
SA-JACOBS	Jacobs Chuck
SA-JLATCH	J-latch Chuck

iMRI

iMRI Instrument System

IMRI-MOTOR	iMRI Motor
IMRI-FOOT-CTRL	iMRI Air Control
IMRI-SHORT	iMRI Short Attachment
IMRI-SHORT-HD	iMRI Short HD Attachment
IMRI-CRANI-A	iMRI Crani-A Attachment
IMRI-S-4B	4 mm Fluted Ball
IMRI-S-1004TD	1 mm x 4 mm Twist Drill w/Stop
IMRI-S-1R	1.4 mm x 12.8 mm Fluted Router
IMRI-A-CRN	1.85 mm x 16 mm Fluted Router
IMRI-SHD-6B	6 mm Fluted Ball
IMRI-SHD-6R	1.12 mm x 6.35 mm Fluted Router
IMRI-OIL	iMRI Synthetic Oil

Irrigation System

IRR-PUMP-II	Powered Irrigation Pump
IRRIGATE-TUBE	Sterile Tubing and Hoseclips
IRRIGATE-TUBE-HF	High Flow Sterile Tubing and Hoseclips
IRR-CLIP-10	Irrigation Clip for SHORT
IRR-CLIP-20	Irrigation Clip for QD8, QD8-S, B-QD8
IRR-CLIP-30	Irrigation Clip for SHORT-HD, B-SILVER
IRR-CLIP-40	Irrigation Clip for MEDIUM-HD, LONG-HD, B-BLUE-S, B-VIOLET
IRR-CLIP-50	Irrigation Clip for MEDIUM, LONG, LONG-S, B-TURQ-L
IRR-CLIP-60	Irrigation Clip for QD11, QD11-S, QD14, QD14-S, B-QD11, B-QD11-S, B-QD14, B-QD14-S
IRR-CLIP-70	Irrigation Clip for MIA16, B-GREY-MIA-16, B-GREY-MIA20

BM XM MM

Pneumatic Handpieces

BLACKMAX-N	Swivel/Angle Motor Assembly
BLACKMAX-N-LP	Low Pressure Swivel/Angle Motor Assembly
XMAX	XMax Motor

BM XM MM

Pneumatic Accessories

ACB	Attachment Cleaning Brush
AL-III-IRR-MA7	Foot Control for Irrigation System, MA7 Adapter
AUTOLUBE-III	Foot Control
AUTOLUBE-III-DISS	Foot Control, DISS Adapter
AL-III-DISS-20	Foot Control with 20' Hose, DISS Adapter
AUTOLUBE-III-FG	Foot Control with Foot Guard
AUTOLUBE-III-FG7	Foot Control with Foot Guard, MA7 Adapter
AUTOLUBE-III-IRR	Foot Control for Irrigation System
AUTOLUBE-III-MA7	Foot Control, MA7 Adapter
AUTOLUBE-III-NK	AutoLube/Foot Control with Nitrogen Keyed Adapter
AUTOLUBE-III-20	Foot Control with 20' Hose
AL-III-30	Foot Control with 30' Hose
AL-III-DISS-30	Foot Control with 30' Hose, DISS Adapter
B-CASE-TRAY	Sterilization Case with Tray
FILTER-II	AutoLube II & III Filters (5/pkg.)
HOSE	20-Foot Air Extension Hose
HOSE-5	5-Foot Air Extension Hose
HOSE-10	10-Foot air Extension Hose
HOSE-10-DISS	10-Foot Air Extension Hose with DISS Adapter
HOSE-15-DISS	15-Foot Air Extension Hose with DISS Adapter
HOSE-20-DISS	20-Foot Air Extension Hose with DISS Adapter
HOSE-DISS	10-Inch Air Extension Hose with DISS Adapter
MM-CASE-TRAY	Sterilization Case with Tray
MM-CTR-RACK	Cutting Burr Rack
OIL-II	Instrument Oil w/1 Replacement Filter for AutoLube -II and -III
PG	Pistol Grip for Black Max
SCHRADER-F	SCHRADER Adapter, Female
XM-CASE	Sterilization Case with Tray

Warranty and Return Policies

The Anspach Effort®, Inc. ("Anspach®") warrants that this product is free of defects in workmanship and material for a period of two (2) years after the date of shipment of the product if used and maintained in accordance with manufacturer's instructions. Anspach reserves the right to invalidate this Warranty if a) the product is used with components not manufactured by Anspach, b) the product is used with components, accessories, or dissection tools reprocessed by other than Anspach, or c) if repairs to the product are not performed by Anspach. Anspach's obligation under this Warranty shall be limited to repair or replacement of the product as determined by Anspach, in its sole discretion. Dissection tools are supplied sterile and are for single use only. Minimal Access Attachment Bearing Sleeves are non-repairable and therefore not subject to the repair or replacement provisions of the Warranty.

THE WARRANTY SET FORTH ABOVE IS THE SOLE AND EXCLUSIVE WARRANTY GIVEN BY ANSPACH WITH RESPECT TO THE PRODUCT. ANSPACH GIVES AND MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, OTHER THAN THAT EXPRESSLY SET FORTH IN THIS PARAGRAPH. NO PERSON IS AUTHORIZED TO GIVE OR MAKE ANY OTHER REPRESENTATION OR WARRANTY OR TO MODIFY THE WARRANTY DESCRIBED ABOVE IN ANY WAY WITHOUT LIMITING THE GENERALITY OF THE FOREGOING, NO IMPLIED WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, TITLE, NONINFRINGEMENT, USAGE OR TRADE, COURSE OF DEALING OR COURSE OF PERFORMANCE IS GIVEN OR MADE BY ANSPACH WITH RESPECT TO THE PRODUCT, AND IN NO EVENT SHALL ANSPACH HAVE ANY LIABILITY OR OBLIGATION WHATSOEVER UNDER OR IN CONNECTION WITH ANY SUCH WARRANTY WITH RESPECT TO THE PRODUCT.

Requests for warranty service shall be made only upon issuance of a Return Material Authorization (RMA) number from Anspach and shipment of the product to Anspach or such other authorized repair center as Anspach may designate. Product for which a RMA number is obtained shall be shipped to Anspach (or other such location as may be mutually agreed upon by Anspach and Customer) by common carrier. Product returned without a return authorization number shall not be afforded warranty service, and all costs associated with such unauthorized returns shall be borne by Customer.

⚠ All Instrument System components returned for servicing or repair should be properly cleaned and sterilized as applicable prior to shipping.

⚠ Transmissible Spongiform Encephalopathies (TSE). Anspach will not authorize or accept the return of products that directly contact patients or are contaminated with a patient's body fluids who is suspected or confirmed with a Transmissible Spongiform Encephalopathies/Creutzfeldt-Jakob Disease (TSE/CJD) diagnosis. Anspach recommends that all Anspach products used on a patient confirmed with a TSE/CJD diagnosis be incinerated. Anspach dissecting tools used on a patient suspected of a TSE/CJD diagnosis must be incinerated.

Contact your Sales Consultant for replacement of product incinerated under this policy or for temporary equipment while original equipment is quarantined. Contact the Anspach Customer Service Department regarding TSE contamination for additional information.



Distributor:
Synthes GmbH
Eimattstrasse 3
CH-4436 Oberdorf
www.synthes.com

ECREP

Anspach Europe Ltd.
Aston Court, Kingsmead Business Park
Frederick Place, High Wycombe
HP11 1LA, United Kingdom



Manufacturer
The Anspach Effort, Inc.
4500 Riverside Drive
Palm Beach Gardens, FL 33410
USA



All technique guides are available as PDF files at
www.synthes.com/lit

CE
0086