We,

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D – 14469 Potsdam / Germany

want to inform you about a recall / field safety corrective action of the products listed in the attachment.

This Field safety corrective action is a precautionary corrective action. There has been neither any incidence nor any near incidence.

During production the mounting personnel stated that some of the pressure adjustment springs to be mounted in the valve for pressure control were outside specification. These springs were scrapped and not assembled.

However all finished products passed the 100% final control without any deviations.

There is a very low suspicion that the adjustments springs of a very few products might not stand the average stress forces put on the spring during lifetime thus negatively affecting the pressure control of the valve.

Because of this suspicion, however improbable it may be, the safety officer has decided to recall the affected products (23pieces) to have them additionally tested. Lots on stock were already tested successfully and 46 pieces passed without any objection.

We intend to test all products in the field in order to prove an correct function of the products.

We would like to stress that there is only a low suspicion for a defect product and that we want to assure 100% medical device safety.

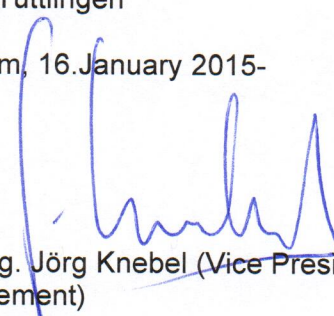
Should the application of a defect spring be confirmed against our expectations, the shunt patient should be observed medically for a period of at least three weeks, if the patient has any shunt disfunction symptoms (overdrainage). Should there be any of these symptoms, the valve needs to be revised.

Please send concerned products back to

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Potsdam, 16. January 2015-

Christoph Miethke GmbH & Co. KG  
Regulatory Affairs

  
Dipl.-Ing. Jörg Knebel (Vice President Quality  
Management)

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Aescualp AG,