



EF015 European Form	Field Safety Notice	Page 1 of 2
-------------------------------	----------------------------	-------------

Urgent FIELD SAFETY NOTICE
Dolomite rollators: Futura, Melody, Legacy, Maxi+, Alpha,
Symphony, Jazz 520 and Jazz 600.
Amendment of User Manual

Date: April 1, 2015

Ref: **XXX**

Important Safety Information:

In the interest of the safety of our customers we would like to bring to your attention certain amendments of the manuals of all models of below Invacare Dolomite rollators:

- Futura
- Melody
- Legacy
- Maxi+
- Alpha Basic and Alpha Advanced
- Symphony
- Jazz 520 and Jazz 600

We are aware of several incidents where a front wheel has fallen off a rollator. We are sure that an improvement to the maintenance procedures can mitigate this risk. Please note that sections of the User Manuals were reviewed and amended during 2013-2014. Especially the statement in the User Manual that the expected service life of these products is five years when used in accordance with safety instructions, maintenance intervals and correct use is an important addition.

Current revisions of the User Manuals for the above rollators can be downloaded from Invacare website – www.XXX.

To further simplify the maintenance and care of our products, Invacare would like to inform you also about a new Service Manual for these rollators. This document contains all information needed for the service of these rollators. The Service Manual contains information about service life, warranty, maintenance and technical data. The Service Manual states explicitly that control and maintenance of the product must be carried out regularly.

The Service Manual for the above rollators can also be downloaded from Invacare website – www.XXX.

Please ensure that the information in this notice is made available to all affected technical aid centers, service companies and dealers within your organization, and that the advice stated in these Manuals is adhered to.

If you have any questions relating to this bulletin, or should you require any additional information, please contact:

Technical Services Department

Author :	A.Romann	Released :	T. Pradignac	Version :	02
Date :		Date:			



EF015 European Form	Field Safety Notice	Page 2 of 2
-------------------------------	----------------------------	-------------

Tel: XXX

Fax: XXX

Invacare confirms that the relevant Competent Authorities have been informed of this Field Safety Corrective Action.

Author :	A.Romann	Released :	T. Pradignac	Version :	02
Date :		Date:			