
Urgent Field Safety Notice

Implant Pick-Up EV 5.4 Short

Ref: 25522, Lot: 186641, Manufacturing Date: 2016-04-25

Date: 2016-Jun-03

Dear dental professional or dental lab,

The following is to confirm in a written format the information you have received previously by phone or email.

Details on affected devices:

Implant Pick-Up EV 5.4 Short

Ref: 25522

Lot: 186641

Manufacturing Date: 2016-04-25

www.dentsplysirona.com

Description of the problem:

Dental implant systems are used to replace missing teeth in patient's mouth. One of the accessories that are used in the treatment of the patients are the "Impression Posts" or "Implant pick-ups".

This product is intended to be used by the dentist as a transfer key from the situation in the patient's mouth to the impression sent to the dental technician to manufacture a model of the patient's dental situation. The product is re-usable and will be kept in the office sorted by color, which reflects the different sizes used for the different implants sizes.

The package for the Implant pick-up 5.4 - Brown contains a brown pick-up but with the diameter of 4.8. 4.8 is the diameter for EV Blue but both 5.4 and 4.8 had the same size screw.

If the incorrect size is used it might give a faulty impression since the pick-up will not reflect the correct implant situation (slightly different height and position) of the patients mouth. There is no clinical risk as this product is only used for taking impressions. The possible risk is a prosthetic solution that might require rework.

Status of product impacted

486 In total have been manufactured, 227 products are with customers

Advise on action to be taken by the user:

1. The Implant Pick-up 5.4 short, Ref: 25522, Lot: 186641, shall not be used. Please, return the product(s) to DentsplySirona. Correct product(s) will be sent as replacement.
2. Further information will be provided by our Sales rep in cases where products from the faulty Lot have been used and how to handle this. We would ask you to make an appointment with any patient who was treated using this product and verify that the prosthetic work is of good quality.
3. If you have unpacked the Implant Pick-up 5.4 short, Ref: 25522, Lot: 186641, and stored it together with other Implant Pick-ups of the size 5.4. Please, return all Implant Pick-ups of the size 5.4 for replacement.
4. In case you need any further assistance, please refer to the contact person indicated below.

Transmission of this Field Safety Notice:

This notice needs to be passed on to all those who need to be aware within your organization or to any organization where the potentially affected devices have been transferred.

Please transfer this notice to other organizations on which this action has an impact.

Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action.

Contact

Torsten Meyer-Elmenhorst

Medical Device Safety Officer (MPA EU)

Sicherheitsbeauftragter für Medizinprodukte (MPG EU)

DENTSPLY Implants Manufacturing GmbH

Steinzeugstrasse 50

D-68229 Mannheim

Tel. +49 (0) 621 4302-1370

Fax. +49 (0) 621 4302-2370

E-mail: implants-safetyofficer@dentsply.com

