«Hospital_Name»

«Users_Name»- «Department»

«Customer_Address»

«Zip_Code» «City» - «Country_name»

Reference: 92036067-FA

2 November 2016

Field Safety Notice - Urgent Medical Device Recall LOTUS™ Edge Valve System

Dear «Users_Name»,

Boston Scientific is implementing a voluntary Field Safety Corrective Action (FSCA) for the recently launched LOTUS™ Edge Valve System to remove all units from the field. The specific list of units is included in this letter.

There have been seven reports where the valve could not be fully locked and released. In six cases, the valve was retrieved and removed, resulting in procedure prolongation. In one case the valve was positioned in the descending aorta. In all seven cases, the patient was successfully treated with a new valve. Boston Scientific has not received any reports of other serious adverse events associated with this failure mode; however potential complications could include stroke and/or vessel trauma.

There is no safety issue for patients who previously received a LOTUS Edge valve since the issue only occurs during the implant procedure and does not impact the performance of the implanted valve.

This FSCA affects all UPNs and batches of the LOTUS Edge Valve System.

Note: The current generation LOTUS Valve System continues to be available and is not affected by this removal.

Our records indicate that your facility received some of the concerned product. The **table below provides a complete list of all affected products**, including Product Description, Material Number (UPN) and Lot numbers. Please note that **only the devices listed below are affected. No other Boston Scientific product is involved in this Field Safety Notice.**

Further distribution or use of any remaining product affected by this action should cease immediately.

Product Description	UPN	Lot Number	Expiration Date Range
LOTUS™ Edge Valve System	H749LVS230	See attached listing for lot numbers	13 February 2017 through 14 July 2017
	H749LVS250		
	H749LVS270		
	H749LVSCL230		
	H749LVSCL250		
H749LVSCL270			

INSTRUCTIONS:

- 1- **Please immediately discontinue use of the Boston Scientific product reported in the list and remove all of the affected units from your inventory**, regardless of where these units are stored in your facility. Segregate the units in a secure place, pending return to Boston Scientific.
- 2- **Please complete the attached Verification Form even if you do not have any product to return.**
- 3- **When completed, please return the Verification Form to your local Boston Scientific office** for the attention of Fax: 80 30 80 05 - Email: NordicProdQuality@bsci.com on or before **17 November 2016**.
- 4- **If you have products to return**, please package them in an appropriate shipping box and **contact Victoria Bolin - Tel: +46 (0)70-25 96 732 of your local Boston Scientific office**, to arrange return.
- 5- Please pass this notice to any healthcare professional from your organization that needs to be aware and to any organization where the potentially affected devices have been transferred (If appropriate). Please provide Boston Scientific with details of any affected devices that have been transferred to other organizations (if appropriate).

Your Competent Authority is being notified of this Field Safety Notice.

We regret any inconvenience that this action may cause, and we appreciate your understanding as we act to ensure patient safety and customer satisfaction.

If you have any questions or would like assistance with this Field Safety Notice, please contact your local Sales Representative.

Yours sincerely,

Marie Pierre Barlanga
Quality Department
Boston Scientific International S.A.

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