FIELD SAFETY NOTICE - UPDATE

Cardiac Surgery Mycobacterium Risks
Disinfection and Cleaning of Sorin Heater Cooler Devices

FSCA identifier:	9611109-11/11/16-008-C
Affected Devices:	Sorin Group perfusion system – Heater-cooler 1T, Heater-cooler 3T and FlexTherm devices (catalog number: 16-02-50, 16-02-80, 16-02-81, 16-02-82, 16-02-83, 16-02-85, 16-02-95, 16-70-00)
Date:	December 01, 2016
Attention:	Hygiene Specialists, Cardiac Surgery Operating Room Responsible, Risk/ Safety Managers, Distributors, Clinicians, Perfusionists and other users of these devices
Reason:	LivaNova has issued an updated Field Safety Notice based on recent information released by the United States.

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¹ LivaNova PLC is a U.K. holding company with a number of wholly-owned subsidiaries, including Sorin Group Deutschland GmbH and Sorin Group USA, Inc. In this document, we refer to all entities using the brand name LivaNova.

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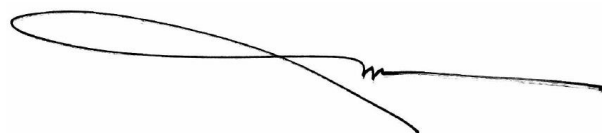
Vi takker for dit samarbejde i denne situation.

Med venlig hilsen

Thierry Dupoux
Vice President Quality Assurance
LivaNova

Attachment 1

MMWR

Attachment 2

EuroSurveillance

Attachment 3

LivaNova 2015 FSN

Attachment 4

Customer Response Form

FIELD SAFETY NOTICE UPDATE

Cardiac Surgery Mycobacterium Risks
 Disinfection and Cleaning of Sorin Heater Cooler Devices
 Reference # 9611109-11/11/16-008-C

Please complete and return this completed form within two working days

1. We HAVE reviewed and understand the attached Field Safety Notice **yes** **no**
2. We DO NOT understand the attached Field Safety Notice and request more information **yes** **no**

Please list the serial numbers for Sorin Heater Cooler device(s) in use at your facility:

Product Code	Product description	Affected Serial Number

Customer Name:	<<Print Your Company name here>>	
Country:	<<Print Your Country here>>	
Contact Name:	<<Print Your Contact Name here>>	
E-mail:	<<Print Your E-mail address here>>	
Fax Number:	<<Print Your Fax No. here>>	
Phone Number:	<<Print Your Phone No. here>>	
SUBMITTED BY	SIGNATURE	DATE
...../...../.....

To return the form or if you have any questions regarding this Field Safety Notice or deep-disinfection service, please contact us at:

E-mail:	sorin.fsn@stericycle.com
Fax Number:	+44 – 2080806540 (international)
Phone Number:	8082-6009

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