



DiaMed GmbH  
Pra Rond 23  
1785 Cressier FR / Switzerland  
Phone: +41 (0)26 674 51 11  
Fax: +41 (0)26 674 54 45

Cressier, 03/05/2017

## Field Safety Notice 001-17\_Follow-up letter II

Dear Customer,

Through the investigations performed, we were able to identify the main root cause and contributory factors leading to the unexpected results described in the FSN 001-17.

The root cause is a non-homogeneous silicone coating process into the glass vials. As a permanent correction, another source of vials will be used for the production of the ID-cells listed as "affected devices displaying the issue".

These vials allow recovering the full quality of the product, starting from lots numbers listed according to Appendix I and for any subsequent lots numbers.

For already delivered products, FSN recommendations still apply.

Concerning the other devices from the same product range (ID-DiaCell ABO and ID-DiaCell papainized cells), we have performed a close monitoring and will keep doing. So far, no event has been reported, confirming these products are most probably out of scope.

In case of any questions, in the first instance, please contact our Customer Service Laboratory:

**[slabor\\_cressier@bio-rad.com](mailto:slabor_cressier@bio-rad.com)**

Our representatives are briefed to help you in managing this situation.

We apologize for any inconvenience that may have been caused by this action and we appreciate your prompt cooperation in this matter.

Yours sincerely,

Site Quality Management Representative,  
Immunohematology Division

Diane Galea

Vice President and General Manager,  
Immunohematology Division

Ann Madden



DiaMed GmbH  
Pra Rond 23  
1785 Cressier FR / Switzerland  
Phone: +41 (0)26 674 51 11  
Fax: +41 (0)26 674 54 45

**ANNEX 1**

**Urgent: Field Safety Notice / 001-17-Follow-up Letter II**  
**Reply Form for Customers**

**PRODUCT:**

Product Name	Catalog No
ID-Dia (Diego) Positive	004134 / 004134VJ
ID-DiaCell SF	003640
ID-DiaCell Pool	003630 / 003631
ID-DiaCell ABO/I-II	003610
ID-DiaCell ABO/I-II-III	003618
ID-DiaScreen I-II-III-IV-VP-VIP	004316
ID-DiaCell I-II	003613 003613VJ
ID-DiaPanel	004114 004114VJ
ID-DiaCell I-II-III	004310 / 004310VJ
ID-DiaScreen I-II-III-IV	004311
ID-DiaCell I-II-III Asia	003614
ID-DiaScreen Prophylax	004330
ID-DiaPanel Plus 6	004414

**CUSTOMER INFORMATION:**

<b>Hospital / Laboratory</b>	
<b>Address</b> (Street, Postcode)	
<b>Country</b>	
<b>Phone Number</b>	
<b>Undersigning manager name</b>	
<b>Customer Account Number</b>	

**STATEMENT:**

I am aware of information about the field action concerning the above reference product(s) and have proceeded according to the instructions issued by Bio-Rad.

Date / Customer Stamp / Signature: \_\_\_\_\_

Please return this form to your customer service: **[enter local details]**



## Appendix I- B

Please find hereunder our Red Blood Cell Product delivery schedule: if you need information about your group of countries please contact your local customer service.

### Group B

The group of countries B will receive their first delivery with these vials starting from lots numbers listed in the table 1 and according to yearly delivery schedule.

Shipping Date: (Please enter the delivery week for your customer)

**Table 1**

Product Name	Catalog No	Lot No	Expiry Date
ID-Dia (Diego) Positive	004134 / 004134VJ	05980.79.2	03/07/2017
ID-DiaCell SF	003640	06060.56.1	17/07/2017
ID-DiaCell Pool	003630 / 003631	06070.74.2	03/07/2017
ID-DiaCell ABO/I-II	003610	45002.84.1	03/07/2017
ID-DiaCell ABO/I-II-III	003618	45012.84.1	03/07/2017
ID-DiaScreen I-II-III-IV-VP-VIP	004316	45070.53.2	03/07/2017
ID-DiaCell I-II	003613 003613VJ	45151.81.1 4515.81.02	03/07/2017 03/07/2017
ID-DiaPanel	004114 004114VJ	45161.10.1 4516.10.02	03/07/2017 03/07/2017
ID-DiaCell I-II-III	004310 / 004310VJ	45183.70.2	03/07/2017
ID-DiaScreen I-II-III-IV	004311	45200.53.2	03/07/2017
ID-DiaCell I-II-III Asia	003614	45330.25.2	03/07/2017
ID-DiaScreen Prophylax	004330	45660.80.1	03/07/2017
ID-DiaPanel Plus 6	004414	45670.91.1	31/07/2017



## Appendix I- A

Please find hereunder our Red Blood Cell Product delivery schedule: if you need information about your group of countries please contact your local customer service.

### Group A

The group of countries A will receive their first delivery with these vials starting from lots numbers listed in the table 2 and according to yearly delivery schedule.

Shipping Date: (Please enter the delivery week for your customer)

**Table 2**

Product Name	Catalog No	Lot No	Expiry Date
ID-Dia (Diego) Positive	004134 / 004134VJ	05980.80.1	17/07/2017
ID-DiaCell SF	003640	06060.56.1	17/07/2017
ID-DiaCell Pool	003630 / 003631	06070.75.1	17/07/2017
ID-DiaCell ABO/I-II	003610	45002.85.1	17/07/2017
ID-DiaCell ABO/I-II-III	003618	45012.85.1	17/07/2017
ID-DiaScreen I-II-III-IV-VP-VIP	004316	45070.54.1	17/07/2017
ID-DiaCell I-II	003613 003613VJ	45151.82.1 N/A	17/07/2017 N/A
ID-DiaPanel	004114 004114VJ	45161.11.1 N/A	17/07/2017 N/A
ID-DiaCell I-II-III	004310 / 004310VJ	45184.27.1	17/07/2017
ID-DiaScreen I-II-III-IV	004311	45200.54.1	17/07/2017
ID-DiaCell I-II-III Asia	003614	45330.26.1	17/07/2017
ID-DiaScreen Prophylax	004330	45660.80.2	17/07/2017
ID-DiaPanel Plus 6	004414	45670.91.2	31/07/2017