bioMérieux NORDICS
Customer Service Department
Direct no.: +46 31 68 58 58
Fax no.: +46 31 68 48 68
e-mail:
customersupport.nordic@biomerieux.com
Our Ref.: FSCA – 3351 - eMAG® system :
anomalies correction

København, 10 05 2017

VIGTIG PRODUKTINFORMATION (Field Safety Notice)
FSCA – 3351 - eMAG® system : anomalies correction

Kære Kunde,

Vores arkiver indikerer at Deres laboratorium anvender vores eMAG® system til oprensning af nukleinsyrer (ref. 418591).

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
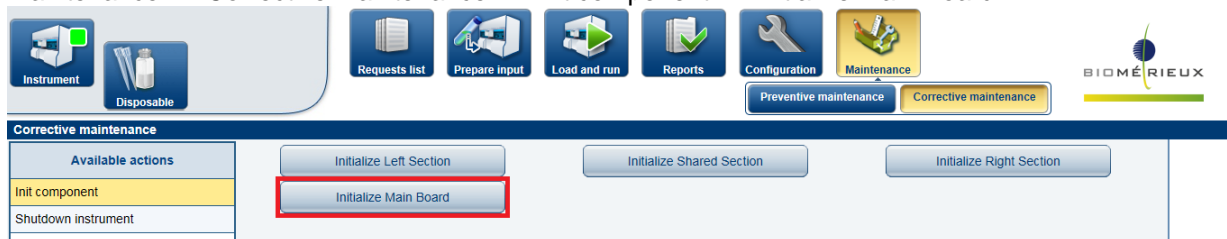
Maintenance >> Corrective Maintenance >> Init component >> Initialize Main Board.

The screenshot shows the software interface for 'Corrective maintenance'. The 'Maintenance' menu is expanded, and 'Corrective maintenance' is selected. Under 'Available actions', 'Initialize Main Board' is highlighted with a red box. Other actions include 'Init component' and 'Shutdown instrument'.

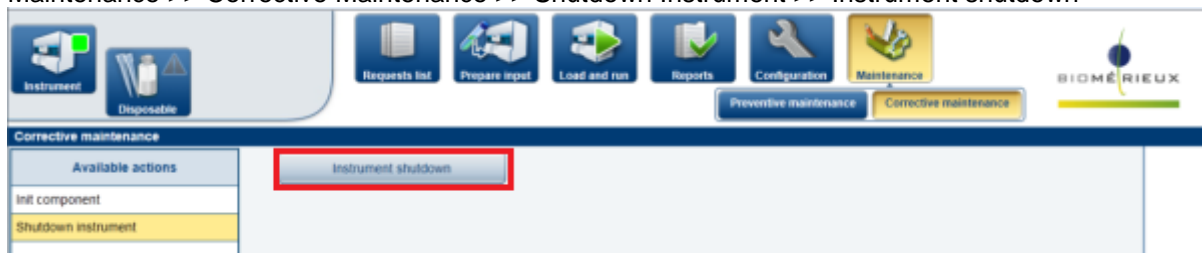Maintenance >> Corrective Maintenance >> Shutdown Instrument >> Instrument shutdown

The screenshot shows the software interface for 'Corrective maintenance'. The 'Shutdown instrument' menu item is highlighted with a red box. Other actions include 'Init component' and 'Shutdown instrument'.

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Med Venlig Hilsen,

A handwritten signature in black ink, appearing to read "Anne Lykke Pedersen".

Anne Lykke Pedersen
Field Application Specialist
bioMérieux Denmark

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NORWAY
bioMérieux Norge AS
Tlf. 2152 0023
Vat.nr/Org.nr 981 062 310

Attachment A: Acknowledgement Form.

URGENT PRODUCT CORRECTION NOTICE

FSCA 3351 – eMAG® system : anomalies correction
TO BE RETURNED TO YOUR BIO-MERIEUX CUSTOMER SERVICE AT THE FOLLOWING

CUSTOMERSUPPORT.NORDIC@BIOMERIEUX.COM

Name of the laboratory:

City:

Customer number:

- I acknowledge receipt of the bioMérieux letter regarding the “eMAG® system: anomalies correction”
- I will implement the required actions as indicated in the Urgent Product Correction Notice.
- Have you received reports of illness or injury related to the eMAG® system: anomalies correction?

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bioMérieux Sweden AB
bioMérieux Suomi Oy