



## Re: OCS™ HEART PERFUSION MODULE INSTALLATION

### Dear Customer:

TransMedics® is committed to optimizing our customers' experience with the OCS™. This Product Information Bulletin describes the recommended sequence for installing the perfusion module into the OCS Console.

The Instructions For Use (IFU) describes the proper method for installing the Perfusion Module into the OCS Console. In order to ensure proper performance, such as maintaining the configured blood temperature, the perfusion module should be installed into the Console while the system is in Standby Mode. When the user transitions the system from Standby Mode to Run Mode, the perfusion module will be included in the system Self Test.

**CAUTION—** To ensure proper system operation, include the HPM in the system's Self-Test by installing the HPM in the OCS™ Heart Console while the system is in Standby Mode.

### Contact Us

TransMedics® is committed to continually improving the products we provide. Should you have any questions on this Product Information Bulletin or recommendations for future enhancements please contact your local support representative or Customer Support at +1 978 552 0999.



TransMedics, Inc.

200 MINUTEMAN ROAD, SUITE 302, ANDOVER, MA 01810  
Main +1.978.552.0900 | Service +1.978.552.0999 | Fax +1.978.552.0978  
www.transmedics.com

TransMedics®, OCS™ and the TransMedics logo are trademarks of TransMedics, Inc., Andover, MA, USA. All rights reserved.  
© 2015 by TransMedics, Inc. All rights reserved.

p. 1 of 1