

**URGENT FIELD SAFETY NOTICE**  
**Polysorb™ Braided Absorbable Suture**  
**Endo Stitch™ Loading Unit with Polysorb™ Suture**  
**Maxon™ Monofilament Absorbable Suture**  
**Biosyn™ Monofilament Absorbable Suture**

May, 2017

Medtronic reference: FA770

**Attention: Risk Management Director and O.R. Materials Management**

Dear Valued Customer:

The purpose of this letter is to advise you that Medtronic is recalling specific item codes and production lots of Polysorb™ braided absorbable suture, Endo Stitch™ loading unit with Polysorb™ suture, Maxon™ monofilament absorbable suture, and Biosyn™ monofilament absorbable suture.

**Issue Description:**

This Field Safety Corrective Action (FSCA) is being conducted due to the potential for an incomplete seal on the inner package, which may result in premature degradation of the suture material. The use of products with this condition may increase the potential for the suture to break. This packaging defect does not impact the sterility of the product. There have been no reports of serious injury associated with these issues.

Medtronic requests that you quarantine and return any unused products of the items/lots detailed below. Unused products from the affected item codes and lots should be returned as described in the Required Actions section below. If you have distributed the Polysorb™ braided absorbable suture, Endo Stitch™ loading unit with Polysorb™ suture, Maxon™ monofilament absorbable suture, Biosyn™ monofilament absorbable suture listed on Attachment A, please promptly forward the information from this letter to those recipients. All unused products from the affected item codes and lots must be returned.

This FSCA affects only the item codes and lots listed on Attachment A.

This action is being taken with the knowledge of the Competent Authority in your country. Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action. We request that you contact Medtronic if you experienced quality problems or adverse events.

- Email Medtronic Regulatory Affairs at: [NordicRA@Covidien.com](mailto:NordicRA@Covidien.com)

Page 1 of 7

**Actions:**

1. Please quarantine and discontinue use of the affected item codes and lots listed on Attachment A.
2. Please return affected product as follows:

	Customer with inventory	Customer with zero inventory	Where to send the completed form
Purchased <b>directly</b> from Medtronic	Please complete the attached Returns Verification Form in its entirety. Upon receiving your form, Medtronic Customer Care will contact you to organize the return of your products. You will receive credit for unused device(s) that you return.	Complete form and check the box indicating "no inventory"	E-mail or fax the completed form to the Medtronic contact provided on the verification form.
Purchased from a <b>distributor</b>	Complete <b>all</b> fields on the form and contact your distributor directly to arrange for return of product	Complete form and check the box indicating "no inventory"	E-mail or fax the completed form to your Distributor & to the Medtronic contact provided on the verification form.

We regret any inconvenience this may cause. We are committed to patient safety and appreciate your prompt attention to this matter. If you have any questions regarding this communication, please contact your Medtronic representative.

Sincerely,



J. Bryan Dannettell  
 Vice President, Quality Assurance  
 Surgical Innovations  
 Medtronic



Sebnem Yavas Hoffsten  
 Regulatory Affairs Manager Nordic Countries  
 Medtronic

**Attachment A**

Item Number	Product Description	Lots	Item Number	Product Description	Lots
2CL852	POLYSORB* 2 VIOLET 2X36" BGS-29 (2X90CM)	D4L0467X	CL13MG	POLYSORB 2/0 18" UNDYED GS-21 DT	D5A1034X
3CL812	POLYSORB* 0 30" VIOLET GS-21 REG ATTCH	D5A1024X	CL13MG	POLYSORB 2/0 18" UNDYED GS-21 DT	D5A1035X
CL10MG	POLYSORB* 0 5X18" VIOLET GS-21 DT	D5A0169X	CL13MG	POLYSORB 2/0 18" UNDYED GS-21 DT	D5B0193X
CL10MG	POLYSORB* 0 5X18" VIOLET GS-21 DT	D5A0303X	CL14MG	POLYSORB 0 5X18" UNDYED GS-21 DT	D4L1756X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A0304X	CL14MG	POLYSORB 0 5X18" UNDYED GS-21 DT	D4L1758X
CL10MG	POLYSORB* 0 5X18" VIOLET GS-21 DT	D5A0305X	CL14MG	POLYSORB 0 5X18" UNDYED GS-21 DT	D5A1745X
CL10MG	POLYSORB* 0 5X18" VIOLET GS-21 DT	D5A0406X	CL2M	POLYSORB* 0 5 X 30" UNDYED GS-21 DT	D4K1415X
CL10MG	POLYSORB* 0 5X18" VIOLET GS-21 DT	D5A0478X	CL2M	POLYSORB* 0 5 X 30" UNDYED GS-21 DT	D5A1026X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A0524X	CL2M	POLYSORB* 0 5 X 30" UNDYED GS-21 DT	D5A1028X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A0525X	CL52MG	POLYSORB 2/0 5X18" UNDYED GS-10 DT	D4K2361X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A0526X	CL52MG	POLYSORB 2/0 5X18" UNDYED GS-10 DT	D4M0071X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A0889X	CL52MG	POLYSORB 2/0 5X18" UNDYED GS-10 DT	D5A1862X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A0890X	CL60M	POLYSORB* 1 VIOLET 5X30" GS-26	D4L1131X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A0961X	CL622M	POLYSORB* 2 VIOLET 5X24" GS-24 D-TACH* (	D4L0692X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1031X	CL64M	POLYSORB 1 30" VIOLET GS-25 DT	D4J1987X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1032X	CL659M	POLYSORB* 2 VIOLET 30" GS-26	D4M0542X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1033X	CL665M	POLYSORB* 2 30" VIOLET GS-25 DT	D4K1044X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1392X	CL6M	POLYSORB 1 30" VIOLET GS-21 DT	D4L1431X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1393X	CL71M	POLYSORB* 0 VIOLET 5X30" HGS-21 D-TACH	D4M0368X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1394X	CL885M	POLYSORB 1 5X30" VIOLET GS-22 DT	D4J0841X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1395X	CL885M	POLYSORB 1 5X30" VIOLET GS-22 DT	D4K0619X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1396X	CL885M	POLYSORB 1 5X30" VIOLET GS-22 DT	D4L0147X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1743X	CL885M	POLYSORB 1 30" VIOLET GS-22 DT	D4L0427X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5A1861X	CL885M	POLYSORB 1 30" VIOLET GS-22 DT	D4M0370X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5B0014X	CL886M	POLYSORB* 0 5X30" VIOLET GS-22 DT	D4K1616X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5B0261X	CL97MG	POLYSORB* 1 5 X 30" VIOLET HOS-12 DT	D4L1159X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5B0264X	CL997MG	POLYSORB* 0 VIOLET 5X30" HGS-20 D-TACH	D4L1563X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5B0423X	CL997MG	POLYSORB* 0 VIOLET 5X30" HGS-20 D-TACH	D4M1020X
CL10MG	POLYSORB* 0 18" VIOLET GS-21 DT	D5B0424X	CLT633M	POLYSORB 2 VL 5 X 30" BGST-29 DT	D4K2021X
CL11MG	POLYSORB 1 5X18" VIOLET GS-21 DT	D4J0442X	GL182M	POLYSORB 3-0 30" VIOLET CV-25 DT	D4K2438X
CL11MG	POLYSORB 1 5X18" VIOLET GS-21 DT	D4K2262X	GL223M	POLYSORB 2-0 30" VIOLET V-30 DT	D4L1371X
CL11MG	POLYSORB 1 5X18" VIOLET GS-21 DT	D4M0073X	GL33MG	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A0307X
CL11MG	POLYSORB 1 5X18" VIOLET GS-21 DT	D5B0260X	GL33MG	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A0529X
CL11MG	POLYSORB 1 5X18" VIOLET GS-21 DT	D5B0263X	GL33MG	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A0963X
CL13MG	POLYSORB 2/0 5X18" UNDYED GS-21 DT	D4L1968X	GL33MG	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A1401X
CL13MG	POLYSORB 2/0 5X18" UNDYED GS-21 DT	D5A0070X	GL33MG	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A1422X
CL13MG	POLYSORB 2/0 5X18" UNDYED GS-21 DT	D5A0071X	GL33MG	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5B0149X
CL13MG	POLYSORB 2/0 5X18" UNDYED GS-21 DT	D5A0072X	GL33MG	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5B0266X

## Attachment A Continued

Item Number	Product Description	Lots	Item Number	Product Description	Lots
GL33MG	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5B0844X	GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5B0267GX
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A0410X	GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5B0538GX
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A1037X	GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5B0539GX
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A1397X	GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5B0540GX
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A1398X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4L0058X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A1399X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4L0108X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A1400X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4L0637X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A1746X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4L0638X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A1747X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4L0639X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5A1870X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4L2006X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5B0148X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4L2010X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5B0150X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4L2201X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5B0243X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4M0267X
GL34MG	POLYSORB 4/0 5X18" VIOLET CV-25 DTACH	D5B0536X	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D4M0362X
GL62MG	POLYSORB 2/0 5X18" VIOLET V-20 DTACH	D4L1180GX	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A0170X
GL62MG	POLYSORB 2/0 5X18" VIOLET V-20 DTACH	D5A0308GX	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A0172X
GL62MG	POLYSORB 2/0 5X18" VIOLET V-20 DTACH	D5A0528GX	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A0479X
GL62MG	POLYSORB 2/0 5X18" VIOLET V-20 DTACH	D5A1042GX	GLJ50M	POLYSORB 3/0 5X18" VIOLET CV-25 DTACH	D5A1041X
GL62MG	POLYSORB 2/0 5X18" VIOLET V-20 DTACH	D5B0427GX	L14	POLYSORB 0 18 VIO PRECUT 226T	D4L1165X
GL63MG	POLYSORB 3/0 5X18" VIOLET V-20 DTACH	D4L1177GX	L14	POLYSORB 0 18 VIO PRECUT 226T	D5A0247X
GL63MG	POLYSORB 3/0 5X18" VIOLET V-20 DTACH	D4L1179GX	L14	POLYSORB 0 18 VIO PRECUT 226T	D5A0248X
GL63MG	POLYSORB 3/0 5X18" VIOLET V-20 DTACH	D4L1962GX	L14	POLYSORB 0 6X18" VIOLET PRECUT	D5A0546X
GL63MG	POLYSORB 3/0 5X18" VIOLET V-20 DTACH	D4L2077GX	L15	POLYSORB 1 6X18" VIOLET PRECUT	D4L1161X
GL63MG	POLYSORB 3/0 5X18" VIOLET V-20 DTACH	D4L2202GX	L15	POLYSORB 1 18 VIO PRECUT 122J	D5A0096X
GL63MG	POLYSORB 3/0 5X18" VIOLET V-20 DTACH	D5A0306GX	L24	POLYSORB 0 18 UNDYED PRECUT	D4L1164X
GL63MG	POLYSORB 3/0 5X18" VIOLET V-20 DTACH	D5A0480GX	L25	POLYSORB 1 18" UNDYED PRECUT	D5A0551X
GL63MG	POLYSORB 3/0 5X18" VIOLET V-20 DTACH	D5A1039GX	L26	POLYSORB* 2 UNDYED 12X18" PRECUT	D4J0736X
GL66M	POLYSORB 2/0 5X30" VIOLET V-20 DT	D4K1263GX	L26	POLYSORB* 2 UNDYED 12X18" PRECUT	D4K0212X
GL67M	POLYSORB 3/0 30" VIOLET V-20 DT	D5A0513GX	L26	POLYSORB* 2 UNDYED 12X18" PRECUT	D5A0402X
GL67M	POLYSORB 3/0 30" VIOLET V-20 DT	D5A0515GX	L27	POLYSORB 0 12X18" VIOLET PRECUT	D5A0249X
GL68M	POLYSORB 4/0 5X30" VIOLET V-20 DT	D4L0759GX	L27	POLYSORB 0 18" VIOLET PRECUT	D5A0755X
GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5A0171GX	L27	POLYSORB 0 18" VIOLET PRECUT	D5A1219X
GL69MG	POLYSORB 3/0 5X18" UNDYED V-20 DTACH	D5A0409GX	L27	POLYSORB 0 18" VIOLET PRECUT	D5A1220X
GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5A0962GX	L27	POLYSORB 0 18" VIOLET PRECUT	D5A1315X
GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5A1748GX	L28	POLYSORB* 2 VIOLET 12X18" PRECUT	D4K2022X
GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5A1805GX	L28	POLYSORB* 2 VIOLET 12X18" PRECUT	D4L1546X
GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5A1869GX	L34	POLYSORB* 0 12X24" VIOLET PRECUT	D4K2023X
GL69MG	POLYSORB 3/0 18" UNDYED V-20 DT	D5B0242GX	L34	POLYSORB* 0 12X24" VIOLET PRECUT	D4L1547X

## Attachment A Continued

Item Number	Product Description	Lots	Item Number	Product Description	Lots
L35	POLYSORB* 1 12X18" VIOLET PRECUT	D4K1272X	LL222	POLYSORB* 3-0 VIOLET 98" REEL	D4K2315X
L35	POLYSORB* 1 18" VIOLET PRE-CUT	D4L1740X	LL222	POLYSORB* 3-0 VIOLET 98" REEL	D5A1231X
L35	POLYSORB* 1 18" VIOLET PRE-CUT	D4M1189X	LL222	POLYSORB* 3-0 VIOLET 98" REEL	D5A1803X
L35	POLYSORB* 1 18" VIOLET PRE-CUT	D4M1224X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4K1392X
L35	POLYSORB* 1 18" VIOLET PRE-CUT	D5A0756X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4K1612X
L4	POLYSORB 0 18 U/D PRECUT 126R	D4K0211X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4K2139X
L4	POLYSORB 0 18 U/D PRECUT 126R	D4L1163X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4L0279X
L4	POLYSORB 0 18 U/D PRECUT 126R	D5A0876X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4L0822X
L40	POLYSORB* 0 6X30" VIOLET PRECUT	D4M1190X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4L1005X
L40	POLYSORB* 0 30" VIOLET PRECUT	D5A0228X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4L1399X
L40	POLYSORB* 0 6X30" VIOLET PRECUT	D5A0549X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4L2111X
L404	POLYSORB 0 3X30" VIOLET PRECUT	D4L1545X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D4M1186X
L404	POLYSORB 0 30 VIO PRECUT 227C	D5A0757X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A0102X
L405	POLYSORB 1 30 VIO PRECUT 224E	D5A0875X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A0105X
L41	POLYSORB* 1 30" VIOLET PRECUT	D4K2024X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A0106X
L41	POLYSORB* 1 30" VIOLET PRECUT	D4L1162X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A0107X
L41	POLYSORB* 1 30" VIOLET PRECUT	D5A0548X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A0112X
L42	POLYSORB* 2 VIOLET 6X30" PRE-CUT	D5A0098X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A0113X
LL102	POLYSORB 3/0 60 U/D REEL 104R	D5A1291X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A1233X
LL104	POLYSORB 0 60 U/D REEL 203G	D4K1537X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A1290X
LL104	POLYSORB 0 60 U/D REEL 203G	D5A1294X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5A1339X
LL111	POLYSORB 4/0 60" REEL VIO 217T	D4L1509X	LL223	POLYSORB* 2-0 VIOLET 98" REEL	D5B0302X
LL112	POLYSORB* 3-0 VIOLET 60 REEL	D4L1170X	LL224	POLYSORB* 0 VIOLET 98" REEL	D4L0689X
LL112	POLYSORB* 3-0 VIOLET 60 REEL	D4L2106X	LL224	POLYSORB* 0 VIOLET 98" REEL	D4L1166X
LL112	POLYSORB* 3-0 VIOLET 60 REEL	D5A0110X	LL224	POLYSORB* 0 VIOLET 98" REEL	D4L1677X
LL112	POLYSORB* 3-0 VIOLET 60 REEL	D5A0771X	LL224	POLYSORB* 0 VIOLET 98" REEL	D4L1867X
LL112	POLYSORB* 3-0 VIOLET 60 REEL	D5A0811X	LL224	POLYSORB* 0 VIOLET 98" REEL	D4L2107X
LL113	POLYSORB* 2-0 VIOLET 60 REEL	D4L0065X	LL224	POLYSORB* 0 VIOLET 98" REEL	D4M1187X
LL113	POLYSORB* 2-0 VIOLET 60 REEL	D4L0515X	LL224	POLYSORB* 0 VIOLET 98" REEL	D5A0723X
LL113	POLYSORB* 2-0 VIOLET 60 REEL	D4L1429X	LL224	POLYSORB* 0 VIOLET 98" REEL	D5A0724X
LL113	POLYSORB* 2-0 VIOLET 60 REEL	D4L2118X	LL224	POLYSORB* 0 VIOLET 98" REEL	D5A1298X
LL113	POLYSORB* 2-0 VIOLET 60 REEL	D4M0410X	LL225	POLYSORB* 1 VIOLET 98" REEL	D4J0947X
LL113	POLYSORB* 2-0 VIOLET 60 REEL	D5A0104X	LL225	POLYSORB* 1 VIOLET 98" REEL	D4L0704X
LL113	POLYSORB* 2-0 VIOLET 60 REEL	D5A0109X	LL225	POLYSORB* 1 VIOLET 98" REEL	D4L2029X
LL114	POLYSORB* 0 VIOLET 60 REEL	D4L1866X	LL225	POLYSORB* 1 VIOLET 98" REEL	D4L2109X
LL114	POLYSORB* 0 VIOLET 60 REEL	D5A0869X	LL225	POLYSORB* 1 VIOLET 98" REEL	D5A0823X
LL221	POLYSORB* 4/0 98" VIOLET REEL 110T	D4L1169X	LL225	POLYSORB* 1 VIOLET 98" REEL	D5A0951X
LL221	POLYSORB* 4/0 98" VIOLET REEL 110T	D4M0237X	LL226	POLYSORB* 2 VIOLET 98" REEL	D4L0640X
LL221	POLYSORB* 4/0 98" VIOLET REEL 110T	D5A1288X	LL226	POLYSORB* 2 VIOLET 98" REEL	D4L0641X
LL221	POLYSORB* 4/0 98" VIOLET REEL 110T	D5A1297X	LL226	POLYSORB* 2 VIOLET 98" REEL	D4L0642X

**Attachment A  
Continued**

Item Number	Product Description	Lots
170052	ENDO STITCH* POLYSORB* 0 48" VIO DLU SU	J4L1477X
170053	ENDO STITCH*POLYSORB* 2/0 48" VIO DLU SU	J5A0702X
170053	ENDO STITCH*POLYSORB* 2/0 48" VIO DLU SU	J5A0703X
170053	ENDO STITCH*POLYSORB* 2/0 48" VIO DLU SU	J5A0704X
170053	ENDO STITCH*POLYSORB* 2/0 48" VIO DLU SU	J5A0705X
170053	ENDO STITCH*POLYSORB* 2/0 48" VIO DLU SU	J5A0921X
170053	ENDO STITCH*POLYSORB* 2/0 48" VIO DLU SU	J5A1206X
170053	ENDO STITCH*POLYSORB* 2/0 48" VIO DLU SU	J5A1207X
170056	ENDOSTITCH POLYSORB 0 48" UNDYED DLU SU	J4M0853X
170056	ENDOSTITCH POLYSORB 0 48" UNDYED DLU SU	J4M0855X
170056	ENDOSTITCH POLYSORB 0 48" UNDYED DLU SU	J5A0192X
170057	ENDOSTITCH POLYSORB 2/0 48" UNDYED DLUSU	J5A1097X

Item Number	Product Description	Lots
8886603433	MAXON* 4/0 5X30 GREEN V-20 D*TACH	D4L1156GX
8886621933	MAXON 4/0 5X18" GREEN CV-24 D-TACH	D4K0396X
8886621933	MAXON 4/0 5X18" GREEN CV-24 D-TACH	D4K2053X
8886621933	MAXON 4/0 5X18" GREEN CV-24 D-TACH	D4L0541X
8886621933	MAXON 4/0 5X18" GREEN CV-24 D-TACH	D4L1084X
8886621933	MAXON 4/0 5X18" GREEN CV-24 D-TACH	D4L1272X
8886621933	MAXON 4/0 5X18" GREEN CV-24 D-TACH	D4M0394X
XX2109	MAXON 5/0 5X18 GR CV-23 D-TACH	D4K1485X
GMJ34M	BIOSYN* 4/0 18" VIOLET CV-25 DT	D4K0010X

Item Number	Product Description	Lots
US1540	CUSTOM SUTURE PACK	A5E0919X
US1614	CUSTOM SUTURE PACK	A5C1345X
US1672	USPACK-SUTURE KIT	A5D0789X
14111316-03	SCHRITTMACHER ANGIO X3	228111X

Attachment B

**Item #** GL-33-MG

**COVIDIEN™**  
**Polysorb™**  
Braided Absorbable Suture

**CV-25**  
TAPER  
1/2 ⌀ 22 mm

**3-0** 2 Metric  
5X18" 45 cm VIOLET

LOT XXXXXXXXXXXXX Use by YYYY-MM-DD

MP 5 x 5 D-Tach

**Lot #**

---

**Item #** XX-2109

**COVIDIEN™**  
**Maxon™**  
Monofilament Absorbable Suture

**CV-23**  
TAPER  
1/2 ⌀ 17 mm

**5-0** 1 Metric  
5X18" 45 cm GREEN

LOT XXXXXXXXXXXXX Use by YYYY-MM-DD

MP 5 x 5 D-Tach

**Lot #**

---

**Item #** 170053

**COVIDIEN™**  
**Polysorb™**  
Absorbable Single Stitch Reload

**ES-9**  
TAPER  
9 mm

**2-0** 3 Metric  
48" 120 cm

LOT XXXXXXXXXXXXX Use by XXXX-XX Mfg XXXX-XX

12  
VIOLET / VIOLETTE / VIOLETT /  
VIOLA / MORADO / VIOLETA

**Lot #**