

Drägerwerk AG & Co. KGaA, 23542 Lübeck

To our customers of
Infinity Acute Care System Workstation Critical Care – Evita Infinity V500
Infinity Acute Care System Workstation Neonatal Care – Babylog VN500
Evita V300

Availability of SW 2.51

July 2017

Dear Madam/Sir,

We recently informed you about the market introduction of SW 2.50 for the above mentioned devices. SW 2.50 supports a newly developed power supply module (M7.3+) and introduced a new battery concept together with the possibility of determining the operating time of the installed batteries.

We would like to inform you about further improvements which will be rolled out to all devices containing SW 2.50 and an M7.3+.

The following changes will be introduced with the installation of SW 2.51:

- Change of wording for SW and Instruction for Use (US-English only)
- Configurable battery check
 - Battery check and associated functions can be activated/deactivated
 - For further information please refer to the respective Instruction for use.
- Correction of requesting a battery check although valid battery check data are available
- Introduction of new firmware for M7.3+
 - Improvement of displayed charge status after storage of devices with partially discharged batteries

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23558 Lübeck, Germany
Postal address:
23542 Lübeck, Germany
Tel +49 451 882-0
Fax +49 451 882-2080
info@draeger.com
www.draeger.com
VAT no. DE135082211

Bank details:
Commerzbank AG, Lübeck
IBAN: DE95 2304 0022 0014 6795 00
Swift-Code: COBA DE FF 230
Sparkasse zu Lübeck
IBAN: DE15 2305 0101 0001 0711 17
Swift-Code: NOLADE21SPL

Registered office: Lübeck
Commercial register:
Local court Lübeck HRB 7903 HL
General partner: Drägerwerk Verwaltungs AG
Registered office: Lübeck
Commercial register:
Local court Lübeck HRB 7395 HL

Chairman of the Supervisory Board
for Drägerwerk AG & Co. KGaA
and Drägerwerk Verwaltungs AG:
Prof. Dr. Nikolaus Schweickart
Executive Board:
Stefan Dräger (chairman)
Rainer Klug
Gert-Hartwig Lescow
Dr. Reiner Piske
Anton Schrofner

We apologize for any inconvenience the implementation of this improvement may cause for you and your organisation and thank you for your support.

Please contact your local DrägerService representative to obtain more information about this software release and to schedule installation.

Best regards,



Frank Ralfs
Head of Product Management
Respiratory Care



Andreas Neumann
Senior Product Manager
Respiratory Care