

FIELD SAFETY NOTICE

Product:

- VP-7000 video processor

Type of Action: Version Upgrade of VP-7000 to Ver. 1.011

Dear Customers,

FUJIFILM is issuing this Field Safety Notice for our VP-7000 video processor.

This Field Safety Notice is intended to inform you about the following:

- what the problem is
- the actions that should be taken by the customer/user

This document contains important information for the continued safe and proper use of your equipment.

Please review the following information with all members of your staff who need to be aware of the contents of this communication. It is important to understand the implications of this communication.

Description of the problem

It has been reported that when a 700 series scope, with a substance such as mucosa and blood remaining on the LG lens, was connected to the VP-7000/BL-7000 system, smoke-like water vapour was emitted or coagulation/fixation of the substance was found on the lens. This phenomenon was caused because the substance remaining on the LG lens of the scope was heated by absorbing the light from the light source almost at the maximum level.

If this occurs, it is necessary to remove the substance remaining on the distal end of the scope or replace the scope. This leads to cancellation of the examination or prolonged examination time.

In order to prevent this phenomenon, the software is upgraded so that the light volume at the scope distal end is reduced.

Actions to be taken by customer/user

(1) According to the procedure of the operation manual, visually and manually check the following points.

- The light guides are free from scratches, cracks, stains or disengagement.

(2) According to the procedure of the reprocessing manual, please brush the distal end of the endoscope with the endoscope completely immersed in a detergent solution and remove debris on the external surface of the distal end of endoscope using a clean toothbrush.

ACTIONS PLANNED BY FUJIFILM

FUJIFILM service personnel will contact all of the medical facilities where the applicable products have been installed to arrange for this correction and visit to take corrective measures.



Field Safety Notice Reference: FSN 20170905 VP-7000 video processor

FUJIFILM is committed to providing products and services of the highest quality. Your satisfaction with FUJIFILM products and with our response to this issue is very important to us. If you have any questions about this matter, please contact your local FUJIFILM office.

Yours sincerely,

FUJIFILM

FIELD SAFETY NOTICE

Customer Feedback Form

Please complete this feedback form and return it by fax or email.
Thank you for your cooperation.

Customer/Facility Name:

Address:

Instrument Serial Number:

I confirm that I have received and understand the attached notice.

This notice does not apply to my facility.

The device has been transferred to another organisation.

Customer Name:

Position:

Signature:

Date:

Phone number:

If we have the wrong contact information, please correct it below:

Customer/Facility Name:

Address:

Please FAX or email this completed form to: