Brignais, le 04/07/2019

---

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*The ENDALIS company initiates a field safety notice following:*

- *The safety alert issued by the Canadian Competent Authority, Health Canada, April 17<sup>th</sup>, 2019, on the potential risks, associated with the use of intragastric balloons in weight loss therapy, of*
  - o *Gastric and esophageal perforation,*
  - o *Acute pancreatitis,*
  - o *Spontaneous balloon overinflation.*

- *Following a case of acute pancreatitis reported to the company ENDALIS in May 2019. (Vigilant incident issued by Endalis to the Competent Authorities)*

*The complications of acute pancreatitis and spontaneous balloon overinflation were not mentioned in the end-ball instructions, ENDT110.*

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The complications of acute pancreatitis and spontaneous balloon overinflation will be added in chapter 9 of the instructions for use (end-ball, ENDT110).

<b>Before : Chapter « Complications »</b>	<b>After : Chapter 9 : « Complications »</b>
<p>The following complications were mentioned:</p> <ul style="list-style-type: none"> <li>- <b>Complications linked to the insertion or extraction of the balloon</b></li> <li>- <b>Mechanical complications due to the balloon:</b> <ul style="list-style-type: none"> <li>• Ulceration of the gastric wall which could result in perforation has been described (heaviness of the balloon filled with 100% liquid) necessitating an urgent surgical operation with risk to life.</li> <li>• Digestive tract obstructions are possible where there is migration, which is seen if the balloon is not sufficiently filled or is partially deflated. It can be a gastric obstruction if the balloon becomes impacted in the gastric antrum or an intestinal obstruction which sometimes requires a surgical operation.</li> </ul> </li> </ul> <p>The risk of these complications is greatly increased if the maximum duration of six months for the implantation of the intra-gastric balloon is not respected.</p> <ul style="list-style-type: none"> <li>- <b>Functional complications (side effects)</b></li> <li>- <b>Metabolic complications</b></li> <li>- <b>Failures</b></li> </ul>	<p>Addition of 2 new complications :</p> <ul style="list-style-type: none"> <li>- <b>Complications linked to the insertion or extraction of the balloon</b></li> <li>- <b>Complications mécaniques dues au ballon</b> <ul style="list-style-type: none"> <li>• Ulcerations of the gastric wall can lead to perforation due to the heaviness of the balloon filled with fluid, necessitating an urgent surgical operation with risk to life.</li> <li>• Digestive tract obstructions are possible where there is migration, which is seen if the balloon is not sufficiently filled or is partially deflated. It can be a gastric obstruction if the balloon becomes impacted in the gastric antrum or an intestinal obstruction which sometimes requires a surgical operation.</li> </ul> </li> </ul> <p>The risk of these complications is greatly increased if the maximum duration of six months for the implantation of the intra-gastric balloon is not respected.</p> <ul style="list-style-type: none"> <li>• A risk of spontaneous balloon overinflation due to gas production in the balloon has been described in the literature.</li> <li>• Acute pancreatitis is possible due to mechanical pressure exerted by the balloon on the pancreas through the gastric wall.</li> </ul> <ul style="list-style-type: none"> <li>- <b>Functional complications (side effects)</b></li> <li>- <b>Metabolic complications</b></li> <li>- <b>Failures</b></li> </ul>

#### 4- **Actions destinées aux clients (utilisateurs) / Advise on action to be taken by the user:**

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*This notice needs to be passed on all those who need to be aware within your organisation or to any organisation where the potentially affected devices have been transferred. (If appropriate)*

*Please transfer this notice to other organisations on which this action has an impact. (If appropriate)*

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*If you have any questions, please contact us:*

Pays/Country	Adresse/Address	Contact	Téléphone /Phone	E-mail
France	ENDALIS 1 Allée des Tilleuls 69530 Brignais	Pierre-André DENIS Marie-Ange BOILLETOT	(33) 435575700	<a href="mailto:padenis@endalis.com">padenis@endalis.com</a> <a href="mailto:qualite@endalis.com">qualite@endalis.com</a>

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*The ENDALIS company confirms that the French health Competent Authority (ANSM) was informed of the FSN*

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*The undersigned confirms that he/she has received the FSN of 4 July 2019 and he/she will act accordingly.*

Pays / Country:

Nom / Name:

Date et Signature / Date and signature :