Vores ref.: MNE/19-191

Bobigny, den 8. januar 2020

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Information om aktuel IFU (IN121 V06-2018):	Information ændret i IFU:
<p style="text-align: center;">INDICATIONS OF USE</p> <p>PETERS SURGICAL instruments are solely designed for laparoscopic surgery. Depending on each instrument and its model, they may be employed for:</p> <ul style="list-style-type: none"> <li>- mechanical freeze incision,</li> <li>- monopolar incision via high-frequency (HF) current,</li> <li>- monopolar coagulation via high-frequency (HF) current,</li> <li>- mechanical dissection,</li> <li>- atraumatic or traumatic prehension of tissues</li> </ul> <p>Maximum assigned voltage: <b>5,000 at peak voltage</b> (the generator should be set at a lower voltage).</p> <p>Such instruments should only be handled by qualified practitioners having gained due experience thereof.</p> <p>Make sure that the voltage assigned to any accessory used in combination with a HF generator is superior to that of the generator itself (power setting, mode); please refer to IEC 60601-2-2 requirement. Please also refer to the Instructions' Manual of the HF generator.</p>	<p style="text-align: center;">Information ændret i IFU:</p> <p>PETERS SURGICAL instruments are solely designed for laparoscopic surgery. Depending on each instrument and its model, they may be employed for:</p> <ul style="list-style-type: none"> <li>- mechanical freeze incision,</li> <li>- monopolar incision via high-frequency (HF) current,</li> <li>- monopolar coagulation via high-frequency (HF) current,</li> <li>- mechanical dissection,</li> <li>- atraumatic or traumatic prehension of tissues</li> </ul> <p>Maximum assigned voltage: <b>4,000 at peak voltage</b> (the generator should be set at a lower voltage).</p> <p>Such instruments should only be handled by qualified practitioners having gained due experience thereof.</p> <p>Ensure compatibility with all other associated instruments; <u>in particular, the connections and voltage supplied by the HF generator should be lower than the assigned voltages of the accessories being used in combination</u> (see IEC 60601-2-2). Check the instruction for use of the cable and of the HF generator regarding its operating, as well as its power and mode settings.</p> <p>NB: Only surgical monopolar cables may be connected to PETERS SURGICAL monopolar instruments</p>

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