



Dexcom, Inc. | Corporate Headquarters
6340 Sequence Drive
San Diego, CA 92121
888.738.3646
dexcom.com

**Field Safety Notice | Dexcom G6 Sensor
FAS-SD-20-003
Advise from Manufacturer**

Date: **(Add date)** December 2020

Attention: Valued Customer, Head of Healthcare Facility and/or Medical Device Liaison Officer

Details on affected devices:

This field safety notice applies to Dexcom G6 Sensor Models STS-GS-002 and STS-GS-003.

Description of the problem:

A new adhesive patch was implemented for the G6 sensor to improve patch performance and reliability in October 2019. All G6 sensors have the new patch material. We have seen significant benefit of this change for most patients; however, we are aware of a smaller number of patients challenged with varying degrees of skin irritation resulting in an increase in the complaint rate for skin irritation.

The risk associated is acute allergic or irritant contact dermatitis causing skin irritation that may result in symptoms such as itching, burning, and/or rashes at the site of adhesive patch application. These rashes are infrequent but at times may be severe and the irritation can include redness, swelling, and blistering. The symptoms and rashes vary greatly and Dexcom has received some reports of patients requiring medical intervention associated with the skin irritation. The risk of skin irritation leading to hospitalization is unlikely.

The risk of skin irritation is inherent in any product with an adhesive component and there are some patients for whom the product will not be suitable. As manufacturers our aim is to produce a device that can work for as many patients as possible and to provide appropriate support and assistance to those patients for whom the device is not suitable.

As we continue looking at ways to make our devices a usable option for more of the patient population, we are aware that 3rd party barrier creams or patches have helped some patients who would not otherwise be able to use the G6. Please visit the FAQ section of our website at www.Dexcom.com for more information.

We have not tested or validated these possible solutions, which will be specific to each individual patient and for that reason, the use of barrier creams or patches needs to be determined by those best placed to assess your individual needs. It may be important to discuss your individual situation and needs with your healthcare professional, as well as the short-term and long-term health effects of skin irritation.

Advise on action to be taken by the user:

- Refer to the Using Your G6 Guide for information on skin irritation around the sensor site and/or discuss your individual situation and needs with your healthcare professional.

Dexcom

Dexcom, Inc. | Corporate Headquarters
6340 Sequence Drive
San Diego, CA 92121
888.738.3646
dexcom.com

Contact:

For product troubleshooting or other **Technical Support enquires:** **(Add local number)**

The undersign confirms that this notice has been provided to the appropriate Regulatory Agency.

Sincerely,

Dexcom Quality Compliance

MASTER TEMPLATE