

To our customers of  
Dräger Breathing System Filters  
SafeStar55 (MP01790)

Mai 2022

## Important Safety Notice

### Potentially obstructed breathing system filter

#### Only the following products are affected:

Breathing System Filter SafeStar 55 (MP01790; GTIN: 04048675420828)  
with lot number LT2103 (other lots are not affected)

Dear Sir or Madam,

Within our global vigilance activities, we have become aware of one case in which an obstructed breathing system filter SafeStar 55 was used on a patient during anesthesia. The patient reportedly became hypoxic and had to be reanimated.

To detect obstructions, every filter is subject to a 100% resistance test. If the test fails, the respective filter will automatically be destroyed. Our analyses revealed the obstruction of the SafeStar 55 was due to a human error during a service measure of the manufacturing equipment. During the service measure the automatic destruction had to be deactivated and the filters were sorted manually. During that limited time the filter in question was added to the "passed" box although having failed the resistance test. Our analyses of the supply chain and deliveries indicates that there are most likely no more potentially affected SafeStar 55 in the market – however, it can't be excluded with absolute certainty.

The SafeStar 55 Instructions for Use contain a corresponding warning and advise the user to check all system components for obstructions, damage, and foreign matter before installation.

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53-55  
23558 Lübeck, Germany  
Postal address:  
23542 Lübeck, Germany  
Tel +49 451 882-0  
Fax +49 451 882-2080  
info@draeger.com  
www.draeger.com  
VAT no. DE135082211

Bank details:  
Commerzbank AG, Lübeck  
IBAN: DE95 2304 0022 0014 6795 00  
Swift-Code: COBA DE FF 230  
Sparkasse zu Lübeck  
IBAN: DE15 2305 0101 0001 0711 17  
Swift-Code: NOLADE21SPL

Registered office: Lübeck  
Commercial register:  
Local court Lübeck HRB 7903 HL  
General partner: Drägerwerk Verwaltungs  
AG  
Registered office: Lübeck  
Commercial register:  
Local court Lübeck HRB 7395 HL

Chairman of the Supervisory  
Board for Drägerwerk AG & Co.  
KGaA and Drägerwerk  
Verwaltungs AG:  
Stefan Lauer  
Executive Board:  
Stefan Dräger (chairman)  
Rainer Klug  
Gert-Hartwig Lescow  
Dr. Reiner Piske

If however, an obstructed filter is used on a patient, ventilation will be restricted and the patient might desaturate. SafeStar 55 is tested and released for the use with devices for both anesthetic and respiratory use. These devices feature both pressure and volume monitoring. In case of an obstruction, corresponding alarms will be generated indicating restricted ventilation of the patient.

Dräger has already taken measures to ensure that all filters are subject to both resistance test AND automatic sorting and automatic destruction of poor parts respectively.

Actions to be taken:

**Please check your stock for filters with lot number "LT2103" and return them to Dräger.**

Dräger will replace all returned filters free of charge.


Please ensure that all users of the above-mentioned products and other persons within your organization are made aware of this Important Safety Notice. If you have provided the products to third parties, please forward a copy of this information.

Please keep this information available to all users at least until the you have checked your stock.

The responsible authorities have been notified of this action.

**Identification of the affected medical devices:**

According to our records you have received potentially affected breathing system filters SafeStar 55 manufactured by Drägerwerk AG & Co. KGaA with lot number LT2103. To identify affected products in your stock, please check the Lot number on the label on the packaging:

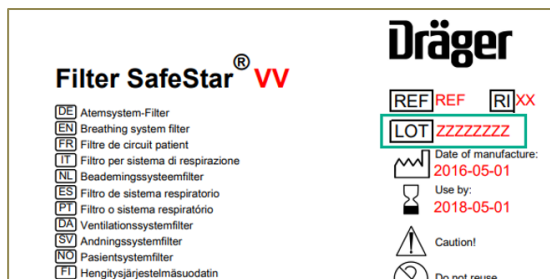

## Contact:

If you have any questions, please do not hesitate to contact your local Dräger representative.

We apologize for any inconvenience caused by this measure.

We thank you for your support.

Kind regards

Theresa Strampfer

Product Manager  
Product Management  
Business Unit Hospital Consumables & Accessories  
Medical Division