Date: 2023-09-29

Field Safety Notice
Hegenberger Retractor – A1390
Missing Instructions for Use

Dear Customer,

We have been made aware that some boxes of the Hegenberger Retractor (A1390) have been shipped without Instructions for Use (IFU).

Our investigation suggests that this issue is limited to Lot Number 4227 however we are continuing to investigate in order to determine if any other lots have been affected.

The purpose of this Field Safety Notice (FSN) is to provide the IFU to you so that you can provide them to the affected users.

Please refer to the following pages, which provide further information and actions for you to take. Please follow the “Action to be taken by Customer/User” section of this notice as soon as possible.

This issue has been reported to the appropriate regulatory agencies.

I sincerely regret the inconvenience that this may cause you. Your satisfaction with Cetro Medical’s products and with our response to this issue is very important to us.

Please contact your local Hegenberger Retractor representative if you have any questions or concerns about this Field Safety Notice and/or the actions we are asking you to follow.

Sincerely,

Lena Nurbo
Head of Quality Assurance & Regulatory Affairs

Field Safety Notice
Hegenberger Retractor – A1390
Missing Instructions for Use

1. Information on Affected Devices	
1.	1. Device Type(s) Postpartum vaginal retractor
1.	2. Commercial name(s)* Hegenberger Retractor
1.	3. Unique Device Identifier(s) (Basic UDI-DI) 73401943139SCN
1.	4. Primary clinical purpose of device(s) The product is sterile, single use and intended to allow healthcare professionals access to postpartum inner lesions by mechanical separation of vaginal walls.
1.	5. Device Model/Catalogue/part number(s) A1390
1.	6. Software version N/A
1.	7. Affected serial or lot number range Lot No. 4227
1.	8. Associated devices N/A

2. Reason for Field Safety Corrective Action (FSCA)	
2.	1. Description of the product problem We have received reports that at least some of the products in lot number 4227 were shipped without instructions for use (IFU).
2.	2. Hazard giving rise to the FSCA We assess the risk as low because the Hegenberger Retractor is easy to use and customer training is provided to new users, however the lack of printed IFU can potentially lead to improper use of the device, which can cause minor injury to patients.
2.	3. Probability of problem arising IFUs are provided on the basis of one printed IFU per 10 Hegenberger Retractors. Therefore, it is not expected that a copy of the IFU must be consulted each time the device is used. As such, we assess the probability of minor injury to patients as low, where the IFU is not available to the clinician when using the device.
2.	4. Predicted risk to patient/users The risk of harm to patients/users is low, as outlined in 2.3 above.
2.	5. Further information to help characterise the problem N/A – IFU was not shipped with product.
2.	6. Background on Issue We received a report from a Distributor in Ireland who indicated they could not find the IFU with 25 boxes of product in their inventory. When investigating the production of lot 4227, a discrepancy was noted between the number of products produced and the

4. General Information		
4.	1. FSN Type	New
4.	2. For updated FSN, reference number and date of previous FSN	N/A
4.	3. For Updated FSN, key new information as follows:	
	N/A	
4.	4. Further advice or information already expected in follow-up FSN?	No
4.	5. If follow-up FSN expected, what is the further advice expected to relate to:	
	No follow-up FSN expected.	
4.	6. Anticipated timescale for follow-up FSN	N/A
4.	7. Manufacturer information (For contact details of local representative refer to page 1 of this FSN)	
	a. Company Name	Cetro Medical AB
	b. Address	Nitgatan 11 , SE- 333 33 Smålandsstenar, Sweden
	c. Website address	https://cetromedical.se/
4.	8. The Competent (Regulatory) Authority of your country has been informed about this communication to customers. Yes	
4.	9. List of attachments/appendices:	Hegenberger Retractor Instructions for Use
4.	10. Name/Signature	Lena Nurbo Head of Quality Assurance & Regulatory Affairs
		

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1. Open the pouch and remove the Hegenerber Retractor. Hold the Hegenerber Retractor as shown in the image.

2. Use your index finger as shown in the picture and press the sides of Hegenerber Retractor together before inserting. **Do not use gel or lubrication on Hegenerber Retractor.**

3. The patient must be in a supine position with the legs elevated.

4. Keep the sides of Hegenerber Retractor compressed while inserting it in the vagina. Follow the direction of the pelvic canal.

5. Place Hegenerber Retractor all the way in so that the circles are resting on labia majora.

6. Correct placement of Hegenerber Retractor is shown in this image.

Removing Hegenerber Retractor

7. Gently press the sides of Hegenerber Retractor together before removing.

It is important that you use your index finger to ensure the Hegenerber Retractor is clear of the cervix.

Press down on the top part and gently retract Hegenerber Retractor from the vagina.

Contraindications

Hegenberger Retractor is only intended for use on women following vaginal delivery.

Hegenberger Retractor is only intended for use by trained clinical professionals after understanding the instructions for use.

The patient must be in a supine position with elevated legs.

Not to be used for pediatric patients or for placement in patients who have not delivered or in cases where force would be necessary for insertion.

Not to be used in cases involving post-partum haemorrhage.

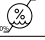
Not to be used without informed consent from the patient.

Not to be used during pregnancy.

Not to be used for anterior wall procedures.

Not for reuse.

Explanation of labeling on labels and instructions for use.

 See Instructions for use	 Manufacturing day	 Don't reuse
REF Article number	 Expiration date	 Don't sterilize for reuse
LOT Lot number	 Don't use if package is damaged	 Temp range 0 to 30 ° C
MD Medical Device	 Humidity 0-70%	STERILE Sterilized with ethylene oxide
 Keep dry	 Protect from sunlight	 Latex free
 In accordance with the Medical Device Directive 93/42 / EEC (0402).		 Manufacturer

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