

07. november 2025

VIGTIGT: MEDDELELSE OM FELTSIKKERHED FOR MEDICINSK Udstyr

Kære, værdsatte kunde:

Kerr udsender hermed en frivillig meddelelse om feltsikkerhed for at identificere en uoverensstemmelse i brugsanvisningen til håndfile hvad angår rengøringsstid. Se **Tabel 1** for del- og partinumre for berørte håndfile.

Det berørte produkt kan været blevet sendt til din facilitet mellem 13. februar 2025 og 22 juli 2025.

Vi er for nylig blevet opmærksomme på, at der er en fejl i brugsanvisningen i afsnittet om rengøring, trin 1, angående ultralydscyklustid. Der er følgende fejl i brugsanvisningen:

“Det anbefales at rengøre håndstopperne i et ultralydsapparat i fem (5) minutter med et enzymatisk ultralydsrengøringsmiddel med neutral pH-værdi (Enzol® Enzymatic Detergent eller tilsvarende).”

Der bør stå følgende i brugsanvisningen:

“Det anbefales at rengøre håndfilerne i et ultralydsapparat i ti (10) minutter med et enzymatisk ultralydsrengøringsmiddel med neutral pH-værdi (Enzol® Enzymatic Detergent eller tilsvarende).”

Der forventes ingen umiddelbare eller langvarige sundhedsmæssige konsekvenser, da disse filer skal inspiceres og steriliseres før brug.

Kerr beder venligst om din hjælp med at udfylde og sende den vedlagte bekræftelsesformular tilbage via e-mail for at bekræfte modtagelsen af denne meddelelse om feltsikkerhed, uanset om produktet findes i din beholdning eller ej.

Hvis du er en autoriseret Kerr-distributør, anmoder vi om, at du identificerer de kunder, der kan have fået tilsendt det berørte produkt, og kontakter dem for at informere dem om problemet.

Vi beklager den ulejlighed, som denne situation forårsager dig og dine kunder.

Eventuelle spørgsmål kan rettes til Kerrs kundeservice på **CustomerService-EMEA@kerrdental.com**.

Mange tak for din tålmodighed og støtte.

Med venlig hilsen,

Holdet hos Kerr



Via Passanti
174 84018 Scafati (SA) Italy
Phone: 00800 30 32 30 32
KerrDental.com

[English Version]

URGENT: MEDICAL DEVICE FIELD SAFETY NOTICE

Dear Valued Customer:

Kerr is initiating a voluntary field notice due to identifying a discrepancy on the Instructions for Use (IFU) for Hand Files regarding the cleaning time. See **Table 1** for affected Hand Files part numbers and Lot Numbers.

The affected product may have been shipped to your facility from 13 February 2025 to 22 July 2025.

It has recently come to our attention that the IFU has an error in the cleaning section step #1 for the ultrasonic cycle time. The IFU incorrectly stated:

“Clean the hand pluggers in an ultrasonic device for five (5) minutes using a neutral-pH enzymatic ultrasonic detergent (Enzol® Enzymatic Detergent, or equivalent) is recommended.”

The IFU should state:

“Clean the hand files in an ultrasonic device for ten (10) minutes using a neutral-pH enzymatic ultrasonic detergent (Enzol® Enzymatic Detergent, or equivalent) is recommended.”

There are no immediate or long-range health consequences expected as these files must be inspected and sterilized prior to use.

Kerr kindly requests your cooperation in completing and emailing back this enclosed acknowledgement form in order to confirm your receipt of this field safety notice letter, regardless of whether you have any product in your inventory.

If you are an authorized Kerr distributor, we request that you identify those customers that may have been shipped the affected product and contact these customers to inform them of the issue.

We sincerely apologize for the inconvenience this situation causes you and your customers.

If you have any questions, contact Kerr Customer Care at **CustomerService-EMEA@kerrdental.com**.

Thank you for your patience and support.

Sincerely Yours,

The Team at Kerr

Acknowledgement Form for Hand File Bekræftelsesformular for håndfil

Please refer to the enclosed Part Number & Lot Number Table, **Table 1**, to determine if you have any of the affected product in your inventory.

Se ovenstående tabel over del- og partinumre, **Tabel 1**, for at afgøre, om det berørte produkt findes i din beholdning.

Please return this acknowledgement form, whether you have the affected product or not to:

Uanset om du har det berørte produkt eller ej, bedes du returnere bekræftelsesformularen til:

HANDFILE.RESPONSE@ENVISTACO.COM

Please check **all** that applies.

Markér **alle** relevante.

☐	*We acknowledge receipt of the Hand File Field Safety Notice. *Vi bekræfter modtagelse af meddelelsen om feltsikkerhed for håndfil.	
☐	I have checked my stock and quarantined inventory. Jeg har kontrolleret min beholdning og mit karantæneanbragte lager.	Please attach a list of affected stock so that customer care can process your return or replacement. Vedhæft en liste over berørt beholdning, så kundeservice kan behandle din returnering eller ombytning.
☐	I have identified customers that received or may have received this device. Jeg har identificeret kunder, som har eller kan have modtaget udstyret.	
☐	I have informed the identified customers of this FSN. Jeg har informeret de identificerede kunder om denne meddelelse om feltsikkerhed.	Please enter date of communication: Angiv kommunikationsdato:
☐	I have received confirmation of reply from all identified customers. Jeg har modtaget bekræftelse fra alle identificerede kunder.	
☐	I have returned affected devices - enter number of devices returned and date complete. Jeg har returneret berørt udstyr – angiv antal returnerede enheder samt dato.	Please attach a list that states the quantity, Part Number, Lot/Serial Number, and Date Returned so that customer care can process your return or replacement. Vedhæfte en liste med antal, delnummer, parti-/serienummer og returdato, så kundeservice kan behandle din returnering eller ombytning
☐	Neither I nor any of my customers has any affected devices in inventory. Hverken jeg eller mine kunder har berørt udstyr i beholdning.	



Via Passanti
174 84018 Scafati (SA) Italy
Phone: 00800 30 32 30 32
KerrDental.com

Contact Person (Print Name)* Kontaktperson (blokbogstaver)*	
Signature* Underskrift*	
Date* Dato*	
Contact Number or E-mail* Kontaktnummer eller -e-mail*	

Mandatory fields are marked with *
Obligatoriske felter er markeret med *

Table 1: Affected Hand Files Part Numbers and Lot Numbers

Tabel 1: Del- og partinumre for berørte håndfile

Part Number	Product Description	Lot Number
06151E	FILES 30MM 10 PURPLE	022512967
06153E	FILES 30MM 20 YELLOW	022512968
06155E	FILES 30MM 25 RED	022512969
06005E	FILES 21MM 20 YELLOW	022587528
06014E	FILES 21MM 25 RED	022587529
06062E	FILES 25MM 10 PURPLE	022587530
06062E	FILES 25MM 10 PURPLE	022587531
06063E	FILES 25MM 15 WHITE	022587532
06064E	FILES 25MM 20 YELLOW	022587533
06065E	FILES 25MM 25 RED	022587534
06066E	FILES 25MM 30 BLUE	022587535
06067E	FILES 25MM 35 GREEN	022587536
06069E	FILES 25MM 40 BLACK	022587537
06161E	FILES 30MM 50 YELLOW	022587538
06162E	FILES 30MM 55 RED	022587539
06163E	FILES 30MM 60 BLUE	022587540
06166E	FILES 30MM 70 GREEN	022587541
06170E	FILES 30MM 80 BLACK	022587542
06014E	FILES 21MM 25 RED	032512344
06064E	FILES 25MM 20 YELLOW	032512345
06155E	FILES 30MM 25 RED	032512346
06163E	FILES 30MM 60 BLUE	032512347
06170E	FILES 30MM 80 BLACK	032512348
14127E	FILES 21MM 08 GREY	032512350
06062E	FILES 25MM 10 PURPLE	032517023
06063E	FILES 25MM 15 WHITE	032517024
06063E	FILES 25MM 15 WHITE	032538878
06063E	FILES 25MM 15 WHITE	032538879
14128E	FILES 25MM 08 GREY	032538883
14128E	FILES 25MM 08 GREY	032538884
06004E	FILES 21MM 15 WHITE	032538886
62250E	FILES 06 25 MM	032538887
06063E	FILES 25MM 15 WHITE	032538888
14128E	FILES 25MM 08 GREY	032538892
06004E	FILES 21MM 15 WHITE	032566849

Part Number	Product Description	Lot Number
61363E	K-FILES 21MM SZ 45-70 ASST. PK/6	022539996
62249E	K-FILE, 06 21MM PK-6	032512352
821-8021E	K-FILES 21MM #15-40 ASST	032538881
821-8021E	K-FILES 21MM #15-40 ASST	032538890
821-8121E	K-FILES 21MM #45-80 ASST	032566858
821-8021E	K-FILES 21MM #15-40 ASST	032598197
821-8125E	K-FILES 25MM #45-80 ASST	042524357
821-8025E	K-FILES 25MM #15-40 ASST	042529156
821-8021E	K-FILES 21MM #15-40 ASST	042539906
821-8025E	K-FILES 25MM #15-40 ASST	042559654
821-8030E	K-FILES 30MM #15-40 ASST	042566431
821-8030E	K-FILES 30MM #15-40 ASST	042578354
821-8130E	K-FILES 30MM #45-80 ASST	042598743
821-8025E	K-FILES 25MM #15-40 ASST	102443201
821-8025E	K-FILES 25MM #15-40 ASST	102463963
62251E	K-FILE, 06 30MM PK-6	112468881
821-8025E	K-FILES 25MM #15-40 ASST	112487949
62249E	K-FILE, 06 21MM PK-6	122467696
821-8021E	K-FILES 21MM #15-40 ASST	122467702
821-8030E	K-FILES 30MM #15-40 ASST	122467703
821-8125E	K-FILES 25MM #45-80 ASST	122467704
821-8130E	K-FILES 30MM #45-80 ASST	122467705
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	022512977
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	022512978
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	022512979
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	022512980
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	022512981
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	022512982
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	022512983
15146E	K-FLEX FILES 25MM SZ 35 GREEN PK6	022512985
15150E	K-FLEX FILES 25MM SZ 40 BLACK PK6	022512986
15222E	K-FLEX FILES 30MM SZ 25 RED PK6	022538982
15294E	K-FLEX FILES 21MM SZ 10 PURPLE PK6	022538985
15298E	K-FLEX FILES 21MM SZ 15 WHITE PK6	022538986
15306E	K-FLEX 25 21MM	022538987

Part Number	Product Description	Lot Number
62250E	FILES 06 25 MM	032566851
14127E	FILES 21MM 08 GREY	032566852
06005E	FILES 21MM 20 YELLOW	032566853
06003E	FILES 21MM 10 PURPLE	032566855
06066E	FILES 25MM 30 BLUE	032566856
06062E	FILES 25MM 10 PURPLE	032598193
06003E	FILES 21MM 10 PURPLE	032598198
06004E	FILES 21MM 15 WHITE	032598199
06014E	FILES 21MM 25 RED	032598200
06063E	FILES 25MM 15 WHITE	032598202
06064E	FILES 25MM 20 YELLOW	032598203
06151E	FILES 30MM 10 PURPLE	032598204
06163E	FILES 30MM 60 BLUE	032598206
06170E	FILES 30MM 80 BLACK	032598208
14127E	FILES 21MM 08 GREY	032598218
06062E	FILES 25MM 10 PURPLE	042516600
06062E	FILES 25MM 10 PURPLE	042516601
06069E	FILES 25MM 40 BLACK	042516602
06003E	FILES 21MM 10 PURPLE	042516603
06004E	FILES 21MM 15 WHITE	042524350
06152E	FILES 30MM 15 WHITE	042524364
06065E	FILES 25MM 25 RED	042524371
62250E	FILES 06 25 MM	042524377
06015E	FILES 21MM 30 BLUE	042524384
06003E	FILES 21MM 10 PURPLE	042529151
06004E	FILES 21MM 15 WHITE	042529153
14128E	FILES 25MM 08 GREY	042529155
06064E	FILES 25MM 20 YELLOW	042529160
14127E	FILES 21MM 08 GREY	042539905
06062E	FILES 25MM 10 PURPLE	042545587
06063E	FILES 25MM 15 WHITE	042545588
06003E	FILES 21MM 10 PURPLE	042545589
06004E	FILES 21MM 15 WHITE	042559651

Part Number	Product Description	Lot Number
15154E	K-FLEX 45 25MM	022538993
15158E	K-FLEX FILES 25MM SZ 50 YELLOW PK6	022538994
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587511
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587512
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587513
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587514
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587515
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587516
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587517
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587518
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587519
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587520
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	022587521
15134E	K-FLEX FILES 25MM SZ 20 YELLOW PK6	022587522
15134E	K-FLEX FILES 25MM SZ 20 YELLOW PK6	022587524
15134E	K-FLEX FILES 25MM SZ 20 YELLOW PK6	022587525
15138E	K-FLEX 25 25MM	022587526
15138E	K-FLEX 25 25MM	022587527
15142E	K-FLEX FILES 25MM SZ 30 BLUE PK6	022587543
15210E	K-FLEX FILES 30MM SZ 10 PURPLE PK6	032512351
821-4021E	K-FLEX FILES 21MM SERIES 1 SST ASST 15/4	032538880
821-4025E	K-FLEX FILES 25MM SERIES 1 SST ASST 15/4	032538882
821-4125E	K-FLEX FILES 25MM SERIES 2 SST ASST 45/8	032538885
821-4021E	K-FLEX FILES 21MM SERIES 1 SST ASST 15/4	032538889
821-4025E	K-FLEX FILES 25MM SERIES 1 SST ASST 15/4	032538891
821-4125E	K-FLEX FILES 25MM SERIES 2 SST ASST 45/8	032566847
821-4130E	K-FLEX FILES 30MM SERIES 2 SST ASST 45/8	032566857
15306E	K-FLEX 25 21MM	032598222
821-4030E	K-FLEX FILES 30MM SERIES 1 SST ASST 15/4	042598190
15122E	K-FLEX FILES 25MM SZ 08 GRAY PK6	102443196
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	102443197
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	102443198
15134E	K-FLEX FILES 25MM SZ 20 YELLOW PK6	102443199

Part Number	Product Description	Lot Number
14128E	FILES 25MM 08 GREY	042559653
06064E	FILES 25MM 20 YELLOW	042559655
14127E	FILES 21MM 08 GREY	042566050
06151E	FILES 30MM 10 PURPLE	042566051
06062E	FILES 25MM 10 PURPLE	042578351
06063E	FILES 25MM 15 WHITE	042578352
06151E	FILES 30MM 10 PURPLE	042578353
06066E	FILES 25MM 30 BLUE	042578355
06067E	FILES 25MM 35 GREEN	042598745
06005E	FILES 21MM 20 YELLOW	042598746
06003E	FILES 21MM 10 PURPLE	102443191
06062E	FILES 25MM 10 PURPLE	102443193
06063E	FILES 25MM 15 WHITE	102443194
14128E	FILES 25MM 08 GREY	102443195
62250E	FILES 06 25 MM	102452904
06062E	FILES 25MM 10 PURPLE	102455801
06063E	FILES 25MM 15 WHITE	102455805
14128E	FILES 25MM 08 GREY	102455806
06003E	FILES 21MM 10 PURPLE	102463962
06004E	FILES 21MM 15 WHITE	112468733
06062E	FILES 25MM 10 PURPLE	112468734
06062E	FILES 25MM 10 PURPLE	112468735
06062E	FILES 25MM 10 PURPLE	112468736
06063E	FILES 25MM 15 WHITE	112468737
06063E	FILES 25MM 15 WHITE	112468738
06064E	FILES 25MM 20 YELLOW	112468740
06065E	FILES 25MM 25 RED	112468741
06066E	FILES 25MM 30 BLUE	112468742
06151E	FILES 30MM 10 PURPLE	112468745
06152E	FILES 30MM 15 WHITE	112468746
14127E	FILES 21MM 08 GREY	112468747
14128E	FILES 25MM 08 GREY	112468748
14129E	FILES 30MM 08 GREY	112468749

Part Number	Product Description	Lot Number
60144E	K-FLEX FILES 25MM SZ 06 PINK PK/6	102443200
15138E	K-FLEX 25 25MM	102452902
15294E	K-FLEX FILES 21MM SZ 10 PURPLE PK6	102452903
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	102455807
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	102455808
15122E	K-FLEX FILES 25MM SZ 08 GRAY PK6	102459625
15134E	K-FLEX FILES 25MM SZ 20 YELLOW PK6	102459626
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	112468750
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	112468751
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	112468752
15126E	K-FLEX FILES 25MM SZ 10 PURPLE PK6	112468753
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	112468754
15130E	K-FLEX FILES 25MM SZ 15 WHITE PK6	112468755
15142E	K-FLEX FILES 25MM SZ 30 BLUE PK6	112468798
15146E	K-FLEX FILES 25MM SZ 35 GREEN PK6	112468803
15150E	K-FLEX FILES 25MM SZ 40 BLACK PK6	112468812
15206E	K-FLEX FILES 30MM SZ 08 GRAY PK6	112468818
15210E	K-FLEX FILES 30MM SZ 10 PURPLE PK6	112468826
15214E	K-FLEX 15 30MM	112468835
15218E	K-FLEX 20 30MM	112468844
15290E	K-FLEX FILES 21MM SZ 08 GRAY PK6	112468851
15298E	K-FLEX FILES 21MM SZ 15 WHITE PK6	112468859
15302E	K-FLEX FILES 21MM SZ 20 YELLOW PK6	112468865
60120E	K-FLEX FILES 21MM SZ 06 PINK PK/6	112468874
15174E	K-FLEX FILES 25MM SZ 80 BLACK PK6	122463440
15242E	K-FLEX FILES 30MM SZ 50 YELLOW PK6	122463441
15250E	K-FLEX FILES 30MM SZ 60 BLUE PK6	122463442
15322E	K-FLEX FILES 21MM SZ 45 WHITE PK6	122463443
15326E	K-FLEX FILES 21MM SZ 50 YELLOW PK6	122463444
15334E	K-FLEX FILES 21MM SZ 60 BLUE PK6	122463445
15154E	K-FLEX 45 25MM	122467662
15158E	K-FLEX FILES 25MM SZ 50 YELLOW PK6	122467663
15162E	K-FLEX FILES 25MM SZ 55 RED PK6	122467664

Part Number	Product Description	Lot Number
06022E	FILES 21MM 55 RED	122463435
06085E	FILES 25MM 70 GREEN	122463438
06005E	FILES 21MM 20 YELLOW	122467546
06014E	FILES 21MM 25 RED	122467547
06015E	FILES 21MM 30 BLUE	122467548
06016E	FILES 21MM 35 GREEN	122467549
06019E	FILES 21MM 40 BLACK	122467550
06020E	FILES 21MM 45 WHITE	122467551
06021E	FILES 21MM 50 YELLOW	122467552
06023E	FILES 21MM 60 BLUE	122467553
06024E	FILES 21MM 70 GREEN	122467554
06067E	FILES 25MM 35 GREEN	122467559
06069E	FILES 25MM 40 BLACK	122467560
06074E	FILES 25MM 45 WHITE	122467561
06075E	FILES 25MM 50 YELLOW	122467562
06076E	FILES 25MM 55 RED	122467563
06077E	FILES 25MM 60 BLUE	122467564
06099E	FILES 25MM 80 BLACK	122467565
06153E	FILES 30MM 20 YELLOW	122467635
06155E	FILES 30MM 25 RED	122467636
06156E	FILES 30MM 30 BLUE	122467637
06157E	FILES 30MM 35 GREEN	122467638
06159E	FILES 30MM 40 BLACK	122467639
06160E	FILES 30MM 45 WHITE	122467640
11346E	HEDSTROM FILES 25MM 25	022512987
11347E	HEDSTROM FILES 25MM 30	022512988
11359E	HEDSTROM FILES 25MM 15-40 ASST	022539997
12787E	HEDSTROM FILES 21MM 50	022539998
12793E	HEDSTROM FILES 21MM 60	022539999
12796E	HEDSTROM FILES 21MM 70	022540000
12832E	HEDSTROM FILES 30MM 45	022540001
12835E	HEDSTROM FILES 30MM 50	022540002
12838E	HEDSTROM FILES 30MM 55	022540003
12841E	HEDSTROM FILES 30MM 60	022540004
11344E	HEDSTROM FILES 25MM 15	022587544
11345E	HEDSTROM FILES 25MM 20	022587545

Part Number	Product Description	Lot Number
15166E	K-FLEX FILES 25MM SZ 60 BLUE PK6	122467665
15170E	K-FLEX FILES 25MM SZ 70 GREEN PK6	122467666
15222E	K-FLEX FILES 30MM SZ 25 RED PK6	122467667
15226E	K-FLEX FILES 30MM SZ 30 BLUE PK6	122467668
15230E	K-FLEX FILES 30MM SZ 35 GREEN PK6	122467669
15234E	K-FLEX FILES 30MM SZ 40 BLACK PK6	122467670
15238E	K-FLEX FILES 30MM SZ 45 WHITE PK6	122467671
15246E	K-FLEX FILES 30MM SZ 55 RED PK6	122467672
15254E	K-FLEX FILES 30MM SZ 70 GREEN PK6	122467673
15258E	K-FLEX FILES 30MM SZ 80 BLACK PK6	122467674
15306E	K-FLEX 25 21MM	122467675
15310E	K-FLEX FILES 21MM SZ 30 BLUE PK6	122467676
15314E	K-FLEX FILES 21MM SZ 35 GREEN PK6	122467677
15318E	K-FLEX FILES 21MM SZ 40 BLACK PK6	122467678
15330E	K-FLEX FILES 21MM SZ 55 RED PK6	122467679
60193E	K-FLEX FILES 30MM SZ 06 PINK PK6	122467695
06033E	REAMERS 21MM 30 (BLUE)	012579172
62252E	REAMER, 21MM, SIZE .06	032598192
821-2025E	REAMERS 25MM #15-40 ASST.	032598196
06038E	REAMERS 21MM SZ 40 BLACK PK6	032598201
14134E	REAMERS 25MM 08 (GREY)	032598219
61433E	REAMERS 30MM SZ 25 RED PK/6	032598223
06110E	REAMERS 25MM 10 (PURPLE)	112468743
06111E	REAMERS 25MM 15 (WHITE)	112468744
06031E	REAMERS 21MM 20 (YELLOW)	122463436
06032E	REAMERS 21MM 25 (RED)	122463437
14133E	REAMERS 21MM 08 (GREY)	122463439
06029E	REAMERS 21MM 10 (PURPLE)	122467555
06030E	REAMERS 21MM 15 (WHITE)	122467556
06033E	REAMERS 21MM 30 (BLUE)	122467557
06034E	REAMERS 21MM 35 (GREEN)	122467558
06112E	REAMERS 25MM 20 (YELLOW)	122467629
06120E	REAMERS 25MM 25 (RED)	122467630
06126E	REAMERS 25MM 30 (BLUE)	122467631
06127E	REAMERS 25MM 35 (GREEN)	122467632
06131E	REAMERS 25MM 40 (BLACK)	122467633

Part Number	Product Description	Lot Number
11348E	HEDSTROM FILES 25MM 35	022587546
11349E	HEDSTROM FILES 25MM 40	022587547
11344E	HEDSTROM FILES 25MM 15	032512354
11359E	HEDSTROM FILES 25MM 15-40 ASST	032566850
12769E	HEDSTROM FILES 21MM 20	032598214
12787E	HEDSTROM FILES 21MM 50	032598215
11359E	HEDSTROM FILES 25MM 15-40 ASST	042524392
11359E	HEDSTROM FILES 25MM 15-40 ASST	042598212
11347E	HEDSTROM FILES 25MM 30	102463964
11344E	HEDSTROM FILES 25MM 15	112468887
11344E	HEDSTROM FILES 25MM 15	112468894
11344E	HEDSTROM FILES 25MM 15	112468899
11345E	HEDSTROM FILES 25MM 20	112468906
11345E	HEDSTROM FILES 25MM 20	112468911
11346E	HEDSTROM FILES 25MM 25	112468913
11346E	HEDSTROM FILES 25MM 25	112468914
11352E	HEDSTROM FILES 25MM 55	122463447
11353E	HEDSTROM FILES 25MM 60	122463448
12817E	HEDSTROM FILES 30MM 20	122463449
12820E	HEDSTROM FILES 30MM 25	122463450
12823E	HEDSTROM FILES 30MM 30	122463451
13292E	HEDSTROM FILES 30MM 45-80 ASST	122463452
11348E	HEDSTROM FILES 25MM 35	122467641
11349E	HEDSTROM FILES 25MM 40	122467642
11350E	HEDSTROM FILES 25MM 45	122467643
11351E	HEDSTROM FILES 25MM 50	122467644
11354E	HEDSTROM FILES 25MM 70	122467645
11359E	HEDSTROM FILES 25MM 15-40 ASST	122467646
11360E	HEDSTROM FILES 25MM 45-80 ASST	122467647
12766E	HEDSTROM FILES 21MM 15	122467648
12769E	HEDSTROM FILES 21MM 20	122467649
12772E	HEDSTROM FILES 21MM 25	122467650
12775E	HEDSTROM FILES 21MM 30	122467651

Part Number	Product Description	Lot Number
06132E	REAMERS 25MM 45 (WHITE)	122467634
14134E	REAMERS 25MM 08 (GREY)	122467661
62253E	REAMER, 25MM, SIZE .06	122467697
821-3021E	TRIPLE-FLEX FILES 21MM S1 SST ASST 15/40 PK/6	012567698
821-3025E	TRIPLE-FLEX FILES 25MM S1 SST ASST 15/40 PK/6	012567699
821-3125E	TRIPLE-FLEX FILES 25MM S2 SST ASST 45/80 PK/6	012567701
25935E	TRIPLE-FLEX FILES 21MM SZ 40 BLACK PK6	032566859
25949E	TRIPLE-FLEX FILES 25MM SZ 35 GREEN PK6	032566861
25936E	TRIPLE-FLEX FILES 21MM SZ 45 WHITE PK6	032566862
25937E	TRIPLE-FLEX FILES 21MM SZ 50 YELLOW PK6	032566863
25938E	TRIPLE-FLEX FILES 21MM SZ 55 RED PK6	032566864
25939E	TRIPLE-FLEX FILES 21MM SZ 60 BLUE PK6	032566865
25953E	TRIPLE-FLEX FILES 25MM SZ 50 YELLOW PK6	032566866
25955E	TRIPLE-FLEX FILES 25MM SZ 60 BLUE PK6	032598187
25965E	TRIPLE-FLEX FILES 30MM SZ 35 GREEN PK6	032598188
25967E	TRIPLE-FLEX FILES 30MM SZ 40 BLACK PK6	032598189
25959E	TRIPLE-FLEX FILES 30MM SZ 08 GRAY PK6	032598191
25963E	TRIPLE-FLEX FILES 30MM SZ 25 RED PK6	032598195
25932E	TRIPLE-FLEX FILES 21MM SZ 30 BLUE PK6	112463446
821-3130E	TRIPLE-FLEX FILES 30MM S2 SST ASST 45/80 PK/6	112463454
821-3121E	TRIPLE-FLEX FILES 21MM S2 SST ASST 45/80 PK/6	122463453
25927E	TRIPLE-FLEX FILES 21MM SZ 08 GRAY PK6	122467680
25928E	TRIPLE-FLEX FILES 21MM SZ 10 PURPLE PK6	122467681
25929E	TRIPLE-FLEX FILES 21MM SZ 15 WHITE PK6	122467682
25930E	TRIPLE-FLEX FILES 21MM SZ 20 YELLOW PK6	122467683
25931E	TRIPLE-FLEX FILES 21MM SZ 25 RED PK6	122467684
25933E	TRIPLE-FLEX FILES 21MM SZ 35 GREEN PK6	122467685
25943E	TRIPLE-FLEX FILES 25MM SZ 08 GRAY PK6	122467686
25944E	TRIPLE-FLEX FILES 25MM SZ 10 PURPLE PK6	122467687
25945E	TRIPLE-FLEX FILES 25MM SZ 15 WHITE PK6	122467688
25946E	TRIPLE-FLEX FILES 25MM SZ 20 YELLOW PK6	122467689
25947E	TRIPLE-FLEX FILES 25MM SZ 25 RED PK6	122467690
25948E	TRIPLE-FLEX FILES 25MM SZ 30 BLUE PK6	122467691



Via Passanti
174 84018 Scafati (SA) Italy
Phone: 00800 30 32 30 32
KerrDental.com

Part Number	Product Description	Lot Number
12778E	HEDSTROM FILES 21MM 35	122467652
12781E	HEDSTROM FILES 21MM 40	122467653
12784E	HEDSTROM FILES 21MM 45	122467654
12809E	HEDSTROM FILES 21MM 15-40 ASS	122467655
12810E	HEDSTROM FILES 21M 45-80 ASST	122467656
12814E	HEDSTROM FILES 30MM 15	122467657
12826E	HEDSTROM FILES 30MM 35	122467658
12829E	HEDSTROM FILES 30MM 40	122467659
13291E	HEDSTROM FILES 30MM 15-40 ASST	122467660

Part Number	Product Description	Lot Number
25951E	TRIPLE-FLEX FILES 25MM SZ 40 BLACK PK6	122467692
25952E	TRIPLE-FLEX FILES 25MM SZ 45 WHITE PK6	122467693
25961E	TRIPLE-FLEX FILES 30MM SZ 15 WHITE PK6	122467694
821-3030E	TRIPLE-FLEX FILES 30MM S1 SST ASST 15/40 PK/6	122467700