URGENT: FIELD SAFETY NOTIFICATION**11.November 2025****Dear Valued Customer,**

Henry Schein Inc. is initiating this Field Safety Notification to inform you of an important update to the Instructions for Use (IFU) for our **Maxima Diamond Burs**. This update provides enhanced guidance on the proper installation and recommended force application during use, aimed at reducing the risk of bur dislodgement during patient procedures. The updated IFU does not alter the device's design or functionality.

The updated IFU is included with this notification and applies to the devices listed in **Appendix A**.

Risk to Health: Improper installation or excessive force during use may increase the likelihood of bur dislodgement. This can result in potential patient harm, including:

- **Oral injury**
- **Aspiration**
- **Ingestion**

The absence of specific guidance in the previous IFU was identified as a contributing factor to this risk. To date, Henry Schein Inc. has received **one (1) adverse event report** related to this issue.

ACTION REQUIRED:

1. Review inventory to confirm if any of the affected products are in your possession.
2. If impacted devices are located, ensure availability of the updated IFU for all clinicians.
3. Complete, sign, date, and return the enclosed Henry Schein Response Form acknowledging the receipt of notification to **customercare@cliniclands.com**.

Other information: If you have any further questions regarding this matter, please contact **customercare@cliniclands.com**. Thank you for your immediate attention to this matter.

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