

Risks to Users

The technical fault described above may occur in isolated cases with the devices listed, posing a risk of injury to users or third parties. In the event of a malfunction, there is a risk of injury from contact with rotating components.

Measures from the manufacturer, Andreas Hettich GmbH

If you own one of the devices listed above, please take the following steps:

- Close the lid on the centrifuge
- Turn off the centrifuge and disconnect it from the power supply
- Please check whether the handle bar on the lid can be swung upward and the lid opened.
- If you own a device where the lid can be opened, we ask that you refrain from using it until further notice.
- Continued use of the device despite this notice is solely at the operator's own risk.
- Please contact Technical Service immediately via the Support Center to coordinate the next steps.

If you have any questions or concerns, please contact our Support Center or your designated Technical Service representative.

We apologize for any inconvenience this may have caused and thank you for your cooperation.

Best regards,
Andreas Hettich GmbH

i.A. Oskar Döring | PRRC

2026/16106

**Andreas Hettich
GmbH**

Föhrenstr. 12, 78532 Tuttlingen
Postfach 2 60, 78503 Tuttlingen
Telefon: (07461) 705-0
Telefax: (07461) 705-1125