Record type: Field Safety Notice	Record ID: FSN-INT-001-SE	Page: 1(1)
Record Name: FSN for Milou System Version 8.0.75/8.1.29 and previous versions	Version: 1.0	FSCA Record ID: FSCA-INT-001-SE

Signature and Approval

Prepared by: David Dahi, QA/RA <u>2026-02-02</u> Date  Signature	
Reviewed by: Patrik Cederhag, CEO <u>2026-02-02</u> Date  Signature	
Approved by: David Dahi, QA/RA <u>2026-02-02</u> Date  Signature	

Created as CAPA immediate action: CA-016.

This is INTERIM CONTROLLED DOCUMENT, valid until replaced by permanent QMS documentation

<i>Record type:</i> Field Safety Notice	<i>Record ID:</i> FSN-INT-002-SE	<i>Page:</i> 1(5)
<i>Record Name:</i> FSN för Milou System Version 8.0.75/8.1.29 och föregående versioner	<i>Version:</i> 1.0	<i>FSCA Record ID:</i> FSCA-INT-002-SE

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<i>Record type:</i> Field Safety Notice	<i>Record ID:</i> FSN-INT-002-EN	<i>Page:</i> 1(5)
<i>Record Name:</i> FSN for Milou System Version 8.0.75/8.1.29 and previous versions	<i>Version:</i> 1.0	<i>FSCA Record ID:</i> FSCA-INT-002-EN

Date: 2026-02-03

Field Safety Notice to the Market
Milou System

To whom it may concern: Primary users (Midwives and physicians) and Secondary users (Administrators, IT Managers, and IT Technicians)

Local Representative Contact Information (name, email, phone, address etc.)
Patrik Cederhag, Chief Executive Officer (CEO) Tel: +46 708 13 24 59 Medexa Diagnostisk Service AB Patrik.Cederhag@medexa.se Krossverksgatan 11B 216 16 Limhamn www.medexa.se

Record type: Field Safety Notice	Record ID: FSN-INT-002-EN	Page: 2(5)
Record Name: FSN for Milou System Version 8.0.75/8.1.29 and previous versions	Version: 1.0	FSCA Record ID: FSCA-INT-002-EN

Field Safety Notice to the Market

Milou System

Identified Risk

1. Information about affected products	
1.	1. Device Type(s) Software for monitoring CTG data
1.	2. Trade name Milou System
1.	3. Unique Device Identifier(s) (UDI-DI) (01)07300009068302
1.	4. Primary clinical purpose of device(s) The intended use of Milou is to monitor and record physiological data of the fetus and mother obtained from electronic CTG devices
1.	5. Device Model/Catalogue/part number(s) Article number 230
1.	6. Software version Version 8.0.75 and earlier versions. Version 8.1.29 and earlier version
1.	7. Affected serial or lot number range Not applicable
1.	8. Associated devices Not applicable

2. Reason for corrective field safety action	
2.	1. Description of the product issue A deficiency has been identified in Milou System where a delay in the presentation of CTG data can occur on the Milou surveillance screen without an error indication being presented to the end user.
2.	2. Hazard giving rise to the FSCA Potential information related to the health of the fetus and mothers may not be displayed in Milou until later. No patient harm has been reported.
2.	3. Probability of problem arising The probability of problems occurring is assessed as low.
2.	

Record type: Field Safety Notice	Record ID: FSN-INT-002-EN	Page: 3(5)
Record Name: FSN for Milou System Version 8.0.75/8.1.29 and previous versions	Version: 1.0	FSCA Record ID: FSCA-INT-002-EN

	4. Predicted risk to patient/users
	The identified deficiency poses a potential risk for the patient, important information about the health of the fetus and mother is not displayed on the surveillance screen until later. A delay in the presentation of CTG data represents a failure in a time-critical function and can result in misleading clinical information. Such a delay can lead to delayed detection of fetal distress, which in turn can cause a potentially serious deterioration in health.
2.	5. Further information to help characterize the problem
	The error occurs in connection with/after restarting the server, e.g. during patching, and no message about connection loss with the server is presented to the end user.
2.	6. Background on Issue
	The problem can occur during or after a server restart, for example during patching. Clients can then reconnect in a way that results in CTG data not being updated correctly in real time. The cause is considered to be an unintended behavior when multiple clients reconnect simultaneously.
2.	7. Other information relevant to FSCA
	All affected customers will be informed of the necessary corrective security measures in the form of extra vigilance in connection with, and especially after, restarting the server service. Vigilance should be maintained so that time stamps on ongoing CTG curves on the surveillance client match the current time. A patch to eliminate the risk of delay in CTG data will also be developed by the manufacturer and released for the mentioned versions.

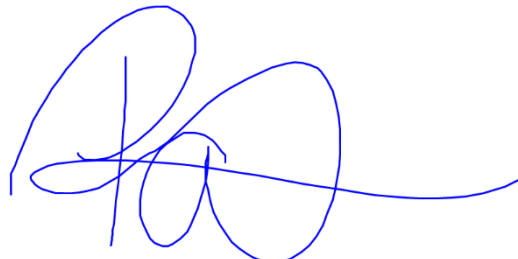
3. Type of Actions to mitigate the risk	
3.	1. Actions to be taken by the User: ☐ Identify Device ☐ Quarantine Device ☐ Destroy Device ☐ On-site device modification/inspection ☐ Follow patient management recommendations ☐ Take note of amendment/reinforcement of Instructions For Use (IFU) ☒ Other ☐ None Provide further details of the action(s) identified: To reduce the risk, surveillance clients should be shut down when the server service is restarted. As soon as the server service is running, monitoring clients can be started. This measure should be implemented until the patched version has been installed.

Record type: Field Safety Notice	Record ID: FSN-INT-002-EN	Page: 4(5)
Record Name: FSN for Milou System Version 8.0.75/8.1.29 and previous versions	Version: 1.0	FSCA Record ID: FSCA-INT-002-EN

3.	2. By when the action should be completed?	As soon as possible
3.	3. Particular considerations for: Not applicable Is follow-up of patients or review of patients' previous results recommended? No	
3.	4. Is customer/user Reply Required? (If yes, specify deadline for return)	Yes, [2026-02-06]
3.	5. Action Being Taken by the Manufacturer: ☐ Product Removal ☐ On-site device modification/inspection ☒ Software upgrade ☐ IFU or labelling change ☐ Other ☐ None Provide further details of the action(s) identified: A patch to prevent CTG data delay will be developed by the manufacturer and released for the aforementioned versions.	
3.	6. When shall the actions be completed	A corrective software update is planned to be provided in Q1 2026. Customers will be notified separately when the update is available.
3.	7. Is the FSN required to be communicated to the patient /lay user?	No
3.	8. If yes, has manufacturer provided additional information suitable for the patient/lay user in a patient/lay or non-professional user information letter/sheet?	
	No	

4. General information		
4.	1. Type of field safety notice to market	New
4.	2. For updated FSN, reference number and date of previous FSN:	Not applicable
4.	3. For Updated FSN, updated information is as follows:	
	Not applicable.	
4.	4. Further advice or information already expected in follow-up FSN?	Not applicable
	5. If follow-up FSN expected, what is the further advice expected to relate to:	
4.	Not applicable	
4.	6. Expected timeline for follow-up FSN:	Not applicable

Record type: Field Safety Notice	Record ID: FSN-INT-002-EN	Page: 5(5)
Record Name: FSN for Milou System Version 8.0.75/8.1.29 and previous versions	Version: 1.0	FSCA Record ID: FSCA-INT-002-EN

4.	7. Manufacturer information (For contact details refer to page 1 of this FSN)	
	a. Company Name:	See page 1
	b. Address:	See page 1
	c. Website address:	See page 1
4.	8. The Competent (Regulatory) Authority of your country has been informed about this communication to customers: Yes	
4.	9. List of attachments/appendices:	Not applicable
4.	10. Name/Signature	Patrik Cederhag Chief Executive Officer (CEO)
		

Transmission of this Field Safety Notice	
	This notice needs to be passed on to all those who need to be aware within your organization or to any organization where the potentially affected devices have been transferred. (As appropriate) Please transfer this notice to other organizations on which this action has an impact. (As appropriate) Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action. Please report all device-related incidents to the manufacturer, distributor or local representative, and the national Competent Authority if appropriate, as this provides important feedback. *

FSN = (Field Safety Notice)

FSCA = (Field Safety Corrective Action)

<i>Record type:</i> Field Safety Notice	<i>Record ID:</i> FSN-INT-002-DK	<i>Page:</i> 1(5)
<i>Record Name:</i> FSN for Milou System Version 8.0.75/8.1.29 og tidligere versioner	<i>Version:</i> 1.0	<i>FSCA Record ID:</i> FSCA-INT-002-DK

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